



Introducing **THEYFIT**, the world's first custom-fit condom with 95 different sizes for your maximum pleasure & comfort.

To confirm your FitKit™ printed correctly, please refer to Step 1 to avoid any uncertainty when using the printed measurements. Disable printer page scaling in case of printing issues, or refer to www.TheyFit.com for further guidance.


Ensure that you are fully aroused before measuring - pornography or asking a partner to measure can be helpful in this matter.

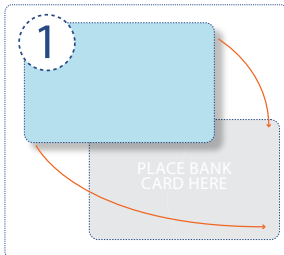


1. Printed correctly? Ensure your FitKit™ has printed to the correct size by placing a bank card onto the illustration below. If your card fits, then so will you.

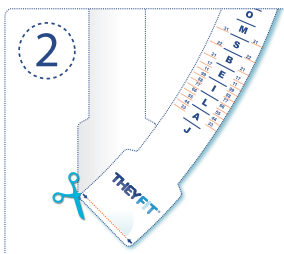

2. Cut out the template; numbers are for width and letters are for length.

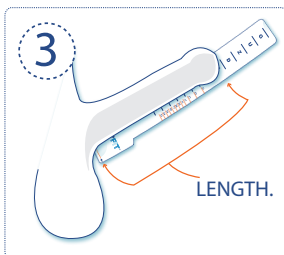

3. To find your length; place the template alongside your firm erection with the bold base line as far as the condom would unroll on the underside. Your length is shown by the letter at the tip of your penis.


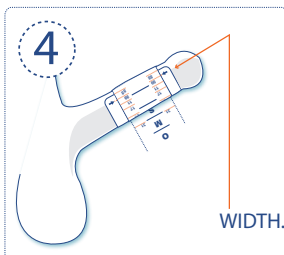


4. To find your width; wrap the template gently around the middle of your firm erection. Where the arrows meet the numbers, your width is shown as that number.



5. Combine your width number & your length letter to create your personal TheyFit® condom code. Visit www.TheyFit.com to find your authorised local TheyFit® retailer.

Please note : your 'custom size' as determined by the FitKit™ is a suggested size. You may want to choose a slightly different size based on your personal preference. If your combination is not shown, to make the best choice, choose a nearby size: better that the condom has more length or less width rather than being too wide or needing more length. A proper fit is both comfortable and held snugly in position. Don't be afraid to try more than one code combination!

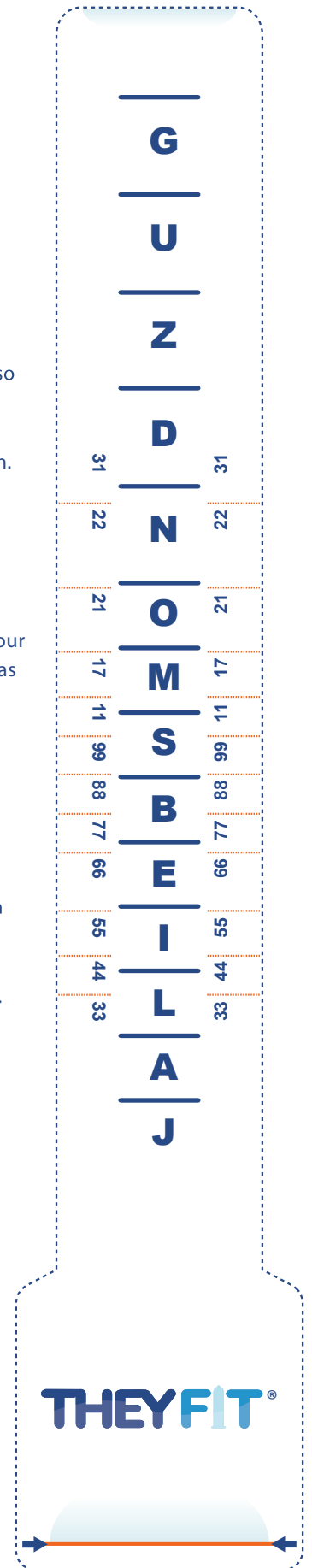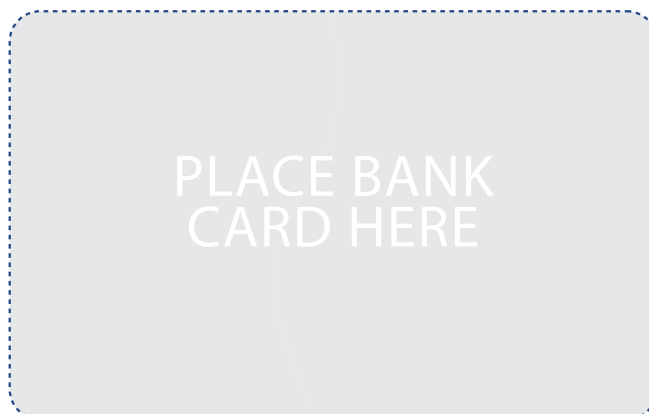

TheyFit®, FitKit™ & the TheyFit® logo are trademarks of TheyFit LLC by © All Rights Reserved. FitKit™ & TheyFit® distribution strictly by authorised representatives only. Warning: TheyFit® might seriously improve your sex life!

JUNE 2014. ENHANCED FitKit™ - VERSION 2014:01 - ALWAYS USE THE MOST UP TO DATE FitKit™ TO MAXIMISE FITTING ACCURACY AND FITTED COMFORT - CHECK THEYFIT.COM FOR MORE DETAILS.