



Council Directive 93/42/EEC
of 14 June 1993 concerning
Medical devices

Bledsoe Achilles Boot Rise Kit

Application Instructions CP020127 Rev K 06/07

TOTAL LEG SUPPORT SYSTEM WITH INFINITE R.O.M. & SHORT APPLICATION TIME



THE BLEDSOE ACHILLES RISE KIT CONTAINS:

- 1 - GRAY INNERSOLE W/ CUTOUT PIECE
- 2 - 18" STRAPS
- 1 - INSTRUCTION SHEET

- 1 - SET OF (4) 10° ACHILLES WEDGES
- 4 - 2" X 2" DOUBLE SIDED HOOK-PIECES

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is generally not intended for patients in excess of 250 lbs (114 kilos). This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

Indications: Indicated for immediate application following injury to or reconstruction of the Achilles tendon.

Contraindications: Contraindicated for unstable fractures or for fractures of the proximal tibia or fibula.

WARNING: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this device, seek medical attention.**

Warranty: This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.bledsoebrace.com or by calling (800) 527-3666.

Manufactured by:

Bledsoe Brace Systems™

2601 Pinewood Drive; Grand Prairie, Texas 75051, USA

Toll Free Tel. **1.888.BLEDSOE** (253.3763)

Or 1.800.527.3666 • Local Tel. 972.647.0884

Local FAX 972.660.5495

International FAX 972.606.0649

www.BledsoeBrace.com

European Authorized Representative:



Emergo Europe

Molenstraat 15

2513 BH, The Hague

The Netherlands

Tel: (+31) 70 345 8570

Fax: (+31) 70 346 7299

ORTHOMEDICO (European Bledsoe Agent)

+32 54 504005

www.orthomedico.be

For product information or questions pertaining to sales or service, please contact your area sales representative or Bledsoe Brace Systems directly.

Bledsoe Achilles Boot Rise Kit Application Instructions



1. REMOVE BOOT INNERSOLE

Peel out the innersole material inside the foot area of the boot.



2. APPLY KIT INNERSOLE

Apply the kit innersole in the Bledsoe Boot leaving 1/8"(.32cm) space on each end of the innersole. Save the cutout piece for step 8.



3. REMOVE FOREFOOT STRAPS

Disconnect the pile to hook fastener and remove the strap from the D-Ring.

NOTE: The old 2" X 2" double-sided hook pieces will be used to attach the forefoot pad to the straps.



4. NEW FOREFOOT STRAPS

Apply the kit straps through the D-Rings and stick back on themselves with the 2" X 2" double-sided hook pieces.

NOTE: Cut off the excess strap after applying the boot to the patient.



5. REMOVE WEDGES


To change or remove the wedges, pry the upper wedge assembly up and save.



6. Peel out each 10° wedge as necessary for patient progress.



7. Replace the upper wedge assembly.

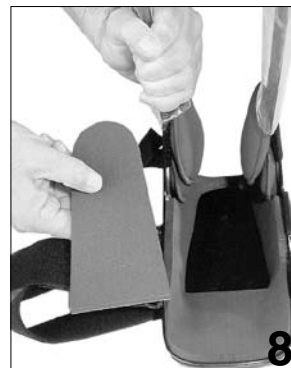

8. REPLACE CUTOUT

After all wedges are removed, the saved cut out piece can be replaced for further rehab.