
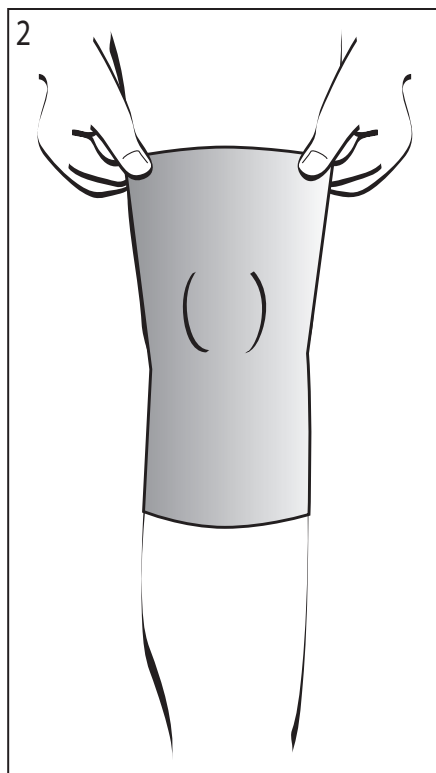


Knee Support Closed Patella

APPLICATION INSTRUCTIONS

CP020411 Rev B 03/12

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

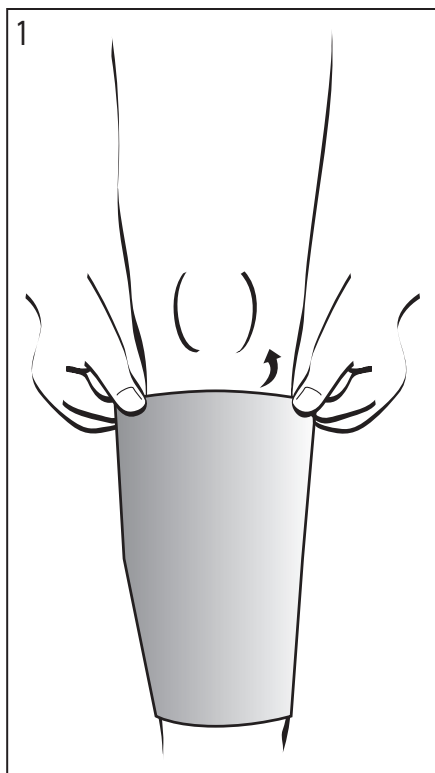

INDICATIONS: Indicated for support of the knee.

CONTRAINDICATIONS: Contraindicated for fractures of the proximal femur or the distal tibia, fibula, and patella.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150F (65 C.) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

CLEANING INSTRUCTIONS: Brace should be hand washed, using warm water and a detergent for delicated fabrics (such as Woolite™). Triple rinse, squeeze and line dry. Do not twist.

WARRANTY: This device is warranted to be free from defects in material and workmanship for a period of 90 days. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the Internet at www.BledsoeBrace.com, or by calling 1.888.BLEDSOE (253.3763).