ENGLISH

- 1 Apply Brace**
Loosen straps and open thigh portion of the brace. Slip the brace on the leg, aligning the hinges with the center of the knee.
- 2 Adjust Patella Buttress**
Adjust the removable patella buttress inside the brace as needed.
- 3 Fit the Brace**
Tension the lateral pull strap located above the knee.
- 4 Secure the Front Thigh Closure**
Den Gurt nach Bedarf für einen gleichmäßigen Sitz anpassen. Adjust the wrap as needed for an even fit. *NOTE: Make sure the hinges fit closely to the knee. If they do not, remove the hinges from the brace bend or flare uprights to achieve proper fit.*
- 5 Securing the Straps**
Starting with the lower strap, loop the strap through the D-ring and pull back to close. Repeat for the upper thigh strap.
- 6 Securing the Lateral Straps**
Secure the lateral patella buttress by pulling on the upper lateral pull strap located on the inside of the thigh. Repeat with the lower lateral pull strap, located on the inside of the calf.
- 7 Inflate Buttress**
Insert the clear end of the black bulb provided into the spout located above the hinge on the outside of the leg. To inflate, squeeze the bulb. To remove air from the buttress, insert the black end of the bulb into the spout and squeeze the bulb.
- 8 Removing the Brace**
Disengage the hook closure and loosen the straps at the calf and the thigh. Slip the brace off the leg.
- 9 Removing the Hinge and Struts**
Disengage the hook material that secures the hinge to the brace. Fold the brace struts as shown, slip the struts out of the sleeves and remove. Reverse this step to re-apply.

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CLEANING INSTRUCTIONS:
Remove hinges from brace. Brace should be hand washed, using warm water and a detergent for delicate fabrics (such as Woolite™). Triple rinse, squeeze, and line dry. Do not twist.

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