

# Elbow Sleeve



## Application Instructions



Council Directive 93/42/EEC  
of 14 June 1993 concerning  
Medical devices

### 1. Apply Brace

Slip brace onto affected arm.

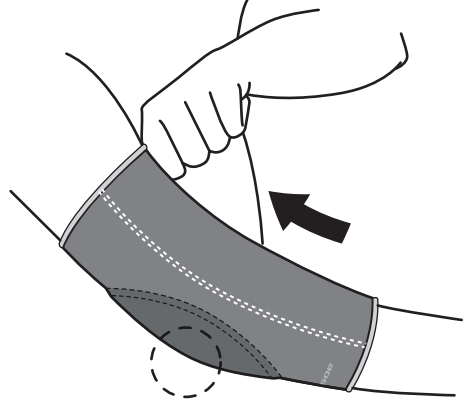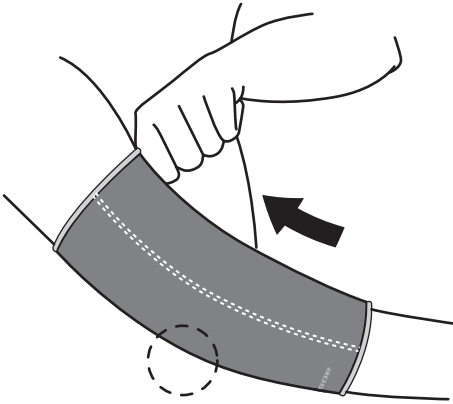


### 2. Position Brace

**For Elbow Sleeve:** Pull brace into position over the elbow.

### For Padded Elbow Sleeve:

Pull brace into position over the elbow; center the padded section of the brace over the center of the elbow.

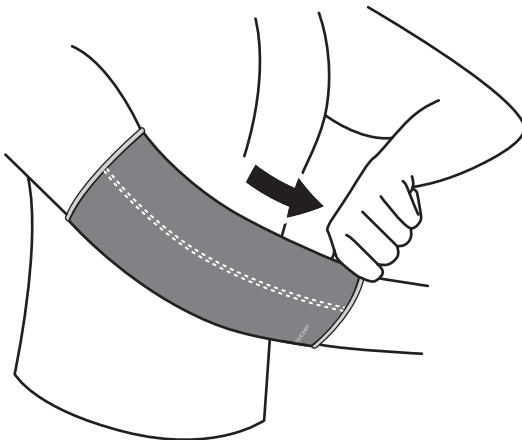

### 3. Remove the Brace

Slide brace off the arm over the hand.

# Elbow Sleeve

## Application Instructions

**Indications:** Elbow strains and sprains, tendonitis and bursitis.

**Warning:** For Single Patient Use Only.

**Caution:** Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

**Caution:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

**Caution:** This device will not prevent or reduce all injuries.

**Caution:** This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65° C.) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

**WARRANTY:** This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at [www.BledsoeBrace.com](http://www.BledsoeBrace.com), or by calling 1-888-BLEDSOE (253- 3763).

© 2012 Medical Technology Inc., dba Bledsoe  
Brace Systems.® All Rights Reserved.  
2601 Pinewood Dr.  
Grand Prairie, TX 75051  
p. 972.647.8004 • f. 972.606.0946  
[www.bledsoebrace.com](http://www.bledsoebrace.com)

EUROPEAN AUTHORIZED REPRESENTATIVE



CP020275 Rev A 08/12