INDICATIONS: Indicated for immediate application to ankles for mild sprains.

CONTRAINDICATIONS: Contraindicated for unstable grade two or three sprains where plantar or dorsi flexion cannot be achieved.

WARNING: Federal Law Restricts this Device to Sale by or on the Order of a Licensed Health Care Practitioner.

WARNING: For Single Patient Use Only.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device will not prevent or reduce all injuries.

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CAUTION: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65° C.) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

WARRANTY: This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.BledsoeBrace.com, or by calling 1-888-BLEDSOE (253-3763).

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M-BRACE

PATIENT INSTRUCTIONS

Council Directive 93/42/EEC of 14 June 1993 concerning Medical devices

CP020195 Rev D 09/12

EUROPEAN AUTHORIZED REPRESENTATIVE

Emergo Europe
Molensstraat 15
2513 BH The Hague
The Netherlands
Tel: (+31) 70 345 8570
Fax: (+31) 70 346 7299

EC

REP

© 2012 Medical Technology Inc., d/b/a
Bledsoe Brace Systems All Rights Reserved
2601 Pinewood; Grand Prairie, Texas 75051, USA
Toll Free Tel: 1.888.BLEDSOE (253.3763)
Local: 972.647.0884 • Local Fax: 972.660.5495
www.BledsoeBrace.com

ENGLISH

*When using the **AIR M-BRACE**, inflate prior to application by placing the inflation tube in the valve at the top of the air bladder. Blow gently.

- 1 Adjust the foam heel pad by centering it evenly under heel.
- 2 Wrap the lower hook closure strap first for alignment.
- 3 Wrap the upper strap around to the hook and adjust for comfort. Be certain the side shells are centered along the ankle and leg.
- 4 Apply pressure to the sides of the therapeutic gel bag to allow an even flow of pressure within the plastic shell. Tighten lower and upper straps until secure and comfortable.
- 5 Adjust the vertical side straps by taking one in each hand and pulling upwards until desired fit is achieved.
- 6 To remove, undo the upper and lower straps, slide heel portion to rear of ankle and release side shells.

GEL COLD THERAPY

Remove therapeutic gel bag from ankle support system. Place in freezer for 2 hours, then place gel bag back into the ankle support and follow fitting instructions. Absorbent sock should be worn before ankle support is fitted. Do Not place entire ankle support into freezer.

PRECAUTIONS

- Patients should always consult their physician or therapist for proper therapy instructions prior to using the therapeutic ankle support system.
- Always wear and absorbent sock before cold therapy is applied.
- The entire ankle support system can be hand washed with mild soap and water. Do not apply artificial heat to dry.
- Only freeze the therapeutic bag. **DO NOT** place entire system in freezer.

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