


Night Splint


APPLICATION INSTRUCTIONS

CP020431 Rev B 02/12


This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.

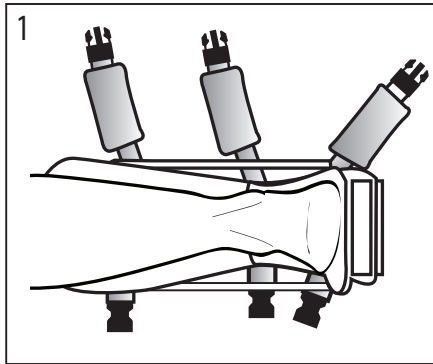


1. Position Foot
Position affected foot inside the splint.

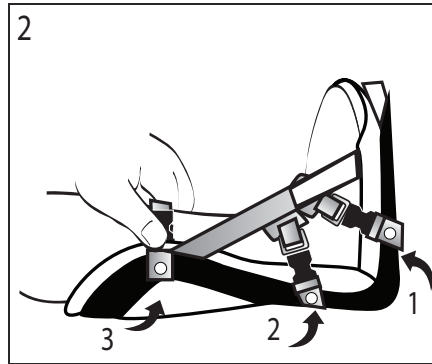

2. Secure Straps
Secure the straps using the buckle fasteners.

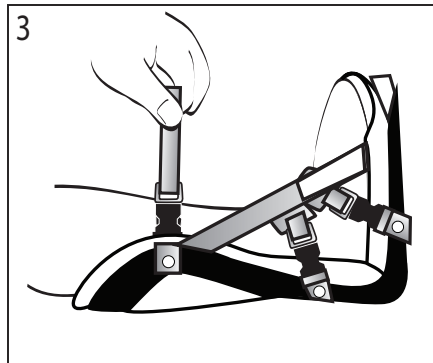

3. Adjust Straps
Adjust straps using the hook and pile fasteners.

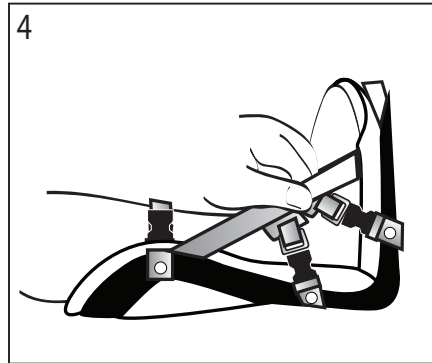

4. Adjustment Straps
Adjust the diagonal straps and secure using the fasteners.

INDICATIONS: Indicated for plantar fasciitis to aid in stretching the Achilles Tendon, maintaining the foot in dorsi-flexed position, and maintaining the length and/or position of the plantar fascia while in nonweight-bearing conditions.

CONTRAINDICATIONS: Contraindicated for use by neuropathic diabetic patients with ulcerations on the plantar surface of the foot.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150F (65 C.) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

WARRANTY: This device is warranted to be free from defects in material and workmanship for a period of 90 days. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the Internet at www.BledsoeBrace.com, or by calling 1.888.BLEDSOE (253.3763).




Council Directive 93/42/EEC
of 14 June 1993 concerning Medical devices

©2012 Medical Technology Inc., d/b/a Bledsoe Brace Systems™.
All Rights Reserved.
2601 Pinewood, Grand Prairie, Texas, 75051, USA
Toll Free Tel: 1.888.BLEDSOE (253.3763) • 1.800.527.3666
Local: 972.647.0884 • Local Fax: 972.660.5495
International Fax: 972.606.0649

www.BledsoeBrace.com

EUROPEAN AUTHORIZED REPRESENTATIVE

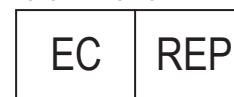

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands
Tel: (+31) 70 345 8570
Fax: (+31) 70 346 7299