


Volare Spinal System


APPLICATION INSTRUCTIONS

CP020240 Rev A 09/08


This device is offered for sale by or on the order of a physician or other qualified medical professional.
This device is not intended for reuse on a second patient.

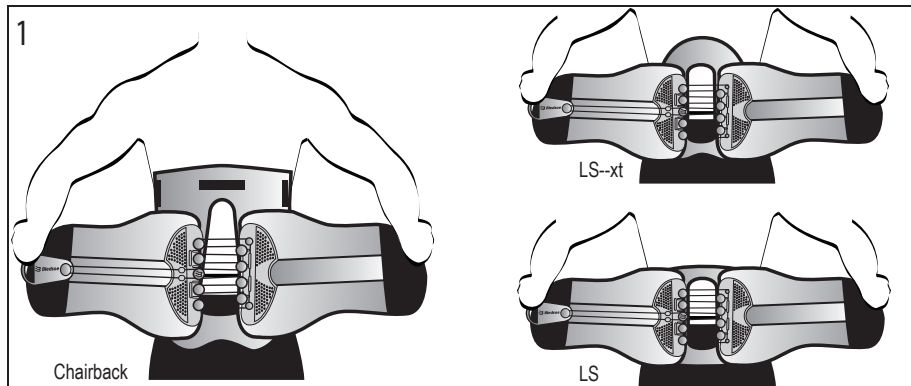

1. Open and extend the corset. The corset should be fully extended each time it is applied. The panel with the handle should be on the left. Center the two ends of the pulley device on the lower back.

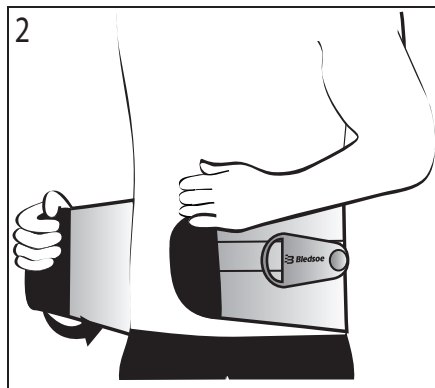

2. Wrap the sides around the abdomen with the right side overlapping the left.

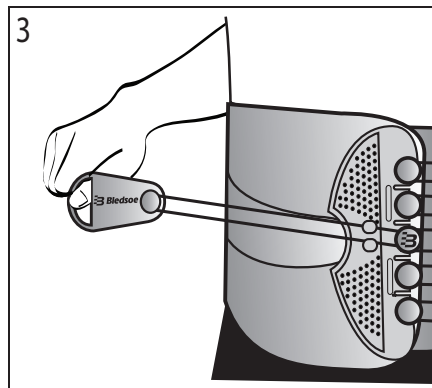

3. Pull the handle to the desired position. When the appropriate amount of tension is reached, place the handle on the Velcro strap to secure.

INDICATIONS: Indicated for back aches and pains, unstable Para-Spinal muscle control, when post surgery brace is no longer required. Also provides continued protection for physical activities, physical rehabilitation, chiropractic use for better adjustment maintenance and pain relief.

WARNINGS: Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this brace, discontinue use and seek medical attention.

WARRANTY: This device is warranted to be free from defects in material and workmanship for a period of 90 days; and for the life of the brace on the pulley system. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.BledsoeBrace.com, or by calling 1.888.BLED SOE (253.3763).



Council Directive 93/42/EEC
of 14 June 1993 concerning Medical devices

MANUFACTURED BY:
BLED SOE BRACE SYSTEMS™
2601 Pinewood; Grand Prairie, Texas, 75051, USA
Toll Free Tel: 1.888.BLED SOE (253.3763) • 1.800.527.3666
Local: 972.647.0884 • Local Fax: 972.660.5495
www.BledsoeBrace.com

EUROPEAN AUTHORIZED REPRESENTATIVE

EMERGO EUROPE Molenstraat 15 2513 BH, The Hague The Netherlands Tel: (+31) 70 345 8570 Fax: (+31) 70 346 7299	
EC	REP