

## 7 HINGE INSTRUCTIONS


### 8a SET RANGE-OF-MOTION (ROM)

Depress Extension Button and slide to desired ROM setting. Depress Flexion Button and slide to desired ROM setting. *NOTE: ROM settings automatically lock into place.*

### 8b LOCK HINGE

The Lock Hinge allows the brace to be locked at a specific setting. The hinge can be locked in 10° increments ranging from -10 to 100.

Bend the arm to desired setting and engage lock. This will lock the brace in place independent of ROM settings. Make sure ROM settings do not interfere with desired lock setting.

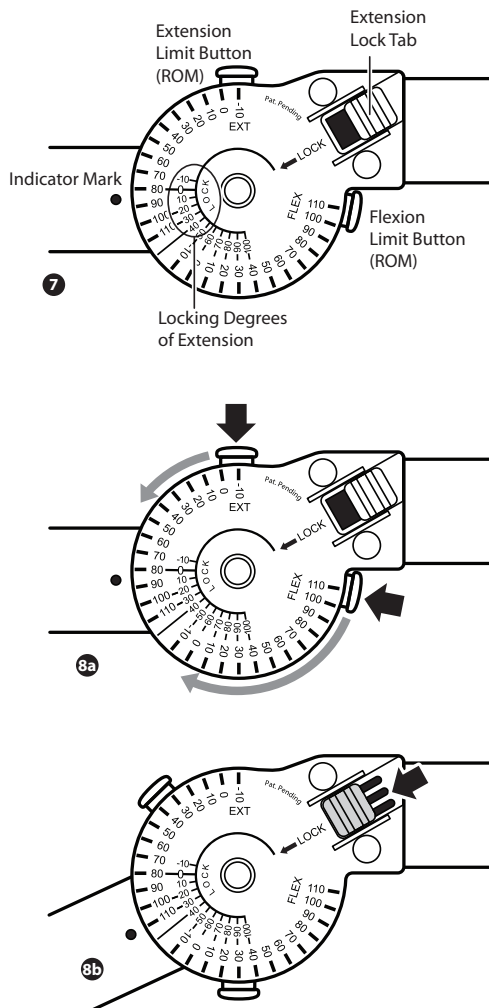

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

**Indications:** Indicated for immediate application following injury or operative procedures to the elbow, or for stable or internally fixed fractures of the distal humerus or proximal to middle radius or ulna where locked or limited motion control of the elbow is desired.

**Contraindications:** Contraindicated for unstable fractures or for fractures of the proximal humerus, distal radius or ulna.


**WARNING:** Do not leave this device in a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.

**WARRANTY:** This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at [www.BledsoeBrace.com](http://www.BledsoeBrace.com), or by calling 1.800.527.3666.



# WEE BOW ELBOW BRACE

## APPLICATION INSTRUCTIONS

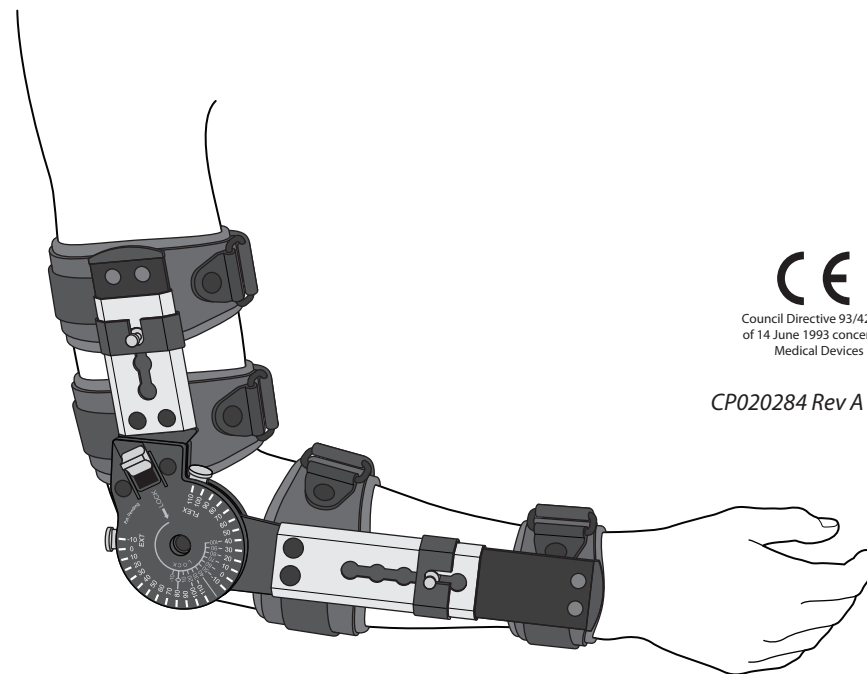

Council Directive 93/42/EEC  
of 14 June 1993 concerning  
Medical Devices

CP020284 Rev A 06/13

© 2013 Medical Technology Inc. d/b/a  
Bledsoe Brace Systems. All Rights Reserved.  
2601 Pinewood Drive, Grand Prairie, Texas, 75051, USA  
Toll Free Tel. 1.888.BLED SOE (253.3763)  
Or 1.800.527.3666 • Local 972.647.0884  
Local FAX 972.660.5495  
[www.BledsoeBrace.com](http://www.BledsoeBrace.com)

EUROPEAN AUTHORIZED REPRESENTATIVE

EC

REP

EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands  
Tel: (+31) 70 345 8570  
Fax: (+31) 70 346 7299

## 1 UNLOCK THE HINGE

The WeeBow Elbow Brace is packaged in the locked position. Slide the Lock button away from the hinge to unlock the brace.

## 2 RELEASE ALL FOUR STRAPS

Pull each strap through the D-ring and attach it back on itself to prevent entanglement.

## 3 POSITION THE BRACE

Position the hinge over the lateral humeral epicondyle (outer elbow).

## 4 SET THE ARM LENGTH

While the hinge is aligned with the elbow (see Step 3), make length adjustments to the forearm.

**4a** Slide the locking clip over to expose the length adjustment button.

**4b** Depress the length adjustment button and extend the plate to the desired length (forearm).

**4c** Secure the locking clip by sliding it over the length adjustment button to lock the button in place.

**4d** Repeat the same process to adjust the humeral arm length.

## 5 APPLY THE BRACE

Position the brace, be sure that the hinge is centered over the elbow joint. Form the black aluminum plates around the forearm and upper arm.

## 6 SECURE THE STRAPS

Beginning with the strap just below the hinge (1), fasten each strap by sliding it through the D-ring and pulling the strap back upon itself to the desired tension and attaching it to the double-sided hook. Fasten the strap closest to the hand next (2), then the strap just above the hinge (3), and last the strap at

