INDICATIONS: Indicated for post-injury or post-operative application to grade 1, 2, or 3 ankle sprains, and stable or internally fixed fractures of the foot or ankle. Also indicated for treatment of stable or internally fixed fractures or fractures with bridging callous of the middle to distal tibia or fibula.

CONTRAINDICATIONS: Contraindicated for unstable fractures or for fractures of the proximal tibia or fibula.

WARNING: Federal Law Restricts this Device to Sale by or on the Order of a Licensed Health Care Practitioner.

WARNING: For Single Patient Use Only.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device will not prevent or reduce all injuries.

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CAUTION: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65° C.) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

WARRANTY: This device is warranted to be free from defects in material and workmanship for a period of 90 days. These warranties apply to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the internet at www.BledsoeBrace.com, or by calling 1-888-BLEDSOE (253- 3763).

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bledsoe

WEE WALKER APPLICATION INSTRUCTIONS

Council Directive 93/42/EEC
of 14 June 1993 concerning
Medical devices

CP020133 Rev K 10/12

© 2012 Medical Technology Inc., d/b/a
Bledsoe Brace Systems All Rights Reserved
2601 Pinewood; Grand Prairie, Texas 75051, USA
Toll Free Tel: 1.888.BLEDSOE (253.3763)
Local: 972.647.0884 • Local Fax: 972.660.5495
www.BledsoeBrace.com

EUROPEAN AUTHORIZED REPRESENTATIVE

Emergo Europe
Molenstraat 15
2513 BH The Hague
The Netherlands
Tel: (+31) 70 345 8570
Fax: (+31) 70 346 7299

EC

REP

ENGLISH

1 Apply Calf Wrap

Slip the foot into the bootie with the toes under the toe piece. Close the forefoot cover over the foot. Trim the excess leaving 2"-3" (5cm-7.6cm) overlap.

2 Wrap the calf section around the leg, and trim the excess leaving 2"-3" (5cm-7.6cm) overlap. Close the hook to pile fastener at the front. Remove the forefoot strap from the uprights. Pull center of strap to loosen.

Note: Do not pull the strap out of the slots on the side of the boot.

3 Slip Foot Into Boot

Slip the foot into the boot with the plastic covers over the uprights to prevent the hook to pile engagement.

4 Position the uprights in line with the leg and remove the plastic covers to engage the hook to pile fastener.

5 Fasten Straps

Pull the ends of the forefoot strap to tighten. Cross each end of the strap over the forefoot and attach to the hook to pile fastener on the opposite uprights.

6 Starting at the ankle, wrap the strap ends securely forward around the sides of the leg to interlock the strap with the uprights **Note:** Bottom strap overlaps part of the forefoot strap to help secure the forefoot strap in place. Repeat this step for remaining strap.

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