Wraptor Ankle Brace

Council Directive 93/42/EEC
of 14 June 1993 concerning
Medical devices

English

1. Position the Brace

Slip foot into the brace ensuring the tongue is between the top of the foot and the laces. (A)

2. Tighten Laces

Tighten and secure laces. (B)

NOTE: For braces with Speed Lacers, pull tabs and secure.

3. Invert Foot

Invert foot and wrap the upper Lateral Strap (outside strap) in a figure 8 around the foot and pull vertically until firm medial (inner) tension is achieved. Secure using hook and loop. (C)

4. Evert Foot

Evert foot and wrap the upper Medial Strap (inside strap) in a figure 8 around the foot and pull vertically until firm lateral (outer) tension is achieved. Secure using hook and loop. (D)

5. Wrap and Secure Calf Strap

Wrap upper strap around the lower calf and secure using the hook and loop fastener. (E)

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Indications: Indicated for chronic ankle instability, sprains or strains, and protection against inversion and eversion ankle injuries.

Contraindications: Contraindicated for fractures of the foot and ankle.

Caution: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention. Caution: This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

Caution: This device has been developed for the indications listed above. Do not leave this device in a car on a hot day, or in any other place where the

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temperature may exceed 150°F (65° C) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

WARRANTY: This device is warranted to be free from defects in material and workmanship for a period of 90 days. This warranty applies to devices that have not been modified or subjected to misuse, abuse or neglect. This device is expressly subject to Bledsoe's Product Line Limited Warranty. Copies of this limited warranty can be obtained from your medical professional, via the Internet at www.BledsoeBrace.com, or by calling 1.888.BLED SOE (253.3763).

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Bledsoe Brace Systems All Rights Reserved