HUMANTECHNIK

sonumaxx

D

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Seite 7

GB

Operation Instructions

Page 11

F

Mode d'emploi

Page 15

NL

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Pagina 19

I

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Pagina 23

E

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Página 27

SE

Bruksanvisning

Sida 31

NO

Betjeningsveiledning

Side 35

FIN

Käyttöohje

Sivu 39

PL

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


Strona 43



HU

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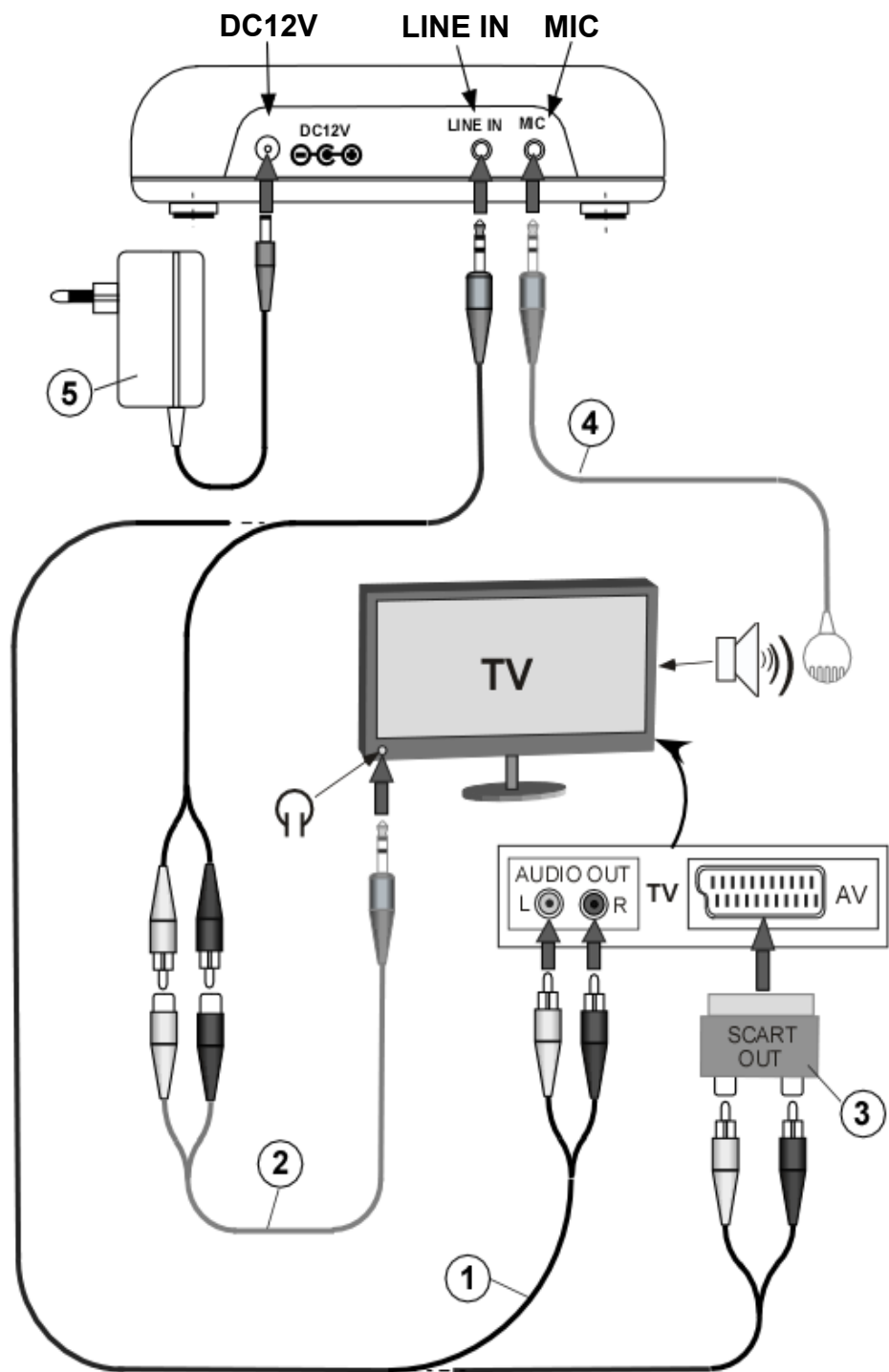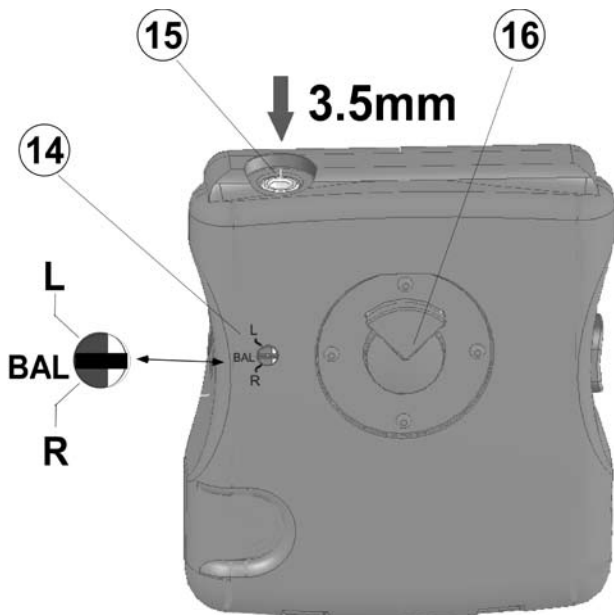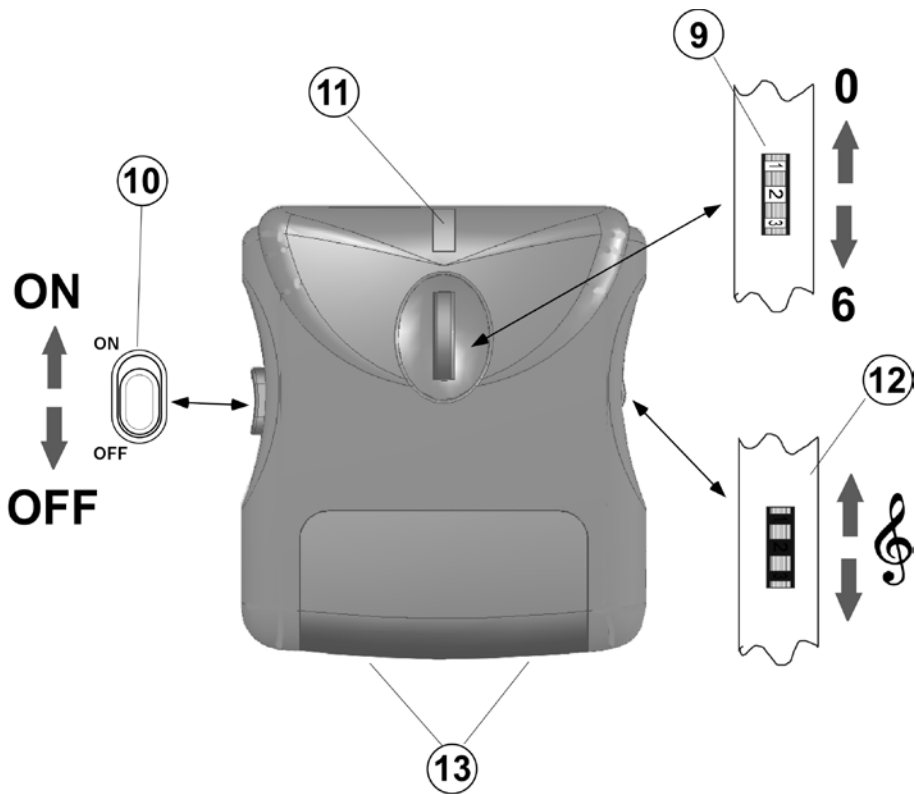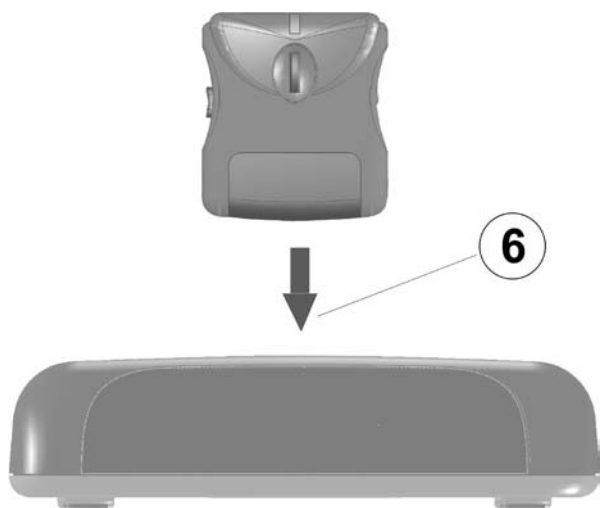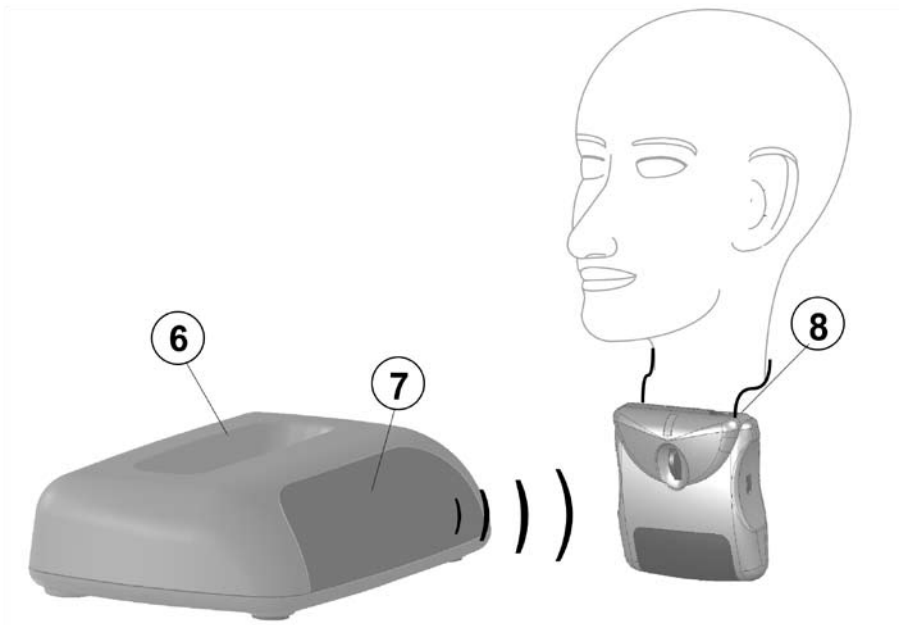Oldal 47

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TV listening system with infrared transmission

Safety Precautions

- Before you start up and use this device, please be sure to read through all these instructions carefully.
- Keep these operating instructions in a safe place so that other users have access to them any time they might need them.
- The device and the power supply unit in particular are designed for use in dry indoor rooms only. Proper operation is guaranteed at a room temperature of 10 °C to 35 °C.
- Do not place the infrared transmitter near heating vents or radiators and avoid any exposure to direct sunlight. Direct sunlight may have a negative effect on infrared sound transmission.
- Always handle the device and any accessories with care and set them up in as dust-free and clean an environment as possible.
- Clean the transmitter and the receiver regularly. The soft silicone earpieces should be cleaned regularly to always ensure proper hygiene. To replace them, please use the soft silicone earpieces that are sold as special accessories.
- Varnishes and furniture polishes can react aggressively with the rubber feet on the transmitter and leave unsightly marks on the surfaces of your furniture.
- Make sure that you only ever use the power supply unit provided with the device to supply electricity to the transmitter.
- Should you not plan to use the system for the period of several days, we recommend that you pull the power supply unit out of the wall outlet.

Important note:

- Your receiver can be set to a high volume. To protect your hearing or the hearing of people with normal hearing, particularly of children, from any damage caused by excessively loud volume, please observe the following information:
 1. Set the volume to low before you put on the receiver.
 2. While using the device, set the volume to a level that is comfortable for you.
- If you would like to use this device while using a cardiac pacemaker or any other medical device, please observe the following information:
 1. Before you use this product together with a pacemaker, consult your doctor or the manufacturer of the pacemaker to make sure that none of the functions of the pacemaker will be affected.
 2. If you use a cardiac pacemaker or any other medical device, be sure to follow the safety precautions/safety guidelines provided by your doctor or the manufacturer of the pacemaker before you use the »sonumaxx«.

Standard components

Please check if all listed items are included:

- »sonumaxx«- Transmitter and Receiver
- Carrying cord with clip
- Screwdriver
- 12 V plug-in PSU
- Connection cable
- Operating Instructions
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Connecting the infrared transmitter

- Plug in the audio connection cable ① that comes with the device into the “LINE IN” connection jack of the transmitter and the other end into the connection jack of your sound source (e.g. the TV).

Other connection adapters are available optionally: a 3.5-mm stereo cinch jack adapter ② which can be connected to a headphone jack, or a SCART connector ③, which can be plugged into the SCART jack of the audio source (marked with AV).

If your sound source is not equipped with a headphone jack or any other connection options, then you can clip a microphone ④ onto the sound source. This microphone is then plugged into the corresponding MIC jack on the transmitter. The microphone is sold separately.

- Connect the power supply unit that comes with the device ⑤ to the transmitter using the “DC12V” connection plug. Then plug the power supply unit itself into a 230V electrical wall outlet.
- Set the under-the-chin receiver into the charging tray ⑥ of the transmitter without using any excessive force. A red LED will light up to indicate that the device is being charged up.
- Before using the receiver for the first time, you should charge the batteries for a period of 24 hours.

Setting up or adjusting the transmitter

- Place the transmitter close to the sound source in a suitable position. There must be a clear line of sight between the front of the transmitter (polished front surface ⑦) and the receiver ⑧. The polished surface of the transmitter must be aimed at the front of the transmitter.

How to attach the carrying cord

- The receiver includes a carrying cord, which can be attached to the rear side of the device.
To attach the carrying cord to the receiver, place the clip on the hook ⑩ on the back of the device. Now turn the clip 180° until it snaps into place. A belt clip is also available optionally.

Connecting headphones / the tele loop

Headphones or a tele loop equipped with a 3.5mm stereo cinch plug can be connected to the jack ⑨ at the receiver.

Headphones and tele loops are sold separately.

Using the receiver

- Use the volume control ⑨ to initially set the receiver to a low volume to protect your hearing. Turn the volume control in the direction of the “6” to increase the volume and in the direction of the “0” to decrease the volume.
- Use the sliding switch ⑩, on the left-hand side of the receiver to switch it on (switch setting: “ON”). The red LED ⑪ will light up to indicate that the receiver has been switched on. Make sure that the sound source connected is switched on and adjust the volume on the receiver to a level that is comfortable for you.
- Use the tone control button ⑫ on the right-hand side of the receiver to adjust the tone. Turn the tone control button upwards to amplify high tones and down to amplify low tones.
- Use the balance control button ⑬ on the rear side of the receiver to adjust the volume balance. Set the balance so that you hear equally well with both ears.

Battery recharging contacts

- We recommend that you keep the battery contacts ⑬ clean at all times so that the battery can always recharge properly and fully (using the recharging contacts in the recharging tray of the transmitter and the recharging contacts on the receiver).

Replacing the battery

- If it becomes necessary to replace the rechargeable batteries after you have used the receiver for a considerable time, please have this done by the specialist from whom you purchased the device.

Warranty

This product is very safe to operate and provides highly reliable service. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw away the packaging. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the complete warranty card is returned accompanied by a copy of the dealer's invoice/till receipt. **Always specify the product number in any event.**

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Rechargeable batteries

The rechargeable batteries that come with the device can be recycled. Please dispose of these rechargeable batteries in receptacles designed for that purpose or return them to a retail outlet. Dispose only of those batteries that have been completely discharged to ensure environmental protection.

Technical data

Modulation frequency:	Stereo
Carrier frequency:	2,3 / 2,8 MHz
Frequency response:	20 - 20000 Hz
Distortion:	< 1 %
Noise level:	typ. 60 dB
Operating range:	0° C to 40° C

Receiver:	Operating time per charge:	approx. 12-13 h
	Maximum volume SPL:	approx. 120 dBA
	Weight:	approx. 73 g

Transmitter:	Power consumption:	approx. 3.6 VA
	Operating voltage:	12 Volt DC
	Supply voltage:	230 Volt AC 50 Hz
	Weight:	approx. 172 g

This device satisfies the following EU directives:
 - 2002/95/EG RoHS directive - 2002/96/EG WEEE directive
 - 2004/108/EG EMC directive - 2006/95/EG Low voltage directive
 Compliance with the directives listed above is confirmed by the CE seal on the device. CE compliance declarations are available on the Internet at www.humantechnik.com.

Technical specifications subject to change without notice.

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