

Clinical Imaging

Macro images captured from a fixed distance

- 1 Turn On
Push & hold power button for 1-2 sec.



- 2 Move Extension Arm
Push or pull forward

- 3 Turn Focus Lever
To Bottom Position



- 4 Press Shutter Button
Push to take photo, which will be displayed on screen.



Press shutter button to return to live view, press again to take another photo.

Dermoscopic Imaging

Lesion images captured with skin contact

- 1 Turn On
Push for 1-2 sec.
- 2 Move Extension Arm
Push or pull back



- 3 Choose Polarization
On-screen icon color indicates status:
X = Polarized
X = Non-Polarized

- 4 Turn Focus Lever
To Top Position



- 5 Press Shutter Button
Push to take photo, which will be displayed on screen.



Press shutter button to return to live view, press again to take another photo.

dermlite **cam**[®]

Quick-Start Guide