



UNIVERSAL FIT

BEDCADDIE™

Sit up in bed with ease!


- **Installs and removes in seconds**
- **Padded easy grip soft handles**
- **Nylon construction**
- **Adjustable length**
- **Fits any frame bed**
- **Padded handle**



Compatible with any metal framed bed. Also compatible with most hospital and nursing home beds as well!

INSTRUCTIONS

Read the safety notice below. Check your existing bed frame for proper set up (A). Loop the lower BedCaddie strap (C) around the bed frame (A). Snap the easy connect into place (B). Lay the BedCaddie up on the bed (D) adjusting the length (B) such that the first handle lies at waist level when lying prone in bed.

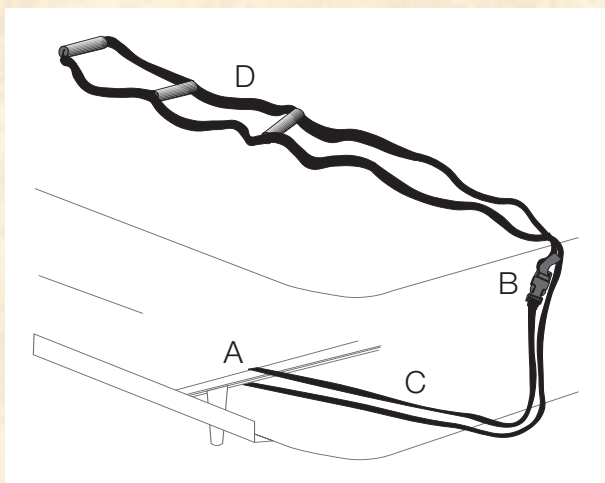

CAUTION

The BedCaddie™ is designed to assist people in sitting up and lying down. It is not designed to carry the full weight of an individual. It is intended for use as an assistive device in conjunction with a bed as shown in the illustration. Before installing, inspect the bedframe that will have the BedCaddie installed on it. Make sure it is properly set up to the manufacturer's specifications. Some bed frames that are assembled using spring clips need to be checked to be sure that the spring clip is fully engaged. Before using the BedCaddie, test by pulling on the handle to ensure secure connection to the frame.

2085

www.stander.com

Made in China

