

vive®



# **FOLDABLE WHEELCHAIR TRANSPORT**

Owner's Manual  
MOB1021

[vivehealth.com](http://vivehealth.com)

## WHAT'S INCLUDED

- Hand grips
- Hand Brakes
- Front and Rear Wheels
- Leg Adjusters
- Foot Plates
- Seat Upholstery
- Arm Rests

## ASSEMBLY INSTRUCTIONS

Tools Needed: 14mm wrench (not included)

WEIGHT CAPACITY: 300 LBS

## CAUTION

Please note that this chair is designed to use for transporting another individual and is not meant for self transportation. This Transport Chair is designed to safely be a stable center of gravity but it is important to try and keep a stable center of gravity of all times while performing activities that may change your center of gravity or a passengers center of gravity. Never reach out so far that it would require you to shift in the seat or bend over out of the chair. When you have to reach for objects that require you to lean, be certain your front wheels are pointing forward as this will stabilize the chair.

### Ramps

Be sure of your own capabilities, limitations and strength before attempting to push another individual up a ramp. Correct techniques on ramp negotiation should be shown to you by a qualified professional. If you must stop on an incline, avoid any and all sudden or abrupt movement when you resume movement. Because of backward angle of the chair, sudden abrupt movement could cause instability. Do not attempt to change direction when moving in incline or decline, this could result in tipping.

### **Tire Maintenance**

Tires should be cleaned occasionally with a damp cloth. Replace if tires become severely worn or cracking appears. Front casters should be checked periodically for cracking and lubricated occasionally. rear wheels should be inspected at least once a month to ensure proper tightness.

### **General Maintenance**

Wipe metal parts with a soft cloth at least once a week. All aluminum parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately. Do not use an abrasive cleanser that will scratch the finish.

### **Un-Folding the Chair**

In a stable position, place both hands on the armrests of the chair. Pull the arm rests slowly away from one another until the seat upholstery is fully extended.



Grab the folded down hand grips and pull them up until the backrest upholstery is full extended and both hand grip bars snap into place locking them at a 90 degree angle.



### Adding the foot rests

Insert the foot rest onto the two metal placement tabs as shown. Swing the footrest as far as it will go into the middle, the locking lever then should fully engage onto the lock. Make sure the lock is engaged by pushing in on the back locking lever. Double check the lock by trying to swing the legs back out. They should hold firm in place if you have fully engaged the lock.



### Adjusting the height of the foot rests

With a 14mm wrench, loosen bolt on the clamp around support assembly. Raise or lower foot plate to desired position that will provide maximum support of thighs. Footrest should not be lower than 2" from the ground to ensure clearance when in motion.




### Removing of the foot rests

Release the lock by pulling out the back locking lever. Swing the footrest out sideways. Lift to remove.

## Folding the Chair

Pull down of the two black taps on the backside backrest upholstery portion of the chair, these will unlock the pins keeping the backrest at a 90 degree angle.

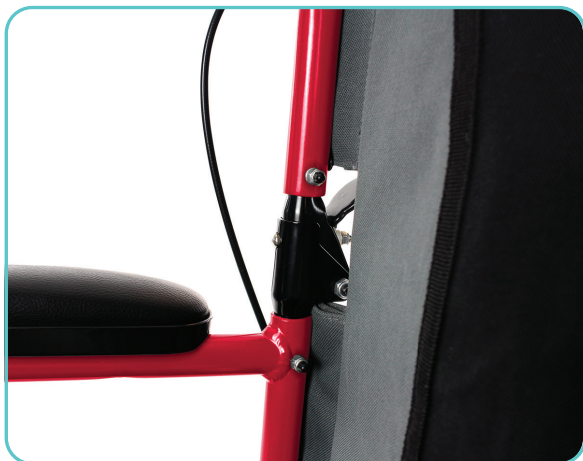

Grab the hand grips and pull them down so they are almost tucked around the base of the chair.



In a stable position, place both hands on the armrests of the chair. Pull the arm rests slowly toward each other until the seat upholstery is folded.

## WARNING

- Do not attempt to use wheelchair without reading carefully all instructions in this manual.
- This wheelchair has not been approved as a seat that can be used within vehicles of any kind.
- Always transfer wheelchair user to a vehicle manufacturers approved seat and use safety restraints available.
- Do not operate this wheelchair on roadways, streets or any surfaces with vehicular traffic.
- Do not attempt to operate on inclines without proper assistance.
- Wheelchair should be on a smooth, stable and level area with the hand brakes engaged by the transporter before transferring the passenger in or out of the wheelchair.
- When transferring in or out of wheelchair, do not stand on or apply weight to foot rests.
- Do not lean over or apply weight to the top of wheelchair back. This can cause wheelchair to tip over.
- Do not stand on wheelchair.
- Abruptly changing directions while going down an incline could cause instability and/or increase the possibility of tipping over.

## QUESTIONS?

Our friendly customer service team is standing by to assist you. Feel free to connect on the phone, email or chat.



[service@vivehealth.com](mailto:service@vivehealth.com)



1-800-487-3808



[vivehealth.com](http://vivehealth.com)