

WALKER GLIDES

MOB1006

USER INSTRUCTIONS

Note: For walker without front wheels, install walker glides on front 2 legs.
For walker with front wheels, install walker glides in the rear 2 legs.

1. Flip walker over and remove the front or rear two rubber feet. See note above.

2. Insert gliders on each of the front or back legs. The arrows on the glider should be pointing towards the direction of forward movement. See Figure 1.

Figure 1



3. Use Phillips screw driver to expand or retract diameter of insert. Tighten screw until gliders are secured to the walker. See Figure 2.

Figure 2



WALKER GLIDES

MOB1006

USER INSTRUCTIONS

Covers: Place the included black covers over each glider if using on wooden, laminated, or tiled floors. Black covers may be hand washed in cold water and air dry.

To Remove: To remove gliders simply loosen screw until gliders are loose enough to come off. Remove and store away.

Warning: These gliders are designed to reduce friction, movement may feel slightly different for a first time user.