Cleaning and Disinfection

Clean and air dry after every use.

Before each use

Wash hands well with soap and water.

If marked Sterile, contents are sterile and ready to use. Do not use if package is opened or damaged. If Non-sterile, place in boiling water for 20 minutes. Remove safely and air dry before use.

After each use

Unless otherwise directed by your Healthcare Professional, prepare the device using any option below. Do not use solvents or abrasives.

Hand Wash:

1. Clean using warm, soapy water.
2. Rinse using clean water.
3. Air dry on clean surface.

Dishwasher:

1. Use the upper rack of a standard household dishwasher.
2. If damp on removing from dishwasher, air dry on clean surface.

Boiling Water:

1. Place in boiling water for 20 minutes.
2. Remove safely from boiling water.
3. Air dry on clean surface.

These are general recommendations, and may be amended to meet hospital protocols.

Storage

After each Cleaning and Disinfection, place the dry device at room temperature in a clean container for storage.

Warnings and Cautions

Sterile package is opened. Do not use if damaged.

Discard according to local rules and regulations.

If uncomfortable during use, stop use and contact your Healthcare Professional.

Use product for its intended use only, as described in this Instructions for Use.

Store out of the reach of infants and children.

EN Ameda Nipple Shield

Breastfeeding Latching Aid with Cut-Out

Cut-out may allow increased mom and baby skin-to-skin contact.

Contents

If sterile box - 20 packs with

Instructions for Use.

If non-sterile box - 1 pack with Instructions for Use.

Indications for Use

For short-term use for babies with latching problems.

Precautions

Device

Use under the guidance of a Healthcare Professional.

Before each use

Wash hands well with soap and water immediately before breastfeeding. Follow Cleaning and Disinfection steps.

After each use

Follow Cleaning and Disinfection steps.

General

Not intended for use as a shield for sore and/or cracked nipples.

Single user only.

Read instructions before use.

Retain instructions as future reference.

Instructions for Use

- 1 Pull back rim to about half the tip.
- 2 Place opening over nipple.

- 3 Unfold on breast.

- 4 If necessary, use fingers to hold shield in place during use.

Remove from breast after feeding.

Follow cleaning and disinfection steps.

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