

## CLEANING INSTRUCTIONS FOR VETERINARY AND CLINICAL USE:

# THE BELLA BED

STEP


1



Remove all liquid and visible solids from the surface of the Bella Bed.

STEP

2

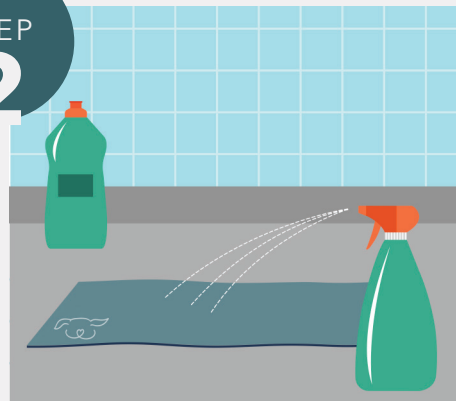

Clean both sides of the Bella Bed by applying the disinfectant product of choice and follow the dilution and contact time as instructed on the label.

STEP


3



Ensure the surface of the Bella Bed stays wet for the entire required contact time, then wipe both sides of the mat with a damp cloth.

STEP

4

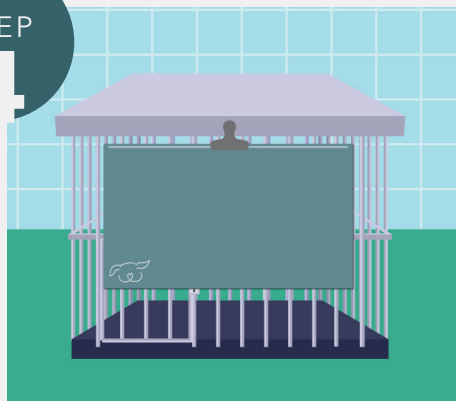

Hang or wipe the Bella Bed dry to be ready for the next patient.

