

Step 2:

Unfold panel to display complete instructions for Step 2

Step 3:

Ship the stool specimen to the lab


Specimen must be returned in the Genova Diagnostics kit box for correct delivery to the lab. Not following these instructions may result in a shipping charge.

- Plan to ship the specimen **Monday – Friday overnight delivery only.**
- Call 1.800.GoFedEx (1.800.463.3339) to schedule shipping. When the automated system asks “How may I help you?” say “Return a Package.” Tell the FedEx representative “I am using a billable stamp” and they will walk you through the process and make it easy.
- Place the **Day One, Day Two and Day Three biohazard bags inside kit box** and place your **completed and signed requisition form** on top before closing. **Do NOT staple or tape box.**
- **Print your name and address** in the section marked “From” on the prepaid shipping envelope label. **DO NOT mark or write in any other sections.**
- Put the kit box into the prepaid mailing envelope and seal the envelope.
- Keep your shipment and tracking numbers for future reference and tracking purposes.

Parasitology (Purge/Random)

IS-481

Patient Instructions

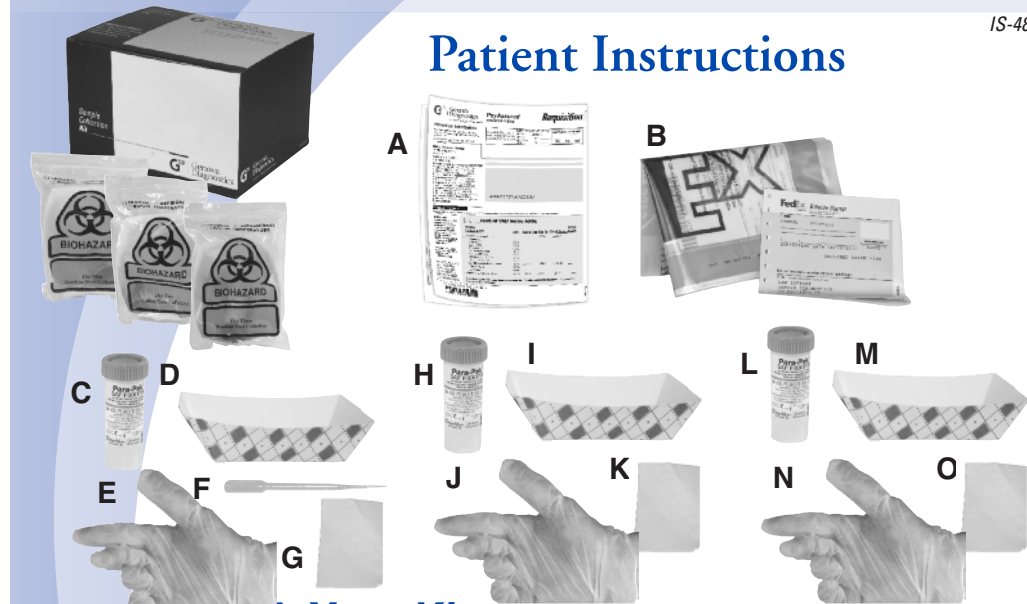

Check Your Kit

A - 1 Requisition Form (to be completed and signed)

B - 1 Prepaid mailing envelope

Day One Bag
Purge Collection

Day Two Bag
Random Stool
Collection

Day Three Bag
Random Stool
Collection

C - 1 Green-top SAF tube

H - 1 Green-top
SAF tube

L - 1 Green-top
SAF tube

D - 1 Collection tray

I - 1 Collection tray

M - 1 Collection tray

E - 1 Glove

J - 1 Glove

N - 1 Glove

F - 1 Disposable dropper

G - 1 Absorbent pad

K - 1 Absorbent pad

O - 1 Absorbent Pad

- If any items are missing or expired, call Client Services at 800.522.4762 and press “1”.
- Keep the kit box for shipping your specimen to the lab.

Step 1:

Important things to know and consider

- If any liquid is spilled from the vials, stop the test immediately. Any liquid lost from a vial invalidates the test results. Call Client Services at 828-253-0621 to obtain new vial(s). **Extreme care** should be used when opening and filling vials. Do not spill or discard any tube fluid. Avoid contact of the eyes or skin with the fluid in the tube. For contact with eyes, wash thoroughly for 15 minutes. For skin contact, wash thoroughly with soap and water. Do not inhale or ingest liquid. For accidental ingestion, contact your healthcare provider at once.
- This kit does *not* contain a laxative. A common over-the-counter laxative such as magnesium citrate may be used. Follow directions on the manufacturer's label for age and dosage instructions as well as pertinent warnings. Consult with your healthcare provider for any questions you may have related to the appropriate choice and/or use of a laxative for this test. On the day of your first collection, **do not eat or drink anything except water before taking the laxative**. You may resume your normal eating and drinking habits 1 hour after taking the laxative.
- The following are **interfering substances** and should be discontinued **unless directed otherwise by your physician**: bentonite clay, castor oil, mineral oil and rectal suppositories. If you are taking a medication that isn't listed as an interfering substance, ask your health care professional before stopping any prescribed substance.
- We do not suggest collecting during an acute infectious illness. Wait at least **4 weeks** from colonoscopy or barium enema before beginning the test.
- **2 weeks before the test**: If taking antibiotics or antifungals, it is recommended that you **wait at least 14 days** after your last dose before beginning the test (unless instructed otherwise by your physician).
- **Eat your usual diet** during the 2 - 3 days prior to the stool collection.

Schedule & Prepare for Stool Collection

- **Plan for Monday-Friday collection only: Specimens must be received in the laboratory within 4 days of Day 3 collection.** If you take the test over the weekend, call FedEx to schedule a pick-up on Monday.
- **Contact FedEx and schedule to ship the specimen overnight delivery** Monday - Friday.
- **Storing your samples**: Send in your samples as soon as you complete the test. Store samples at room temperature or refrigerate. **Do NOT freeze.**
- **Complete the Requisition Form** with all patient and billing information. Be sure it is signed by the Patient/Responsible Party and the healthcare provider.

Prepare your specimen using these collection tubes on these days:

- Day One:** 1 Green-top SAF tube
- Day Two:** 1 Green-top SAF tube
- Day Three:** 1 Green-top SAF tube

Step 2:

Stool Collection

Not following these instructions may affect the test results.

Day One (Purge Collection)



1

Write your name and the date the specimen was collected on the green-top SAF tube label from the Day One biohazard bag. And then take the laxative your physician has provided or recommended at the specified dosage. Drink 8 oz. of water afterwards.



2

Pass your first bowel movement into the toilet and discard as usual. Within 2-4 hours (on average) you should experience mild diarrhea. Pass any formed or soft stool until you have a watery bowel movement. Collect a watery movement using the collection tray in the kit.



3

Open the green-top tube. Stir the sample lightly with the disposable dropper. Use the dropper to collect from contrasting areas of the sample, including blood, pus, etc. Transfer liquid stool into the tube. Add sufficient stool sample for the stool and tube liquid to reach the red "fill" line, as shown on the label. **Do not overfill.**



4

Recap the tube securely. Then, shake well until mixed thoroughly.



5

Be certain caps are tight for shipping. Place your filled tube in the Biohazard bag corresponding to the day's collection. Seal the bag securely. Do *not* refrigerate or freeze the sample.

Day Two (Random Stool Collection)



6

Write your name and the date the specimen was collected on the green-top SAF tube label from the Day Two biohazard bag. If unable to collect on consecutive days, resume the test on the next day you have a bowel movement.



7

Pass urine into the toilet if necessary. Pass stool into one of the collection tubs provided in the kit. Do not let urine or water from toilet touch the stool specimen. Repeat steps 4 through 6 from the Day One instructions.

Day Three (Random Stool Collection)



8

1-6. Repeat the same collection process as Day Two.

Consult your healthcare provider if you have any questions at any time during this test.