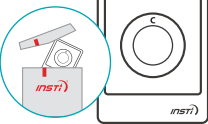
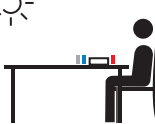



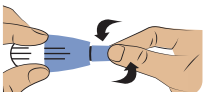
### PREPARATION


**1**  Open test device pouch.

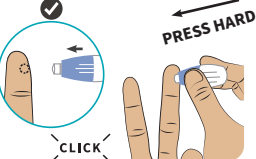
**2**  Place test device on a flat surface.

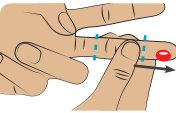
**3**  Remove cap of Bottle 1. Place on flat surface.

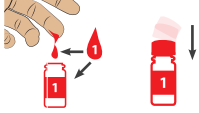
### STEP 1 COLLECT BLOOD


**A**  Twist off tip and put aside.

**B**  Rub finger and hand to increase blood flow.

**C**  Place lancet on the side of finger tip.

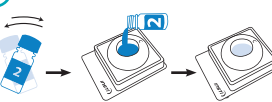
**D**  Rub finger to create a **LARGE** drop of blood.

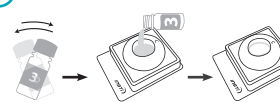
**E**  Let 1 drop **FALL** into Bottle 1. Twist on cap of Bottle 1.

**F**  Apply bandage.

### STEP 2 TEST

**G**  Shake **4 times** and pour all liquid. Wait until liquid disappears.

**H**  Shake and pour all liquid. Wait until liquid disappears.

**I**  Shake and pour all liquid. Wait until liquid disappears.

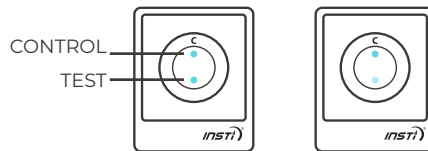
### STEP 3 READ RESULT

Read result right away to **within 1 HOUR.**



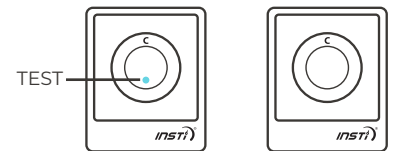
**NEGATIVE**

**TEST** again in 3 months.



**POSITIVE**

**GO TO CLINIC** for confirmatory testing.



**INVALID**

**GO TO CLINIC** for confirmatory testing.

### Disposal Instructions:



Put all items back into the pouch. Throw away pouch in waste bin.

Blood can transmit infectious diseases. Clean up spills. Dispose in accordance to local regulations.

Distributed by:



[www.pasante.com](http://www.pasante.com)

Manufactured by:



[www.insti.com](http://www.insti.com)



SCAN ME TO LEARN MORE

65-1349(A) 11/2022