

Materials in Direct or Indirect Contact with Operator

NewLife Intensity 10:

Concentrator casing	Valtra/ABS/Polystyrene
Mains cable	PVC
Dust filter	Polyester
ON/OFF switch	Thermoplastic
Casters	Nylon
Flow adjustment	ABS/Polycarbonate
Gas outlet	Chrome Plated Brass
Printed labels	Lexan

Cleaning, Care, and Proper Maintenance



WARNING: CLEAN THE CABINET, CONTROL PANEL, AND POWER CORD ONLY WITH A MILD HOUSEHOLD CLEANER APPLIED WITH A DAMP (NOT WET) CLOTH OR SPONGE, AND THEN WIPE ALL SURFACES DRY. DO NOT ALLOW ANY LIQUID TO GET INSIDE THE CONCENTRATOR. PAY SPECIAL ATTENTION TO THE OXYGEN OUTLET FOR THE CANNULA CONNECTION TO MAKE SURE IT REMAINS FREE OF DUST, WATER, AND PARTICLES.

WARNING: CARE SHOULD BE TAKEN TO PREVENT THE OXYGEN CONCENTRATOR FROM GETTING WET OR ALLOWING FLUIDS TO ENTER THE UNIT. THIS CAN CAUSE THE UNIT TO MALFUNCTION OR SHUT DOWN, AND CAUSE AN INCREASED RISK FOR ELECTRICAL SHOCK OR BURNS.

WARNING: DO NOT USE OIL, GREASE, OR PETROLEUM-BASED OR OTHER FLAMMABLE PRODUCTS WITH THE OXYGEN-CARRYING ACCESSORIES OR THE OXYGEN CONCENTRATOR. OXYGEN ACCELERATES THE COMBUSTION OF FLAMMABLE SUBSTANCES.

Cabinet

Turn OFF the unit and disconnect from power before any cleaning or disinfection. DO NOT spray the outer case directly. Use a damp (not wet) cloth or sponge. Spray the cloth or sponge with a mild detergent solution to clean the cabinet. Proceed as directed by the cleaner manufacturer. Device cabinet should be cleaned at minimum between users.



WARNING: ELECTRICAL SHOCK HAZARD. TURN OFF THE UNIT AND DISCONNECT THE POWER CORD FROM THE ELECTRIC OUTLET BEFORE YOU CLEAN THE UNIT TO PREVENT ACCIDENTAL ELECTRICAL SHOCK AND BURN HAZARD. ONLY YOUR EQUIPMENT PROVIDER OR A QUALIFIED SERVICE TECHNICIAN SHOULD REMOVE THE COVERS OR SERVICE THE UNIT.

WARNING: USE ONLY WATER-BASED LOTIONS OR SALVES THAT ARE OXYGEN COMPATIBLE PRIOR TO OR DURING OXYGEN THERAPY. NEVER USE PETROLEUM OR OIL-BASED LOTIONS OF SALVES TO AVOID THE RISK OF FIRE OR BURNS.

WARNING: DO NOT USE LIQUID DIRECTLY ON THE UNIT. A LIST OF UNDESIRABLE CHEMICAL AGENTS INCLUDES BUT IS NOT LIMITED TO THE FOLLOWING: ALCOHOL AND ALCOHOL-BASED PRODUCTS, CONCENTRATED CHLORINE-BASED PRODUCTS (ETHYLENE CHLORIDE), AND OIL-BASED PRODUCTS (PINE-SOL®, LESTOIL®). THESE ARE NOT TO BE USED TO CLEAN THE PLASTIC HOUSING ON THE OXYGEN CONCENTRATOR, AS THEY CAN DAMAGE THE UNIT'S PLASTIC. CLEAN THE CABINET, CONTROL PANEL, AND POWER CORD ONLY WITH A MILD HOUSEHOLD CLEANER APPLIED WITH A DAMP CLOTH (NOT WET) OR SPONGE, AND THEN WIPE ALL SURFACES DRY. DO NOT ALLOW ANY LIQUID TO GET INSIDE THE DEVICE.

Note: Always follow the cannula manufacturer's instructions for proper use. Replace the disposable cannula as recommended by the cannula manufacturer or your equipment provider. Additional supplies are available from your equipment provider.

Note: The Manufacturer does not recommend the sterilization of this equipment.

Filters

At least one time each week, wash the air intake gross particle filter, which is located in the back of the unit. Your Equipment Provider may advise you to clean it more often, depending upon your operating conditions. Follow these steps to properly clean the air intake filter:

Note: Do not operate the unit without the intake gross particle filter in place.

1. Remove the filter and wash it in a warm solution of soap and water.
2. Rinse the filter thoroughly, and remove excess water with a soft, adsorbent towel. Ensure that the filter is completely dry before replacing it.
3. Replace the dry filter.

Reserve Oxygen Supply

Your Equipment Provider may recommend another source of supplemental oxygen therapy in case there is a mechanical failure or a power outage.



CAUTION: The Manufacturer recommends an alternate source of supplemental oxygen in the event of a power outage, alarm condition, or mechanical failure. Consult your physician or Equipment Provider for the type of reserve system required.

Cannula Replacement

Always follow the cannula manufacturer's instructions for proper use. Replace the nasal cannula or oxygen tubing as recommended by the cannula manufacturer or your oxygen provider. Your physician or oxygen provider will provide you with cleaning and replacement instructions.

Additional supplies for replacement are available through your oxygen provider.