

## Filter Cleaning and Replacement

The particle filter must be cleaned weekly to ensure the ease of air flow. Remove filter from the side of the device. Clean the particle filter with a mild liquid detergent (such as Dawn™) and water; rinse in water and dry before reuse. Ensure the filter is completely dry before reinstalling into the concentrator.



## Air Inlet Filter

Replace when filter is dirty according to visual indicator.

The air inlet filter may be replaced by the equipment provider or by the user.



## Output Filter

The output filter is intended to protect the user from inhalation of small particles in the product gas flow. The device includes an output filter conveniently located behind the removable cannula nozzle fitting. Inogen requires that this filter be replaced between patients.

The output filter may be replaced by the equipment provider or by the user.

## Routine Maintenance

No special maintenance, other than weekly particle filter cleaning, needs to be carried out by the user.

## Replacement Parts and Recommended Accessories

Contact your home care provider if you have any questions about equipment. Use only the following replacement parts and accessories with this device:

- Inogen At Home Particle Filter (RP-400)
- Inogen At Home Output Filter Replacement Kit (RP-107)
- Inogen At Home Air Inlet Filter (RP-401)
- Inogen At Home Column Pair (RP-402)
- Humidifier Connector Tube (Salter Labs #50-676)
- Humidifier Bottle (Salter Labs #7600)
- Nasal Cannula (Salter Labs #16SOFT)
- AC Power Cord (RP-109)

## Service

The concentrator is specifically designed to minimize routine preventative maintenance.