

# Motif Duo<sup>®</sup> Quick Guide

## 1. Buttons & Features

### LCD Screen

- A Memory
- B Timer
- C Expression
- D Power
- E Battery
- F Massage
- G Vacuum

### Button Control


- H Power Button
- I Increase Button
- J Mode Button
- K Decrease Button



## 2. Preparing The Duo Pump


### Assembling The Duo Pump

1. Wash hands before touching breast pump components.
2. Place the diaphragm downward onto the diaphragm bottom, so that the cross-lined surface is facing the diaphragm bottom. Cover with the diaphragm top and turn it to lock.
3. Attach the lanyard to the breast pump by using the ring on the bottom of the main body of the pump. Loop the lanyard strap through the ring. Pull the strap through until it is tight.
4. Place the breast shield in the breast shield connector.
5. Connect the valve by placing it in the bottom of the breast shield connector. Screw the bottle into the breast shield connector.
6. Twist the diaphragm assembly onto the top of the breast shield connector.
7. Connect the tubing to the port to the main body of the pump.
8. Connect the other end of the tubing to the back of the diaphragm assembly by inserting the tip of the diaphragm top into the tubing.
9. To clean or store after use unscrew the bottle from the breast shield connector and cover. Remove tubing from diaphragm assembly and pump. Twist the diaphragm assembly off of the breast shield connector. Following cleaning instructions on breast pump parts after each use.

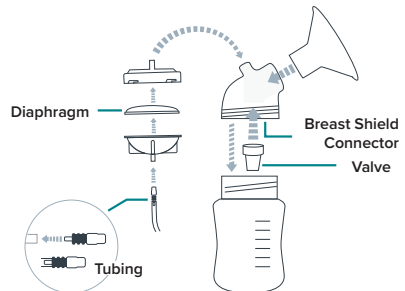

### 3. Using The Duo

#### HOW TO USE THE DUAL PUMP

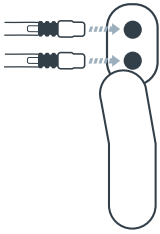

##### Double-Pump Mode

To use the Duo in Double-Pump mode remove the plug from the air flow port and connect the tube from the extra collection kit.



##### Single-Pump Mode

When changing back to Single-Pump mode make sure to cover the air flow port with the plug.

#### TURNING ON/OFF THE DUO PUMP

1. **Press the POWER button.** The device will power ON, indicated by the LCD screen lighting up. The Duo will begin in Massage Mode to help with faster letdown.
2. **Press the MODE button.** This will change the pump to Expression Mode once your milk starts flowing.
3. **Adjust suction to preferred comfort level.**
4. **Use your breast pump with a fully charged battery whenever possible.** Prior to initial use, plug the device into the power adapter and allow the breast pump to charge fully. The battery charge level displayed on the LCD screen can differ from the actual battery capacity. Charge the battery immediately when [Lo Bt] is displayed on the LCD screen. If the breast pump remains unused for a long period of time, make sure that the battery is fully charged before use. Unplug the power adapter from the breast pump immediately after you are finished charging your device.

#### USING MEMORY MODE

**Press the POWER button** for 2 seconds to activate the Memory function. The Memory icon will start flashing, and the pump will start recording your pumping session.

**Adjust the settings during your pumping session as best suits you.**

**Press the POWER button** for another 2 seconds to stop recording. The Memory icon in the screen shows that your program has been stored.

To play your pumping program, **press the DECREASE button** (the same button you press to decrease vacuum intensity) for 2 seconds.

### 4. Warnings & Precautions



**WARNING:** Potentially dangerous situation that could result in serious injury.



**CAUTION:** A possibly dangerous situation that could result in slight bodily injury.



Manufactured for:  
Motif Medical®, LLC  
8 The Green, Suite 5601  
Dover, DE 19901

Phone: (844) 272-8390  
Email: [info@motifmedical.com](mailto:info@motifmedical.com)  
Web: [www.motifmedical.com](http://www.motifmedical.com)

*motif*  
MEDICAL