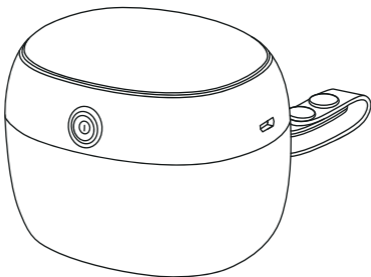
Papablic

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Mini esterilizador UV

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USER MANUAL

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IMPORTANT SAFEGUARDS

Please take the time to read this user manual carefully before using the Mini UV Sterilizer and save it for future reference.

DANGER

- Please do not place close to fire sources.
- Do not clean the base, mains cord, or plug in water and prevent water from entering the appliance through the outlets to avoid any damage to the product or any danger to yourself and others.
- Do not reach for or drop into water or other liquids.
- Never disassemble the base to avoid the risk of electric shock.

WARNING

- This appliance is not intended for use by persons (including children) with reduced physical, sensory, or mental capabilities or a lack of experience and knowledge unless they have been given supervision or instruction concerning the appliance's use by a person responsible for their safety.
- Keep the appliance and mains cord out of the reach of children.
- Never allow children to play with the PE packing bags of the product to avoid the risk of suffocation.
- Do not place the appliance, mains cord, or plug close to hot surfaces, sharp edges, and corners. If the plug, mains cord, or the appliance itself is damaged, do not use it.
- Check if the voltage indicated on the appliance corresponds to the local mains voltage before connecting the appliance.
- Eye damage may result from directly viewing the light produced by the lamp in this apparatus. Always turn off the LED before opening the protective cover.
- UV light radiation can cause serious skin burns and eye injuries. Do not look directly at the UV light while in operation.

IMPORTANT SAFEGUARDS

- Never use this device on pets and food.
- Do not tamper with the lid interlocks to prevent UV exposure.
- Close supervision is necessary when this product is used by, on, or near children or disabled people.
- Use the Mini UV Sterilizer only for its intended use as described in this manual. Do not use attachments not recommended by Papablic.
- This product is for household use only.

CAUTION

- Never use any accessories or parts from other manufacturers or those that Papablic does not specifically recommend. If you use such accessories or parts, your guarantee will become invalid.
- Do not use the appliance if it has fallen or is damaged in any way.
- Please do not use it with a damaged cord or plug. If the appliance is not working as intended, it has been dropped, damaged, left outdoors, or dropped into a liquid, return it immediately to Papablic for repair.
- Do not expose the appliance to extreme heat or direct sunlight. Always place and use the appliance in a dry place. When not in use, please clean and store it in a dry place.
- Do not allow it to be used as a toy. Close attention is necessary when used by or near children.
- Do not handle the plug or appliance with wet hands.
- It must be supplied safely at an extra-low voltage corresponding to the markings on the appliance.
- Environmental conditions, such as temperatures and altitude, may affect the functioning of this appliance.

 **WARNING:** The cover is provided with an interlock to reduce the risk of excessive ultraviolet radiation. Do not defeat its purpose or attempt to service without removing it.

CONGRATULATIONS

Congratulations on your purchase of the **Papablic Mini UV Sterilizer!**

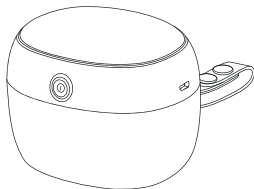
The Portable Sterilizer lets you disinfect your baby's pacifiers quickly and easily. With its small size and convenient power options, you can keep your baby healthy at home or on the go.

This product has been designed and developed with safety and functionality in mind. If the lid is opened during use, the built-in safety protection switch will automatically turn off the UVC light.

INCLUDED PARTS

CHECK WHAT YOU HAVE RECEIVED...

You should have received **1 box**. Please check the contents of the box against the list below. If anything is missing, please contact Customer Service at service@papablic.com.


Sterilizer Unit

USB-C Cable

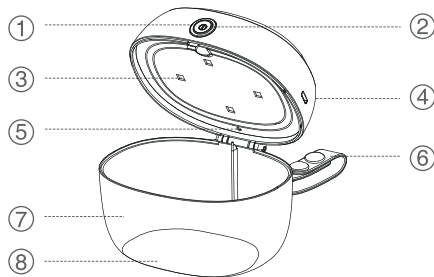INCLUDED PARTS

- ① Indicator Light
- ② Power Button
- ③ UVC LEDs
- ④ USB Type-C Port
- ⑤ Safety Protection Switch
- ⑥ Handle
- ⑦ Base
- ⑧ Mirror

HOW TO USE

SAFETY FEATURE

If the lid is opened during use, the UVC light will automatically turn off to protect the skin and eyes from UV exposure.

BEFORE FIRST USE

The Papablic Mini UV Sterilizer has been cleaned with alcohol wipes before packaging. They may have an alcohol smell when first opened. It is recommended that you sterilize the appliance twice with an empty basket before using it for the first time.

INDICATOR LIGHT CHART

INDICATOR LIGHT	MODE
Blinking Blue Light	Charging
Blue Breathing Light	Disinfecting in 59-Second Mode
Green Breathing Light	Disinfecting in 3-Minute Mode
Solid Light	Fully Charged
No Light	Disinfecting Complete/Turn Off
Blue Light Blinking Five Times Quickly	Needs to Be Charged Immediately

HOW TO USE

OPERATION TIPS

There are two operation modes(59-second / 3-minute) for use.

	OPERATION	INDICATOR LIGHT
How to Start/ 59-Second Mode	Press the power button.	Blue Breathing Light
3-Minute Mode	Press the power button twice.	Green Breathing Light
How to Stop	Press the power button three times.	No Light
Charge	Plugging in.	Blinking Blue Light (The blue light is always on when fully charged).

HOW TO USE

HOW TO USE

1. Wash the pacifier or wipe it clean to remove any debris before sterilization. To open the sterilizer, squeeze it on both sides with one hand (see Fig. 1), place the pacifier into the sterilizing box, then close the lid (see Fig. 2).

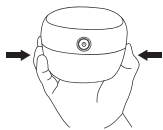
Fig.1

2. Use 59-second mode. Press the power button to begin disinfecting. The blue breathing light will come on until the process is complete (59 seconds) (see Fig. 3).

Fig. 2

Or use 3-minute mode. Press the power button twice to begin disinfection. The green breathing light will come on until the process is complete (3 minutes) (see Fig. 3).

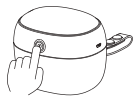
Fig. 3

3. When disinfecting is finished, the indicator light will turn off. Open the lid and remove the pacifier (see Fig. 4).


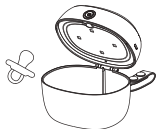
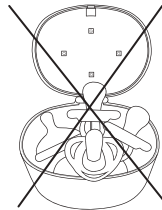
Fig. 4

HOW TO USE

Tips:

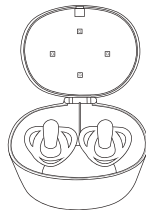Do not insert more than two pacifiers at a time. This device disinfects the surfaces exposed to the UVC light. The pacifier must have unobstructed access to the UVC light.

HOW TO CARE FOR YOUR PAPABLIC UV STERILIZER

MAINTENANCE

Clean the interior of the product for watermarks and stains after each use. Always store the sterilizer in a clean, dry place.

CLEANING

This device is not waterproof. Never immerse the device, the mains cord, or the mains plug in water or any other liquid.

To clean, wipe the surface of the unit with a towel. Remove any water droplets to prevent moisture from damaging the unit's circuitry.

Between use or while carrying it during travel, please place the product into a storage bag to prevent dust contamination and protect the surface from abrasion.

Caution

- Never allow any water on the UV LEDs when cleaning the appliance.
- Do not use chemicals, cleaners, vinegar, or any other cleaning solution on the product.
- Always wipe off the dry water spots after each use. Dry water spots will hinder the performance of the UV lights.

TROUBLESHOOTING

TROUBLESHOOTING

1. If the power button does not respond or the unit will not turn on:

- (A) Check if the power is depleted; charge it if required.
- (B) If the internal circuit is damaged, the unit must be returned to Papablic for after-sales service.

2. The UVC light will not shut off after the lid is opened:

- (A) Check if the product cover detection switch is dirty. Wipe it clean with a cloth.
- (B) Check if the rubber column of the upper cover detection switch is broken.
- (C) If the rubber cover of the upper cover detection switch is broken or the internal circuit is damaged, the unit must be returned for after-sales service.

TECHNICAL SPECIFICATIONS:

Product Name: Mini UV Sterilizer

Model: PPL50

Rated Input: 5VDC $\overline{\text{---}}$ 1A

Dimensions: 3.93 in. (100 mm) x 3.3 in. (84 mm) x 3.54 in. (90 mm)

Net Weight: 4.23 oz. (120 g)

UVC LED Life: 5,000 hours

Input Connector: USB Type-C

Battery Life: 100 minutes

WARRANTY SERVICE

WARRANTY SERVICE

We specifically and expressly warrant that the products manufactured or sold by Papablic to our customers shall be of the highest quality and applied to our warranty service.

For a period of **one (1) year** after the Delivery Date, Papablic warrants that the Product, when used in accordance with the instructions in the applicable documentation, will operate as described in all material respects. During the Warranty Period, defects in the materials or workmanship of components or deliverables specified and furnished by or through Papablic shall be repaired or replaced by Papablic at no cost or expense to Customer.

Papablic will correct any reproducible error in the Product or replace any defective Product provided that such error is reported to Papablic by Customer in writing during the Warranty Period, and that Customer provides all information that may be necessary (receipt of the product, proof of purchase). For the avoidance of doubt, Papablic does not warrant that Customer's use of the Product will be error-free or uninterrupted. If the product is determined damaged intentionally by Customer, it would not be applicable for the Warranty Service.

The above-mentioned Warranty Service may only be applicable for Product acquired on Papablic official website, official store of Papablic on Amazon (USA and Canada) and Walmart. Please contact us directly at service@papablic.com (<http://papablic.com/>) for any question related to our Product, and one of our overly eager customer service representatives will get back to you.

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