


# User Guide


HAP-150



## CHARGING

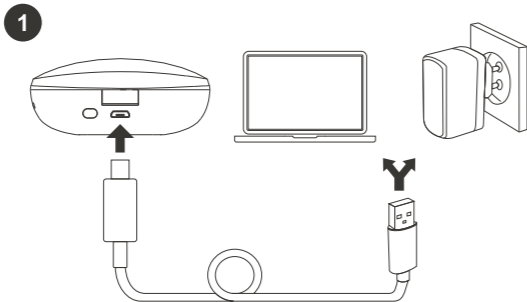

Charge the charging case and hearing aids using the USB cable connected to a computer or wall charger.

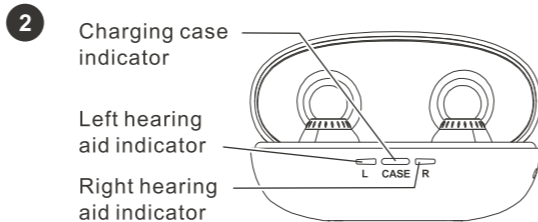


- Charging : Indicator turns orange.
- Fully charged : Indicator turns green.

**⚠ Fully charge the case and hearing aids for the 1st time.**

## CHARGING CASE INDICATORS ILLUSTRATION

	Battery Level	Hearing Aid	Charging Case
Hearing Aid Charging	100%	Green	Green
	> 25%	Orange	Orange
	< 25%	Orange	Flashing Orange
Charging Case Charging	100%	Green	Green
	> 25%	Orange	Orange
	< 25%	Orange	Orange
Not Charging		Off	Off

## INSERT HEARING AIDS

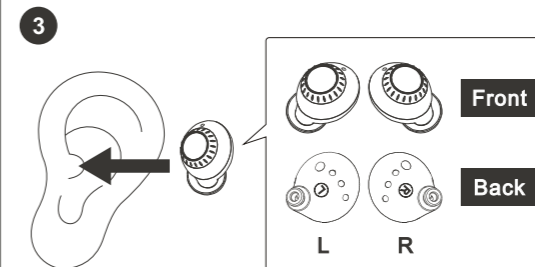


**⚠ Choose a correct ear tip size. A whistling sound may occur if the tip is not firmly inserted into your ear.**

## TURN HEARING AID ON/OFF




- Turn ON : Hearing aid will turn ON automatically in 3 seconds after being inserted in your ear. 2 beep sound can be heard.
- Turn OFF : Hearing aid will turn OFF in 3 seconds after being removed from your ear. Place it in the charging box.

## ADJUST VOLUME

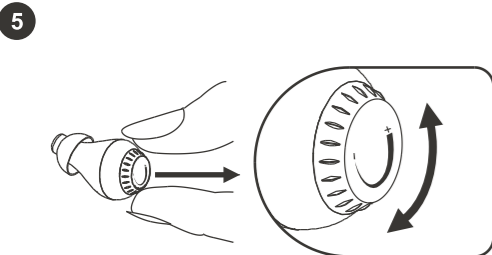

- Rotate the [ Volume Knob ] to adjust volume.


## CHARGING ON-THE-GO

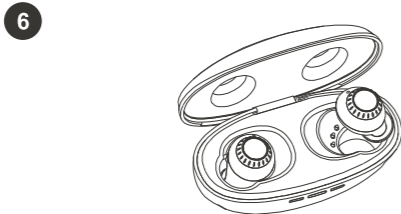

- Place your hearing aids in the charging case.
- A fully charged case provides 3 extra hearing aids charges when on-the-go.


- Charging : Indicator turns orange.
- Fully charged : Indicator turns green, then OFF.

## TURN CHARGING CASE ON/OFF

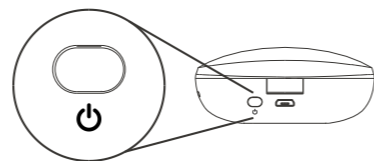

- Turn OFF : Press and hold the [ Power Button ] until 3 orange indicators flash thrice. Charging case is OFF and will stop charging.
- Turn ON : Press and hold the [ Power Button ] until 3 green indicators flash thrice. Charging case is ON and will resume charging.

## CHECK BATTERY LEVEL

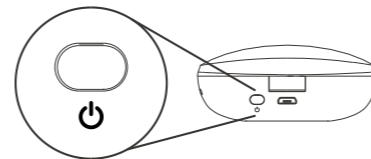

When the charging case is ON, and all indicators are OFF, press the [ Power Button ] on the back of the charging case once. The indicators will turn ON and show the battery level.

## SAFETY INSTRUCTIONS

1. This product is suitable for mild to moderately severe hearing loss only, and not suitable for patients with tympanitis or ear pyogenesis.
2. Avoid listening at high volume for a long period of time to avoid possible hearing damage.
3. Stop using immediately if you feel any discomfort.
4. Please maintain your hearing aid all the time. Avoid the ear tip and microphone being blocked by foreign objects, as it will affect the result of sound amplification.
5. Avoid shaking or dropping your hearing aid.
6. Do not keep your hearing aid in an environment of high humidity or high temperature.
7. Keep out of reach of children.

## SPECIFICATIONS

Charging Time	Approx. 2 hours
Usage Time	Approx. 12 hours
Extra Charges by Charging Case	Approx. 3 times

hopewell / since 1985

© hopewell. All right reserved. The hopewell logo is a registered trademark of Norm Design Ltd. Information in this document is subject to change without notice.

hopewell-living.uk

