


User Guide

HAP-160



CHARGING


1

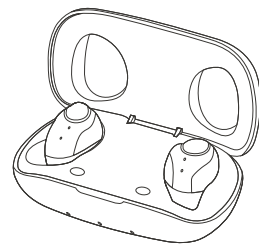

⚠ Turn hearing aids off before charging.


- Place them in the charging case.

2

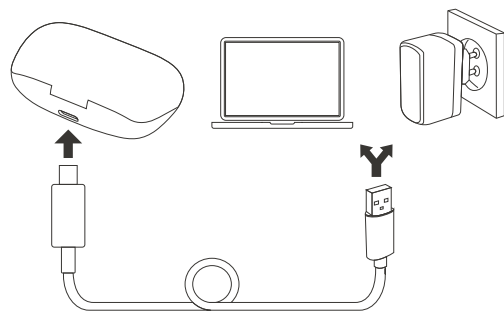

Charge the case and hearing aids using the USB cable connected to a computer or wall charger.

3

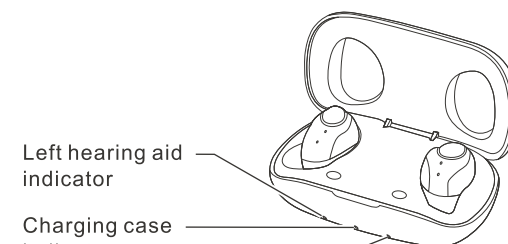

Left hearing aid indicator

Charging case indicator

Right hearing aid indicator

Charging Case

- Charging : Red indicator turns ON.
- Fully charged : Indicator turns OFF.


Hearing Aids

- Charging : Blue indicator turns ON.
- Fully charged : Indicator turns OFF.

⚠ Charge the case and hearing aids for 12 hours for the 1st time.

INSERT HEARING AIDS


4

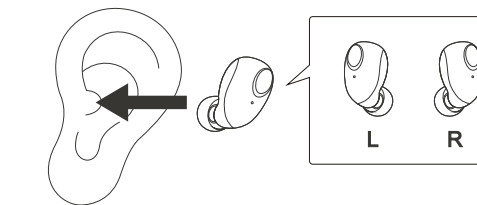

⚠ Choose a correct ear tip size. A whistling sound may occur if the tip is not firmly inserted into your ear.

TURN ON


5

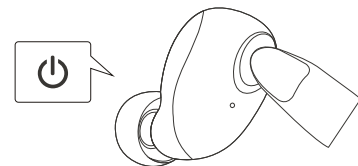

- Tap and hold the hearing aid button for 5 seconds until the green indicator flashes to turn it ON.

ADJUST VOLUME


6

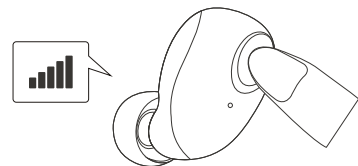

- Tap the hearing aid repeatedly to increase volume.
- 2 beep sound is heard when it reaches the highest volume level (level 5).
- Tap once more and it will return to level 1.

TURN OFF


7

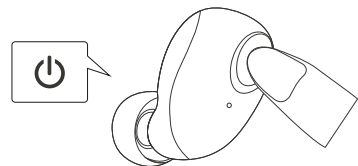

- Tap and hold the hearing aid button for 5 seconds until the blue indicator flashes to turn it OFF.

CHARGING ON-THE-GO

8

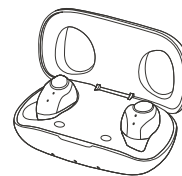

⚠ Turn hearing aids off before charging.

- To charge your hearing aids, place them in the charging case and close the lid.
- A fully charged case provides 4 extra hearing aids charges when on-the-go.

SAFETY INSTRUCTIONS

1. This device is suitable for mild to moderately severe hearing loss only, and not suitable for patients with tympanitis or ear pyogenesis.
2. Avoid listening at high volume for a long period of time to avoid possible hearing damage.
3. Stop using immediately if you feel any discomfort.
4. Maintain your device all the time. Avoid the ear tip and microphone being blocked by foreign objects, as it will affect the result of sound amplification.
5. Avoid shaking or dropping your device.
6. Do not keep your device in an environment of high humidity or high temperature.
7. Charge your device regularly to avoid battery overdischarge.
8. Keep out of reach of children.

SPECIFICATIONS

Charging Time	Approx. 4 - 6 hours
Usage Time	Approx. 30 hours
Extra Charges by Charging Case	Approx. 4 times

hopewell / since 1985

© hopewell. All right reserved. The hopewell logo is a registered trademark of Norm Design Ltd. Information in this document is subject to change without notice.

hopewell-living.uk

