


User Guide

HAP-75U




CHARGING

1

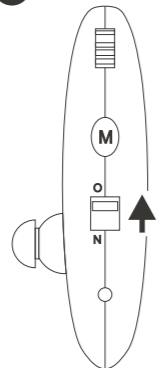


Switch the [ON/OFF Switch] to " O " to turn OFF the hearing aid.

2

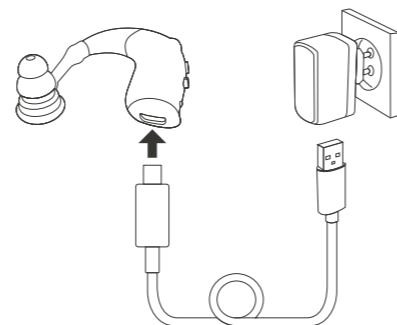

Charge the hearing aid using the USB cable connected to a computer or wall charger.

3

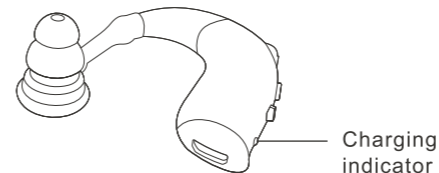


- Indicator turns green while charging.
- Indicator turns OFF when fully charged.

⚠ Charge device for 12 hours the first time.

TURN ON HEARING AID

4

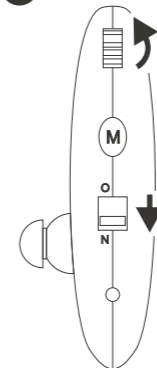


- Turn the volume to " 0 " before use.

- Switch the [ON/OFF Switch] to " N " to turn ON the hearing aid.

WEAR HEARING AID

5

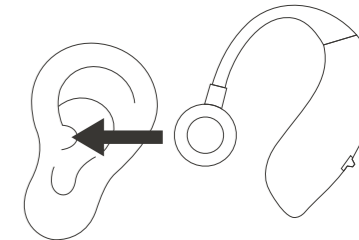


Insert the hearing aid over the top of your ear.

* Choose a correct ear tip size. A whistling sound may occur if the tip is not firmly inserted into your ear.

ADJUST VOLUME


6

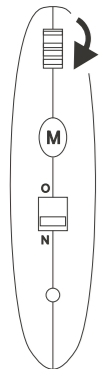

Slowly turn volume up until the loudness suits you best.

SELECT LISTENING MODE

7

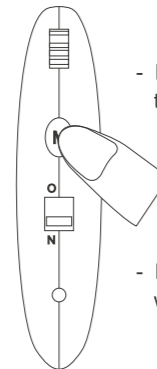

- Press the [M button] repeatedly to select listening mode.

- BEEP sound(s) can be heard while entering each mode.

1 BEEP SOUND - Normal Mode

- Amplifies all sound.
- For general use.

2 BEEP SOUNDS - Noise Reduction Mode

- Reduces background noise, enhances speech.
- For noisy environment.

3 BEEP SOUNDS - Lo Frequency Mode


- Soothes and boosts lo freq. sound.
- For lo freq. environment.

4 BEEP SOUNDS - Hi Frequency Mode

- Soothes and boosts hi freq. Sound.
- For hi freq. environment.

TURN OFF HEARING AID

8

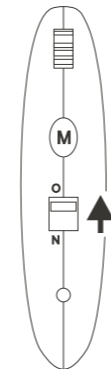

Switch the [ON/OFF Switch] to " O " to turn OFF the hearing aid.

⚠ Switch OFF the hearing aid if it is not in use.

⚠ Charge the hearing aid for at least 12 hours if it has not been used and charged for a long time. Charge the battery often to prevent it from dying.

SAFETY INSTRUCTIONS

1. This product is not suitable for patients tympanitis or ear pyogenesis.
2. Avoid listening at high volume for a long period of time to avoid possible hearing damage.
3. Stop using immediately if you feel any discomfort.
4. Please maintain your hearing aid all the time.
5. Avoid the ear tip and microphone being blocked by foreign objects, as it will affect the result of sound amplification.
6. Clean the ear tip often : Remove the ear tip from the hearing aid. Rinse it with clean water. Allow it to dry before reinstalling it.
7. Avoid shaking or dropping your hearing aid.
8. Do not keep your hearing aid in an environment of high humidity or high temperature.

SPECIFICATIONS

Charging Time	Approx. 4 - 6 hours
Usage Time	Approx. 50 - 60 hours

1. Max. sound output: 130dB ± 3dB
2. Total harmonic distortion: ≤ 8%
3. Frequency range: 500 ~ 3500Hz
4. Input noise: ≤ 32dB
5. Voltage: 3.7V
6. Current: ≤ 2mA

hopewell / since 1985

© hopewell. All right reserved. The hopewell logo is a registered trademark of Norm Design Ltd. Information in this document is subject to change without notice.

hopewell-living.uk

