


User Guide

HAP-76BT




CHARGING

1

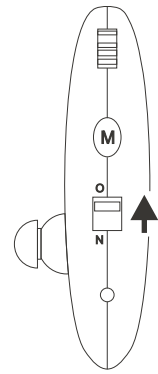


Switch the [ON/OFF Switch] to " O " to turn OFF the hearing aid.

2

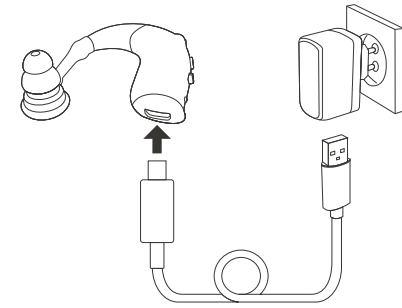

Charge the hearing aid using the USB cable connected to a computer or wall charger.

3

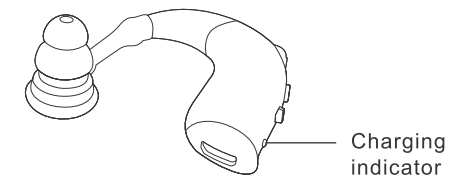


- Indicator turns green while charging.
- Indicator turns OFF when fully charged.

⚠ Charge device for 6 hours the first time.

WEAR HEARING AID

4

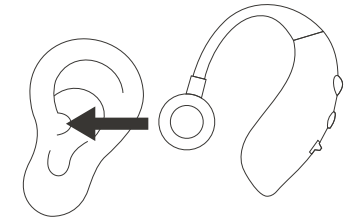


Insert the hearing aid over the top of your ear.

⚠ Choose a correct ear tip size. A whistling sound may occur if the tip is not firmly inserted into your ear.

TURN ON HEARING AID

5

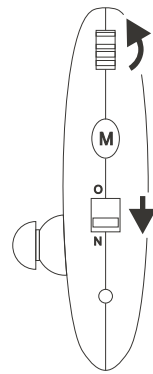


- Turn the volume to " 0 " before use.

- Switch the [ON/OFF Switch] to " N " to turn ON the hearing aid.

ADJUST VOLUME


6

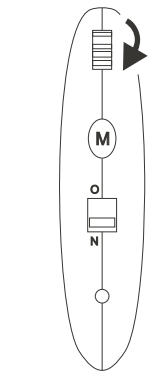


Slowly turn volume up until the loudness suits you best.

PAIR A BLUETOOTH SMART DEVICE WITH YOUR HEARING AID

7

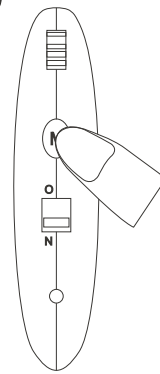


1. Press and hold the [M button] for 3 seconds until a signal sound is heard, and the blue indicator flashes to enter Bluetooth pairing mode.
2. Go to Bluetooth settings on your smart device. Select " ZTQ " on the list to start pairing.

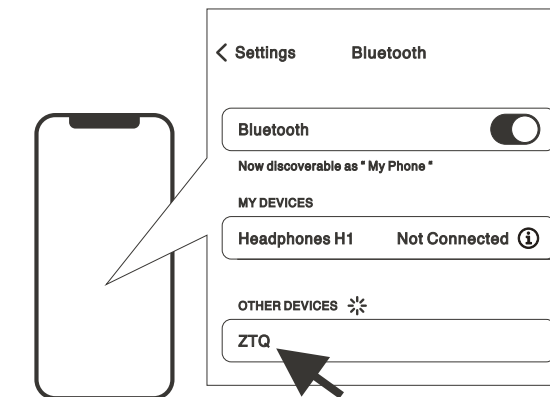

3. An audio message can be heard on your hearing aid when pairing is successful, and the indicator will stop flashing.

From now on, your hearing aid will be automatically connected to your smart device when Bluetooth on your hearing aid is turned ON, until you remove it from your smart device.


⚠ Bluetooth cannot be turned ON when battery of your hearing aid is low, whilst the amplification function will still work normally. Please charge your hearing aid in order to use the Bluetooth function again.

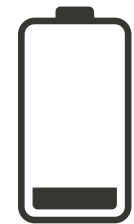

When the Bluetooth function is in use, and the battery runs low, BEEP sound can be heard intermittently. Bluetooth function will be turned OFF shortly, and amplification mode will resume.


TURN BLUETOOTH ON/OFF

8

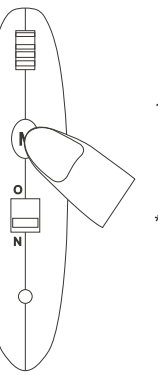

1. To turn Bluetooth OFF: Press and hold the [M button] for 3 seconds.


* Bluetooth will be turned OFF when the hearing aid is turned OFF.

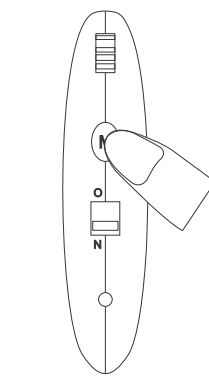

2. To turn Bluetooth ON: Press and hold the [M button] for 3 seconds until a signal sound and an audio message is heard.

BLUETOOTH MODE - ANSWER PHONE CALL

9

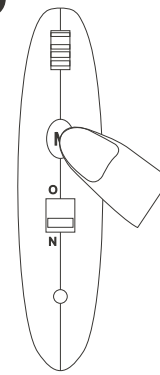


1. Press the [M button] once to answer, or press and hold the [M button] to decline incoming call.
2. Press the [M button] to switch between phone handset and hearing aid mode during a call.
3. Press and hold the [M button] to hang up.

* The hearing aid will automatically return to the amplification mode after hanging up.

BLUETOOTH MODE - ADJUST PHONE CALL VOLUME


10

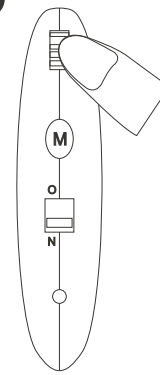

1. Adjust the volume on your hearing aid.
- * Phone call volume can only be adjusted via the hearing aid, but not your smart device.

BLUETOOTH MODE - CONTROL MUSIC PLAYBACK

11

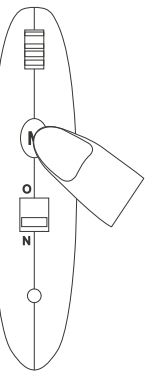


1. When playing music, press the [M button] once to pause playback.
2. Press the [M button] to resume playback.

* The hearing aid will automatically return to the amplification mode after no audio signal is detected for 10 seconds. Bluetooth will be turned OFF automatically if it is not being used for an extended period of time.

BLUETOOTH MODE - ADJUST MUSIC VOLUME


12

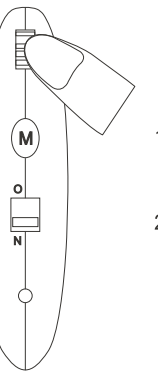

1. Adjust the volume on your smart device.
2. Adjust the volume on your hearing aid.

TURN OFF HEARING AID

13

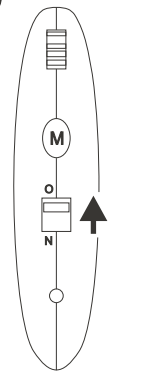

Switch the [ON/OFF Switch] to " O " to turn OFF the hearing aid.

⚠ Switch OFF the hearing aid if it is not in use.

⚠ Charge the hearing aid for at least 6 hours if it has not been used and charged for a long time. Charge the battery often to prevent it from dying.

SAFETY INSTRUCTIONS

1. This device is suitable for mild to moderately severe hearing loss only, and not suitable for patients with tympanitis or ear pyogenesis.
2. Avoid listening at high volume for a long period of time to avoid possible hearing damage.
3. Stop using immediately if you feel any discomfort.
4. Maintain your device all the time. Avoid the ear tip and microphone being blocked by foreign objects, as it will affect the result of sound amplification.
5. Avoid shaking or dropping your device.
6. Do not keep your device in an environment of high humidity or high temperature.
7. Charge your device regularly to avoid battery overdischarge.
8. Keep out of reach of children.

SPECIFICATIONS

Charging Time	Approx. 4 - 6 hours
Usage Time	Approx. 80 - 100 hours

