


# User Guide

HAP-81U



## CHARGING


1

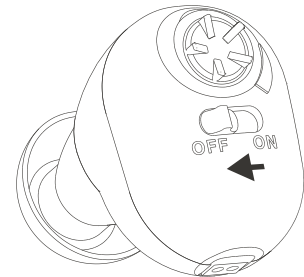

Make sure the [ ON/OFF Switch ] is switched to " OFF " before charging.


## CHARGING

2

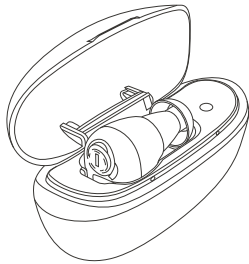


Place the hearing aid into the charging case.

3

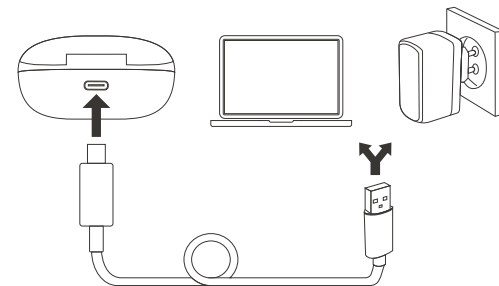

Connect the charging case to a computer or wall charger using the USB cable connected to charge the hearing aid.

4

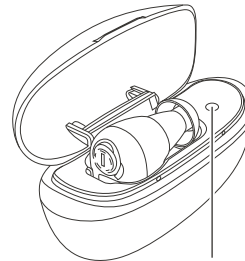


Charging indicator

- Indicator turns red while charging.
- Indicator turns green when fully charged.

**⚠ Charge device for 6 hours the first time.**

## WEAR HEARING AID


5

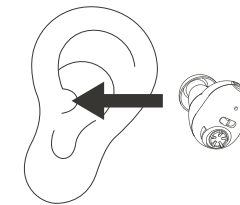

**⚠ Choose a correct ear tip size. A whistling sound may occur if the tip is not firmly inserted into your ear.**

## TURN ON HEARING AID


6

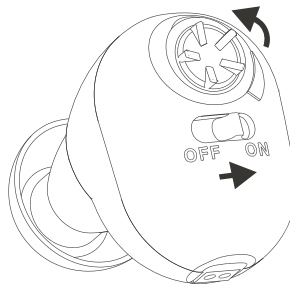

- Rotate the [ Volume Knob ] anticlockwise to minimal before use.
- Switch the [ ON/OFF Switch ] to " ON " to turn ON the hearing aid.

## ADJUST VOLUME


7

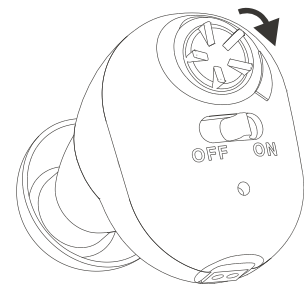

Slowly rotate the [ Volume Knob ] clockwise to turn the volume up until the loudness suits you best.

## TURN OFF HEARING AID

8

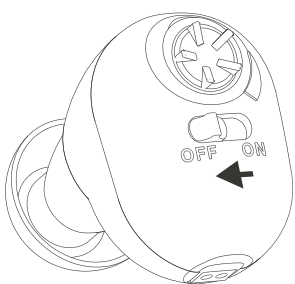

Switch the [ ON/OFF Switch ] to " OFF " to turn OFF the hearing aid.

**⚠ Switch OFF the hearing aid if it is not in use.**

**⚠ Charge the hearing aid for at least 6 hours if it has not been used and charged for a long time. Charge the battery often to prevent it from dying.**

## SAFETY INSTRUCTIONS

1. This device is suitable for mild to moderately severe hearing loss only, and not suitable for patients with tympanitis or ear pyogenesis.
2. Avoid listening at high volume for a long period of time to avoid possible hearing damage.
3. Stop using immediately if you feel any discomfort.
4. Maintain your device all the time. Avoid the ear tip and microphone being blocked by foreign objects, as it will affect the result of sound amplification.
5. Avoid shaking or dropping your device.
6. Do not keep your device in an environment of high humidity or high temperature.
7. Charge your device regularly to avoid battery overdischarge.
8. Keep out of reach of children.

## SPECIFICATIONS

Charging Time	Approx. 2 - 3 hours
Usage Time	Approx. 25 - 30 hours

hopewell / since 1985

© hopewell. All right reserved. The hopewell logo is a registered trademark of Norm Design Ltd. Information in this document is subject to change without notice.

hopewell-living.uk

