


How to use your LOOKEE® Premium Pulse Oximeter -- Quick Guide and Tips

Clinical Restrictions:


As the measure is taken on the basis of arteriole pulse, substantial pulsating blood flow of subject is required. For a subject with weak pulse due to shock, low ambient/body temperature, major bleeding, or use of vascular contracting drug, the SpO2 waveform (PLETH) will decrease. In this case, the measurement will be more sensitive to interference.

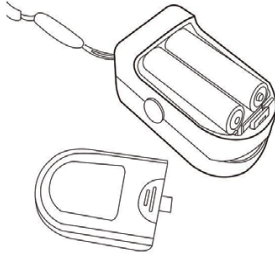

Quick Operation Guide:

1. Please install the 2 x AAA batteries inside the product with the correct directions.

2. Press the bottom to open the probe

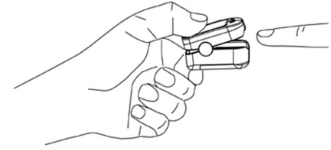


3. Insert the finger into the bottom of the monitor. To get the best results, make sure the center of the fingerprint is aligned with the emission sensor with red LED.



4. Press the button to turn it on. Wait for a couple of seconds to get the readings.


5. Set Up Alarm and Threshold: Press the button to turn on the monitor, and then long press the button again to enter Alarm Setup. Short press the button to move " * " between terms. Select the term, and press and hold the button to make changes. From the yellow main menu Alarm Setup, long press to enter Threshold Setup.

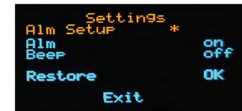

The Most Common Problem: Can't get accurate readings or the reading is jumping around and not stable.

Solutions:

- Please make sure your finger is warm.
- Please make sure the finger doesn't have a long nail and the fingertip can reach the end of finger chamber.
- Adjust your finger position to make sure the center of the fingerprint is aligned with the emission sensor with red LED.
- Please rest your hand on a table or flat surface to avoid moving.

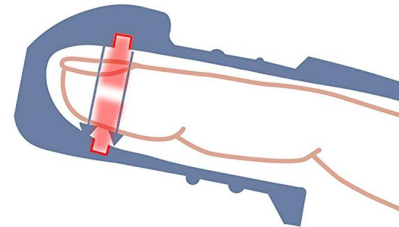

If you still have the problem to get readings?

No worries, our customer support team is here for you anytime:

**7/24 Customer Support: Support@Lookeetech.com
Live Chat with us at www.Lookeetech.com**