


Injection Pad (Paddy & Patrick) - Instructions for Use

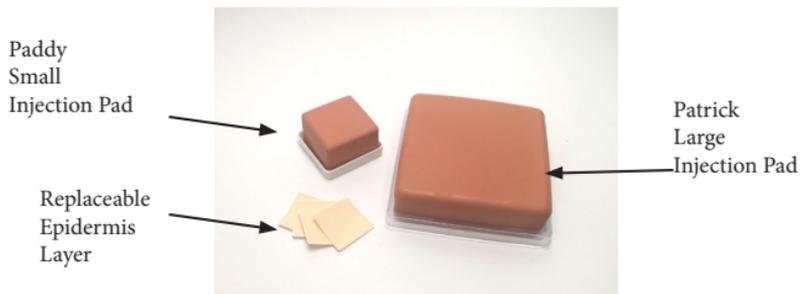

1. Remove the injection pad and epidermis layers from the packaging.
2. Prepare the product for intradermal injection by peeling the backing from the replaceable epidermis layer.
3. Place the epidermis layer on top of the injection pad and press to adhere.

Injection Pad - Instructions for Use



4. Perform the desired injection procedure per clinical protocols. Be aware some injections may cause leakage from the bottom of the pad depending on location and volume.
5. Cleanup by squeezing the pad over a sink or paper towel to remove any residual liquid.

Tips, Tricks, and Notes

- Wipe the top of the pad with rubbing alcohol before applying epidermis layer to improve adhesion.
- Edges of the epidermis layer may curl up if left in a humid environment. Press back on to use per instructions.
- Some liquid may ooze through the epidermis layer to the surface. This is normal.
- Contact Remedy with any questions or problems: info@remedysim.com
267-884-0684