STORZ

KARL STORZ — ENDOSKOPE

22 2010 20-1xx
22 2020 20-yyy

Image 1 HUB™ HD Kerasystem
Image 1 Pure HD Kerasystem

22 2010 20-1xx
22 2020 20-yyy

Image 1 HUB™ HD Camera System
Image 1 Pure HD Camera System

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Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern high quality piece of equipment from KARL STORZ.

This instruction manual is intended to serve as an aid in the proper setup, installation and operation of the Image 1 HUB™ HD and the Image 1 Pure HD. All essential details of the equipment and all actions required on your part are clearly presented and explained. We thus ask that you read this manual carefully before proceeding to work with the equipment. Insert this manual in its protective wallet and keep it available for ready reference in a convenient and conspicuous location near the equipment.

Property rights

This product is protected in the USA by (at least one of the following) US Patent No(s) 5,788,688; 6,397,286; 6,484,221; 6,824,539.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

General warning

Users of KARL STORZ devices should be thoroughly trained in their use and the applicable medical procedures. Use of these devices and accessories should not be undertaken until the user has fully familiarized himself/herself with the instructions for use, assembly and care. Instruction manuals should be carefully studied and be available to the surgical (endoscopic) team during the procedure; it is essential to follow the instructions contained in the instruction manual, paying particular attention to the Cautions and Warnings.

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* HD = High Definition (high resolution)
SD = Standard Definition (standard resolution)

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Camera Control Unit (CCU)

- ① Power switch
- ② Camera connection (HD camera head)
- ③ Camera connection (standard camera head) (behind the 'sliding door')
- ④ BNC video output connector (comp. video)
- ⑤ Light guide receptacle (for future use)
- ⑥ S-Video (Y/C) video output connectors
- ⑦ RGB-sync color video output connectors
- ⑧ HD DVI video output connectors
- ⑨ Potential equalization connector
- ⑩ Power supply cord receptacle
- ⑪ SCB I/O connectors
- ⑫ Keyboard receptacle
- ⑬ RS-232 serial ports
- ⑭ Accessory control outputs for remote control of peripheral units
- ⑮ Module expansion slots
- ⑯ Power fuse holder
- ⑰ SDI video output connectors (optional)
- ⑱ DV video output connector (optional)
- ㉒ Data interface
- ㉑ ICM (Image Capture Module) USB output, optional

Camera Head

- ⑲ Automatic instrument coupler
- ⑳ Focus ring
- ㉑ Focal length adjuster (zoom) (only Image 1™ camera heads H3-P/H3-Z, H3-ZI, S1/S3, A1/A3)
- ㉒ User control button: menu scrolling, Up
- ㉓ User control button: Menu/Select
- ㉔ User control button: menu scrolling, Down
- ㉕ Fixation clamp (only Image 1™ camera heads P1/P3)
- ㉖ “Snap-in” DCI® coupling mechanism
- ㉗ C-mount connection

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NOTE: The numbers on this page are in some instances not consecutive. This is deliberate, in order that the numbering is identical for the matching parts of the Image 1 hub™ HD and the Image 1 Pure HD.

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Camera Control Unit (CCU)

- ① Power switch
- ② Camera connection (HD camera head)
- ⑧ HD DVI video output connectors
- ⑨ Potential equalization connector
- ⑩ Power supply cord receptacle
- ⑪ SCB I/O connectors
- ⑫ Keyboard receptacle
- ⑬ RS-232 serial ports
- ⑭ Accessory control outputs for remote control of peripheral units
- ⑮ Module expansion slots
- ⑯ Power fuse holder
- ⑳ Data interface
- ㉑ ICM (Image Capture Module) USB output, optional

Camera Head

- ⑲ Automatic instrument coupler
- ⑳ Focus ring
- ㉑ Focal length adjuster (zoom)
(only Image 1™ camera heads H3-Z/H3-ZI/
H3-P, S1/S3, A1/A3)
- ㉒ User control button: menu scrolling, Up
- ㉓ User control button: Menu/Select
- ㉔ User control button: menu scrolling, Down

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The KARL STORZ-SCB® interface (KARL STORZ Communication Bus), based on the CAN field bus, permits remote control of equipment functions, as well as remote display of equipment parameters.

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Read the instructions carefully before operating the
equipment!

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Power off

Desconectado

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Potential equalization conductor

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Type CF equipment, defibrillation proof

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DANGER: Risk of explosion if used in the presence of flammable anesthetics.

CAUTION: To reduce the risk of electrical shock, do not remove cover

Refer servicing to qualified service personnel.

Do not store liquids on or above the unit.

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Wizard - Signifies that this feature/option is available in the Setup Wizard.

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Signifies that this feature/option is available in the Options Menu.

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Signifies that this feature/option can be controlled by the User Menu.

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Signifies that this feature/option can be controlled by the camera head buttons.

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Environmental protection use period of 50 years (China RoHS)

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General warning II
Photographs of the equipment IV,VI
Controls, displays, connectors
and their uses V,VII
Symbols employed VIII

General information

Device description 4
Camera heads (Overview) 7

Safety instructions

Warnings and cautions 10
Intended use 15
User qualification 17
Safety precautions at the site of installation 17
Safety precautions
when operating the unit 17

Set-up and installation instructions

Unpacking the equipment 15
Basic equipment 18
Compatible camera heads 19
Installation 20
Installing the unit 20
Connecting a ground line 21
Connecting power 21
Connecting the monitor 21
Optional modules 22
Connecting accessory device(s) for remote
control 24
Instructions on connecting
accessory devices 25
Connecting the keyboard 25
Connecting the HD camera head 26
Connecting the SD camera head 27
Connecting SCB 27
Example of a connection diagram 28
Connections to AIDA control 29
Powering the CCU 30
Possible camera heads 30
P1/P3/H3-P Pendulum camera head 32
D1 DCI® camera head 33
H3-M microscope camera head 34
F3 camera head 36
Adjusting the video monitor 38
Operating instructions 39
Camera head button basics 39
Keyboard basics 39

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Garantiekarte

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Maintenance

Fuse replacement.....	90
Cleaning, disinfection and sterilization.....	91
General care.....	91
Care and handling	91
Preparation Overview.....	93
(Preliminary) cleaning	96
Manual preparation.....	96
Machine preparation	
of autoclavable camera heads	99
Sterilization instructions –	
non-autoclavable camera heads.....	101
Sterilization instructions –	
autoclavable camera heads	110
Use of sterile covers	111
References for cleaning and sterilization	111
Maintenance.....	112
Servicing and repair.....	112
Disposal.....	113
Repair program	113
Limitation of liability.....	114
Manufacturer's warranty.....	114

Technical description

Troubleshooting.....	115
Technical data	118
Camera Control Unit.....	118
Camera heads	120
Technical documentation.....	124
Software User Rights.....	125
General circuit diagram.....	126
Menu diagrams	128

Spare parts, recommended

accessories..... 138

Appendix

Cleaning agents and disinfectants	143
Electromagnetic Compatibility (EMC)	
Information	145

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General description

The Image 1 HUB™ HD is an endoscopic camera control unit for use with both single and three-chip Image 1™ HD and standard (SD) camera heads. Image 1™ S1, S3, A1, A3, F3, P1, P3 and D1 are SD (Standard Definition) camera heads. Image 1™ H3-Z, H3-ZI, H3-P and H3-M are HD (High Definition) camera heads.

The Image 1 HUB™ HD offers the latest features, such as:

- Camera features and functions can be programmed for access via the camera head buttons
- All-digital circuitry for increased image accuracy, less noise in the image, and no image degradation from camera head to video output
- Digital image enhancement and fiberoptic endoscope filtering capabilities to increase the level of contrast and definition of the image
- Eighteen options for exposure control, including patented automatic exposure system and seventeen manual exposure control settings.

High Definition Technology (HD) offers further advantages (when using an HD camera head):

- Six times higher input resolution delivers more detail and depth of focus.
- 16:9 format image acquisition enlarges the field of vision and supports ergonomic viewing.
- The brilliance of color enables optimal diagnosis.
- See 32% more laterally by pulling back the scope with the same magnification as with a standard system, restore any loss of vertical information and keep the lens clear.

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General description

The Image 1 Pure HD is an endoscopic camera control unit for use with 3-chip Image 1™ HD camera heads. Image 1™ H3-Z, H3-ZI, H3-P and H3-M are HD (High Definition) camera heads.

The Image 1 Pure HD offers the latest features, such as:

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High Definition Technology offers further advantages

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The Image 1 HUB™ HD CCU and the Image 1 Pure HD incorporate the KARL STORZ Communication Bus (SCB system) for integration into a complete operating room system. The SCB system allows communication between the KARL STORZ devices connected to the system. From this communication, parameters of the SCB devices can be displayed centrally. Further information on the use and functions of the SCB system can be found in the KARL STORZ-SCB® System instruction manual, which can be requested free of charge from KARL STORZ or downloaded in PDF format by those registered at www.karlstorz.com.

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Camera heads (Overview)

H3-Z Camera head

- Universal - camera head for HD - applications (H3-Z = High Definition [HD] Camera head). Use for example for laparoscopy and arthroscopy
- includes 3 chips (H3-Z), therefore particularly high resolution
- Parfocal Zoom (H3-Z), therefore 2x optical image zoom possible

H3-ZI Camera head

- as H3-Z Camera Head. Has however a 0° cable outlet (cf. H3-Z: 30° cable outlet). Suitable therefore for applications where the camera head is not held in the fist, but instead like a pen.
- Example of use: Vein removal in bypass procedures.

H3-P Camera head

- High Definition (HD) Pendulum camera head
- Pendulum camera heads have a rotating grasping mechanism. This facilitates a 360 degree view of hollow organs.
- Typical areas of application are therefore in urology and gynecology

H3-M Camera head

- 3-chip Camera Head (H3-M), High Definition HD (H3-M). Camera heads specifically for microscopic applications
- with C-mount connection, thus suitable for microscopy

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P1/P3 Camera head

- Pendulum head like the H3-P. However, unlike the HD-Camera head, standard resolution (Standard Definition HD), whereas the H3-P has a particularly high resolution (HD = High Definition)
- P₁ = 1-Chip-camera head
- P₃ = 3-Chip-camera head

S1/S3 Camera head

- Camera head with standard resolution (Standard Definition SD)
- S₁ = 1-Chip-camera head
- S₃ = 3-Chip-camera head

A1/A3 Camera head

- as S1 and S3, however autoclavable (A1/A3)

D1 DCI® Camera head

- DCI® is short for Direct Coupling Interface
- Camera head with standard resolution (Standard Definition SD)
- patented DCI® snap lock
- particularly ergonomically beneficial in the cable outlet

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Insufflator Control – Image 1 HUB™ HD with H3-Z, H3-ZI and H3-P camera head

In conjunction with the H3-Z, H3-ZI and H3-P (HD) camera head, the Image 1 HUB™ HD CCU offers control of the functions involving the KARL STORZ insufflators** electronic ENDOFLATOR® 26430520-1, electronic ENDOFLATOR® LC 26430320-1 and THERMOFLATOR® 26432020-1.

- Flow on/off,
- Flow control,
- Pressure control and
- Volume reset.

* from serial number DB734098-H and software version 470200-0203 or higher

To use the insufflator control, connect the Image 1 HUB™ HD CCU and the insufflator to be controlled using the KARL STORZ communication bus (see 'Connecting SCB', p. 27).

Note: As the 'Camera' controls a 'Ib' medical device in its function as the master unit in the SCB network, the MPG (German Medical Devices Act) class of the CCU changes from 'I' to 'Ib' (see Section: Technical data).

Note: The camera can only control the insufflator in 'Intermittent Flow' mode. If the insufflator (THERMOFLATOR®) is in 'Semi-continuous flow' or 'Pneumoperitoneum initialization' mode, the user will be prompted to switch to 'Intermittent Flow' mode.

Warning: In the event of camera malfunction, always refer to the front panel displays of insufflator.

Warning: After connecting the camera and insufflator via SCB cable, the monitor display must be compared with the insufflator display for functional testing.

Warning: check that the camera insufflator control is operating properly, enable or change at least one function using the camera and check the reaction on the insufflator.

**** Note:** Please observe the instruction manual of the respective insufflator.

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Warnings and cautions

Please read this manual and follow the instructions carefully. The words **Warning**, **Caution** and **Note** convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of the device.

Definitions

Warning: A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result in injury to the patient or physician.

Caution: A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the unit.

Note: A Note indicates special information about operating the unit.

Warning: Read this instruction manual thoroughly and be familiar with its contents prior to using this equipment. Read the section on safety instructions carefully to avoid putting your patients, personnel, or yourself at risk.

Before using the unit on the patient it is imperative that you be acquainted with how the unit operates and is controlled.

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WARNINGS:

Warning: The instructions and interface specifications for medical devices used in combination must be observed precisely.

Warning: Combinations of medical devices are only assured to be safe if

- they are identified as such in the respective instruction manuals or
- the intended purpose and interface specifications of the devices used in combination permit this (cp. IEC 60601-1-1).

Warning: Accessory equipment connected to the analog and digital interfaces (signal inputs and outputs) must be set up and tested according to IEC 60601-1 for medical electrical equipment and additionally to IEC 60950 for data processing equipment. Furthermore, all configurations shall comply with the system standard IEC 60601-1-1. Any person who connects additional equipment to the signal input part or signal output part configures a medical system, and is therefore responsible for ensuring that the system complies with the requirements of the system standard IEC 60601-1-1. If in doubt, consult the technical service department or your local representative.

Warning: Test this equipment prior to each surgical use. In the event that the image becomes unusable during surgery, the camera may be disengaged from the endoscope and the procedure continued optically. If this is not possible, it is left to the discretion of the surgeon how best to proceed. Availability of a spare system is recommended.

Warning: Do not use in the presence of flammable anesthetics. There is a risk of explosion.

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Warning: Danger of electric shock. Do not open the unit. Refer servicing only to qualified personnel. Any opening of the unit by unauthorized persons voids the guarantee. The only exception to this are activities described in this instruction manual.

Warning: Grounding reliability can only be achieved when the equipment is connected to a properly installed “Hospital Only” or “Hospital Grade” receptacle (i.e., approved for use in an operating room environment). Routinely inspect electrical plug and cord. Do not use if inspection reveals damage.

Warning: The electrical installations in the operating room in which the unit is connected and operated must comply with the applicable IEC standards.

Warning: Always unplug the unit before performing any maintenance including cleaning.

Warning: Keep out of reach of patients.

Warning: For reasons of safety, do not simultaneously touch the device output connectors and the patient.

Warning: During discharge of a defibrillator, the image displayed may be interrupted for up to 1 second.

Warning: When cleaning the unit and preparing/sterilizing the unit accessories (“Camera head assembly”), it is essential to follow the instruction manual closely (see e.g., overview table for preparation on page 87).

Warning: Any deviations from the recommended parameters for sterilization should be validated by the user.

Warning: To avoid burns during endoscopic procedures, operators must use caution when employing non-BF/CF type equipment while using electro-surgical devices.

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Warning: High energy radiated light through endoscopes may give rise to high temperatures in front of the light outlet and to the tip of the endoscope. To minimize the risk of burns, always adjust the light source to the minimum illumination intensity necessary to achieve optimum illumination of the endoscopic scene when coupled to the video camera.

Warning: When endoscopes are used with energized endoscopically-used accessories, the patient leakage currents may be additive. This is particularly important if a Type CF endoscope is used, in which case a Type CF endoscopically-used accessory should be used in order to minimize the total patient leakage current.

Warning: Before each use, the outer surface of the portions of the endoscope and any endoscopically-used accessories which are intended to be inserted into the patient should be checked to ensure there are no unintended rough surfaces, sharp edges or protrusions which may cause a safety hazard.

Warning: Gas which may support combustion is sometimes present in the gastrointestinal tract of an unprepared patient and certain patient preparation substances used prior to GI endoscopy can enhance methane production. This is particularly relevant to colonoscopy, but has also been recorded in the upper GI tract and during transurethral resection of the prostate. It has been recorded that hydrogen can accumulate in the bladder above the irrigant solution. Dissipate any flammable gasses in the colon, bladder, or other body cavities prior to use of high frequency surgical equipment.

Warning: When using the camera system in a surgical discipline employing a laser, caution should be taken to prevent eye damage to the operator. The operator should wear protective glasses whenever viewing the surgical site directly through the endoscope.

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Caution: Do not store liquids on or above the device. Avoid allowing fluids to enter the unit.

Caution: Only operate the unit with the voltage stated on the unit's identification plate.

Caution: When replacing fuses, use only fuses of the same rating.

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Note: Disposal or recycling of components at the end of their useful life must be in accordance with local regulations. CCU components contain lithium batteries. Used batteries must be disposed of in an environmentally friendly manner: Send them to an officially appointed regional collection center or return them to the supplier.

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Intended use

The KARL STORZ Image 1 HUB™ HD and the Image 1 Pure HD is a video camera system which can be used with rigid or flexible endoscopes, or with a microscope in the case of the H3-M microscope camera head. The camera head is coupled to the endoscope, or to the microscope in the case of the H3-M camera head. Any compatible Image1 HUB™ SD- and HD- camera head may be used with the Image 1 HUB™ HD Camera Control Unit (CCU), and any compatible Image 1™ HD-camera head with the Image 1 Pure HD CCU (SD = Standard Definition, HD = High Definition). The endoscopic image can be displayed on any standard operating room video monitor.

All standard endoscopic light sources may be used with Image 1™ camera heads.

In addition, the Image 1 HUB™ HD CCU and the Image 1 Pure HD CCU are suitable for controlling certain units designated for use with them, e.g., light source (CCU = Camera Control Unit).

From serial number DB734098-H and software version 470200-0203 or higher, remote control of the following insufflator functions is possible for the Image 1 HUB™ HD CCU in conjunction with the H3-Z, H3-ZI and H3-P (HD) camera heads:

Gas on/off, flow control, pressure control and volume reset.

Please observe the instruction manual of the respective insufflator. You must be aware of how this equipment is used.

Peak Modes (= peak shutter values) are integrated in the H3-Z, H3-ZI and H3-P HD camera heads.

Note: This instruction manual does not replace the instruction manuals of the connected equipment.

Use of Image 1 HUB™ HD and the Image 1 Pure in fields other than those described above is not allowed for safety reasons.

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User qualification

The Image 1 HUB™ HD may only be used by physicians and medical assistants who have a corresponding specialized qualification and who have been instructed in use of the unit.

Safety precautions at the site of installation

The unit may only be used in medical rooms whose electrical systems have been installed in accordance with applicable national regulations.

The unit is equipped with a connector for attaching a ground line. It should be connected before power is applied to the unit in compliance with current national regulations.

It is not intended for use in hazardous zones. This means, for example, that when using easily combustible and explosive inhalation anesthetics or mixtures thereof, the unit must not be operated inside the hazard zone shown in the diagram. This also applies for easily combustible and explosive chemicals, e.g. skin disinfectants and fast-acting surface disinfectants.

Safety precautions when operating the unit

It is the user's responsibility to make sure the equipment is safe and operates properly before use.

During treatment with the Image 1 HUB™ HD the patient must be treated and kept under observation with the usual medical care. This includes keeping a check on the progress of treatment, as well as monitoring sterile application conditions where required by the type of intervention.

CE marked equipment has been tested and found to comply with the EMC limits for the Medical Device Directive 93/42/EEC (EN 55011 Class A and EN 60601-1-2 :2001).

Please observe the information on electromagnetic compatibility in the Appendix (as of p. 138).

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Unpacking the equipment

Carefully unpack the Image 1 HUB™ HD camera control unit (CCU) and remove the unit and its accessories from their packaging. Check for missing items and evidence of shipping damage. File any complaints with the manufacturer or supplier immediately.

If possible, retain the original packing materials for later use; these can come in handy if the unit has to be transported.

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Switching from 50 to 60 Hz

To switch the camera head from 50 Hz to 60 Hz, proceed as follows:

- Connect keyboard
- Simultaneously press FN + Ctrl + Alt + Enter (the FN button is present on the supplied notebook keyboard). This opens the diagnostic menu.
- Select Module Diagnostics
- The camera head can here be switched between 50 and 60 Hz.

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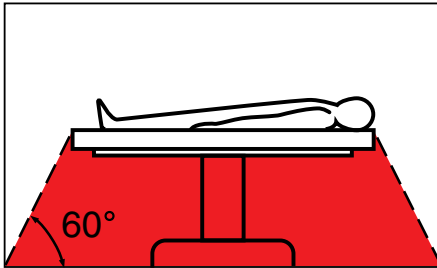
Installation

Note: The Image 1 HUB™ HD camera control unit including the accessories connected may be used only in medical facilities having electrical installations conforming to applicable national, state, and local electrical codes.

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Warning: This unit is not intended for use in hazardous zones. Do not operate the unit within the hazard zone shown in the diagram while explosive anesthetic gases are in use.

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Installing the unit

Place the CCU on a level surface.
Make sure there is sufficient distance on all sides to other instruments (especially radio frequency surgical equipment) and objects.
Before plugging in the unit, make sure that the voltage on the nameplate corresponds to the voltage of the local power line.

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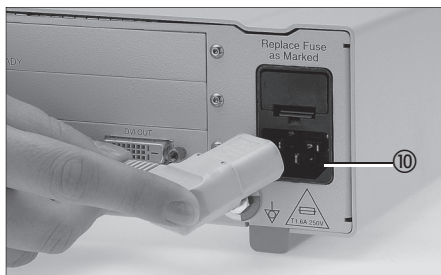
Connecting a ground line

The camera control unit (CCU) is equipped with a potential equalization connector for attaching a ground line ⑨. The unit's ground line should be installed by a qualified electrician.

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Connecting power

Connect power cord. Insert power cord into power cord receptacle ⑩ as far as it will go.

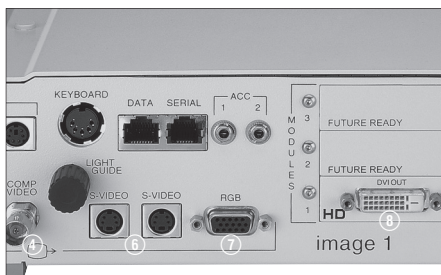
Warning: Only insert the power plug into and remove it from electrical outlets located outside areas subject to explosion hazards.

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Connecting the monitor

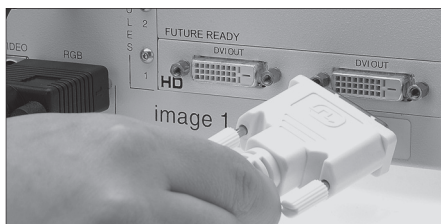
Connect the camera control unit to the video monitor to be used. Depending upon the types of video input connectors provided on the video monitor, this connection can be generated using either the CCU's

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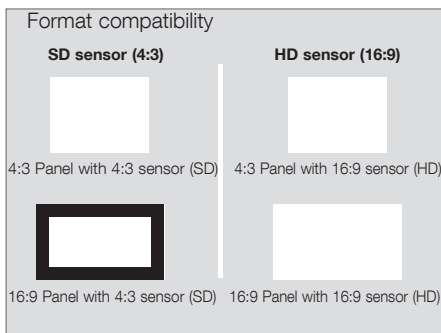
Note: Optimal picture quality is obtained using the CCU's DVI output.

To adjust video monitor settings, please see 'Adjusting the video monitor' section on page 32.

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
The Image 1 HUB™ HD offers several standard definition video outputs: RGB, S-Video, Composite and SDI are offered as module options. The Image 1 HUB™ HD includes two High Definition DVI-D monitor outputs. These will display the HD image when an HD head is plugged in and will display an SD image when a SD head is plugged in. The Various combinations of inputs (SD/HD) and outputs (4:3 Panel/16:9 Panel) result in various image configurations on the monitor screen, as shown in the left chart.

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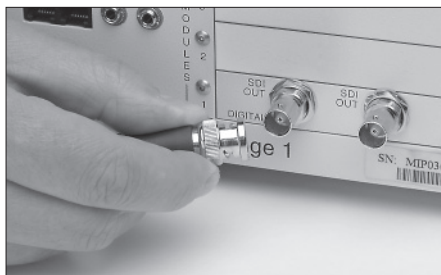Optional modules

Image 1 HUB™ HD with SDI* module
(22201020-102)

Connect the SDI video output connection ⑰ of Image 1 HUB™ HD to the monitor via the BNC cable (monitor with SDI module is required).

* SDI: Serial Digital Interface corresponds to a digitalized Y/C signal with a possible transmission cable length of up to 300 m

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Image 1 HUB™ HD with ICM (22201020-110)

A USB storage device can be connected to the USB output at the front of the CCU. The rear output is for connecting a printer. (Information on supported models is provided on request.)

The arrow in the two photos on the left shows where the USB storage device must be inserted.

This module is described in more detail from p. 59.

Image 1 HUB™ HD with ICM/SDI (22201020-112)

Complies with 22201020-110 with additional SDI module (please refer to above description).

Note: Printers and storage media with external power supplies should be isolated from the line voltage in accordance with IEC 60601-1. In this respect please observe the warning on page 8 (accessory equipment that can be connected to the analog and digital interfaces).

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**Connecting accessory device for remote
control**

If peripheral units such as a video recorder or printer are to be remotely controlled via camera or keyboard, the terminal for peripheral equipment remote control (Remote) must be connected up to sockets ACC1 and/or ACC2 on the camera control unit (suitable upon request).

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**Instructions on connecting
accessory devices**

Optimal image quality is achieved when the camera signal is transmitted to the monitor directly without any detour.

Documentation equipment such as printers or video recorders, on the other hand, can best be monitored when they are connected between the CCU and the monitor. However, intermediate equipment reduces the intensity of the original signal so image quality on the monitor is reduced by each piece of intermediate equipment.

The best solution in such a case is to use two monitors. Once the signal on the main monitor is taken from the CCU direct, a second connection is made from the CCU to the peripheral units and from there to the second monitor with which operation of the peripheral units is monitored.

If there is only one monitor available, it is recommended that the monitor be controlled directly (e.g. with the DVI connection) in order to achieve an optimal live image, and that a second cable connection (e.g. S-Video) be set up with the additional units and from there to the corresponding socket of the monitor. The corresponding input (DVI / S-Video) can then be selected using the on-screen settings menu.

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Connecting the keyboard

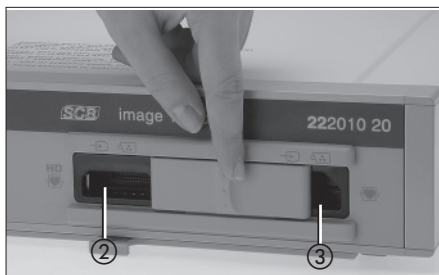
A keyboard is required to enter patient data and program the camera head. To connect the keyboard, plug the keyboard connecting cable into the keyboard receptacle 12 on the CCU's back panel.

For full details of keyboard functions and operations please see "Entering and Editing Patient Information" and "Customizing the Camera" sections of this manual.

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The Image 1 HUB™ HD CCU is equipped with two camera head connection sockets: one for HD camera heads (2), H3-M, H3-Z, H3-ZI, H3-P) and one for standard camera heads (3, SD) (F3, S3, S1, A3, A1, P3, P1, D1).

Note: The camera can only accept one head at a time.

To use a HD camera head, move the 'sliding door' on the front panel to the right, revealing the HD connection socket. To use a standard camera head, slide the door to the left, revealing the SD connection socket.

The Image 1 Pure HD CCU is equipped with a camera head connection socket (for the HD camera heads H3-M, H3-Z, H3-ZI and H3-P)

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Connecting the HD camera head Image 1 hub HD:

Note: Always ensure the camera head connector is completely clean and dry prior to insertion into the CCU.

Plug the camera head connector into the camera head cable receptacle (2) on the CCU's front panel.

Image 1 Pure HD:

Note: Always ensure the camera head connector is dry and clean before connection with the CCU.

Insert the camera head connector into the receptacle for the camera head cable on the front of the CCU housing.

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Connecting the SD camera head

Note: SD camera heads can be connected to the Image 1 hub HD but not to the Image 1 Pure HD.

Note: Always ensure the camera head connector is completely clean and dry prior to insertion into the CCU.

Plug the camera head connector into the camera head cable receptacle ③ on the CCU's front panel.

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Connecting SCB

Note: To avoid mistakenly pulling out the SCB connecting cable, the SCB connector possesses a protection device.

Pull back the protection device of the SCB connector and insert the connector into one of the SCB sockets ⑪. Connect the other end of the cable to the KARL STORZ-SCB® (KARL STORZ Communication Bus) control unit or other SCB units (see KARL STORZ-SCB® System Instruction Manual).

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Note 1: The USB port (ICM) to which a USB-printer is connected (in image) is optional#

- the following applies for Image 1 HUB™ HD (22201020-xxx):

xxx = 1 or 102 or 104 : no USB port
xxx = 110 or 112: includes USB port

- the following applies for Image 1 Pure HD (22202020-yyy):

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
Note 2: The video ports Comp-Video, S-Video and RGB, which are illustrated on the lower left part of the unit's rear panel, are only available with the Image 1 hub HD, and not with the Image 1 Pure HD

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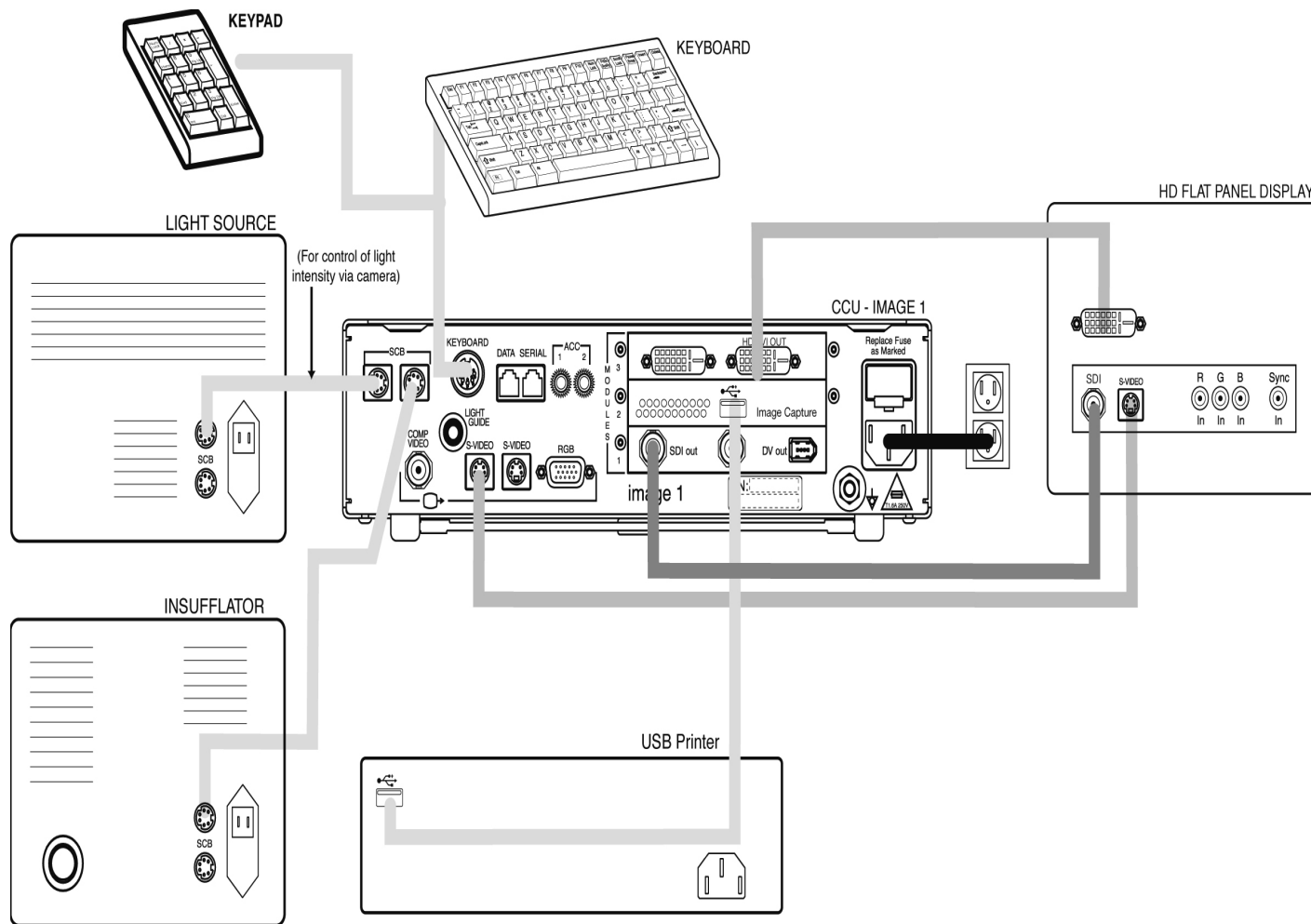Alternative connections between Image 1 HUB™ HD / IMAGE 1 Pure HD and AIDA control

Alternative connections between Image 1 HUB™ HD and AIDA control. The following table shows three alternative connection options between Image 1 HUB™ HD / **IMAGE 1 Pure HD** and AIDA control when the software version AIDA compact II V2.2 is installed on the AIDA control computer. One of the three connections can be used.

The table also show the five alternative connection options between Image 1 HUB™ HD / **IMAGE 1 Pure HD** and AIDA control when the software version AIDA compact II V2.3 HD is installed on the AIDA control computer. One of the five connections can be used.

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```
Options Menu
Exit
Camera Functions...
Patient Information...
SCB...
Setup Wizard (CCU) ▶
Setup Wizard (Camera Head) ▶
Image Capture Module *
```

*nur, wenn ICM vorhanden / *only if ICM is present / *sólo si existe ICM

```
Language Selection
Display (English)
Keyboard (English)
```

```
Next
Save and Exit
Quit Without Saving 1/3
```

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Powering up the CCU

Switch on the camera control unit ① and the video monitor.

Each time the CCU is switched on, color bars will appear on the video test screen for about 2 seconds. If the camera head is **not** plugged into the CCU correctly and completely, the color bars will remain on the monitor.

The color bars will remain visible until the camera head connector is properly and completely inserted into the CCU.

Warning: Test the video camera before each procedure. Ensure that the proper video image appears on all video monitors before beginning each procedure.

Note: The default setting for the screen display and keyboard is 'English'. This can be changed using 'Options Menu – Setup Wizard (CCU) – Language Selection' (8 languages are currently available). A detailed description can be found on p. 70)

Possible camera heads

Your Image 1 HUB™ HD can be equipped with any of the following camera heads:

- H3-P Pendulum camera head
- H3-Z Parfocal Zoom camera head
- H3-ZI Inline Parfocal Zoom camera head
- H3-M microscope camera head
- F3 camera head
- S1/S3 Parfocal Zoom camera head
- A1/A3 Parfocal Zoom camera head
- P1/P3 Pendulum camera head (see p. 32)
- D1 DCI® camera head (see p. 33)

Caution: When handling the camera, take care not to kink the cable or coil it too tightly. For example, do not coil the camera cable around the arm in small loops. Always coil or store the cable in loops at least 15 cm in diameter.

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Connecting endoscopes and light cables

Endoscopes can be connected to all camera heads listed above apart from the H3-M microscope camera head. Light cables are connected to the endoscopes (does not apply to DCI camera head). The connection of endoscopes and light cables is explained using the example of the Image 1™ HD H3-Z camera head.

The Image 1™ HD/SD single and three-chip camera heads have an integrated instrument coupler for the connection of standard endoscopes.

To do this, rotate the outer ring of the coupler towards the tension spring (in the direction of the arrow), insert the DIN endoscope eyepiece and release the outer ring. The tension spring enables the instrument coupler to automatically click into place.

Caution: No further tightening of the outer ring to position the eyepiece is required. This may cause damage to the instrument coupler!

Connect the light cable to the endoscope's ocular by twisting the thumbscrew on the light cable through one-quarter turn.

Note: The entrance and exit faces of the light cable and endoscope must be kept clean if optimal light transmission is to be maintained. Clean both faces with a cotton swab soaked in cleaning solution and then wipe with a cotton swab soaked in 70% isopropyl alcohol.

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**P1/P3 Pendulum camera head**

The camera head is designed so that, irrespective of resectoscope movement, it is always located in the 6 o'clock position.

This is particularly useful for procedures which involve frequent rotation of the endoscope. It is however also possible to position the camera head in any other direction (angle). Turning the CCD sensor correctly readjusts the image on the monitor.

If required, the camera can also be firmly attached to the eyepiece by activating the locking mechanism ⑳ (see arrow).

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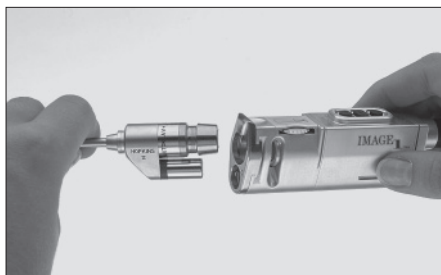
H3-P Pendulum camera head

Description: see P1/P3 camera head on this page.

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**D1 DCI® Camera Head
Connecting an endoscope**

Insert the DCI® **HOPKINS**® endoscopic telescope into the “snap-in” DCI® coupling mechanism. The endoscope will automatically lock in when the telescope is inserted.

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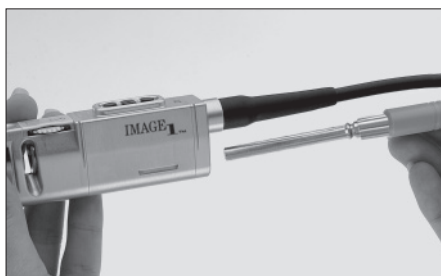
To release the telescope, press down the release mechanism ②6, and pull the telescope out of the camera head.

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Connecting a light cable

Connect special light cable 495 DC or 495 DV to the D1 DCI® camera head. To do this, push the longer end of the light cable connector into the D1 DCI® camera head until it locks into place.

Only grip the light cable by the handle piece. Never pull on the cable.

To disconnect the light cable, pull the cable out from the D1 DCI® camera head.

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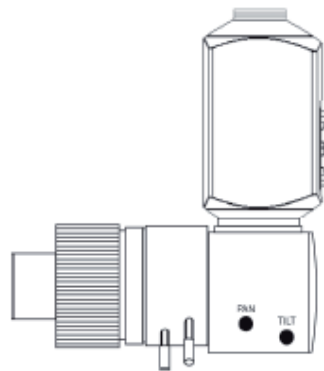
H3-M microscope camera head

The H3-M microscope camera head is connected to a suitable interface on the surgical microscope using a TV adaptor. Suitable interfaces are optical beam splitters and co-observer tubes. These options are available for all surgical microscopes. Alternatively, the H3-M microscope camera head can be directly connected using a special C-mount TV adaptor. This option is not available for all surgical microscopes.

Connecting the H3-M camera head to a surgical microscope

The H3-M camera head requires a TV adaptor. To attach a TV adaptor, gently screw the adaptor clockwise onto the camera head. Connect the camera head cable up to the camera head once the adaptor is installed on the camera.

Connecting and disconnecting the removable H3-M camera head cable

The H3-M camera head comes with a removable 9-meter cable as standard. To connect the camera head cable to the camera head, carefully align the cable key to the camera head receptacle and then gently screw the cable connector clockwise onto the camera head. To disconnect the camera head cable from the camera head, gently unscrew the cable connector counterclockwise.

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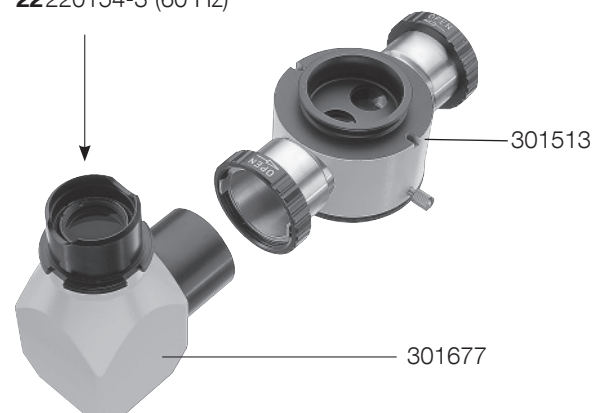
F3 camera head

Positioning of the buttons

The camera head buttons are positioned crosswise to the longitudinal axis of the camera head. You can use the left button to scroll up the menu tree, and the right button to scroll down. You can select a function in the menu tree using the center camera head button.

Calling up the User Menu for the first time

When you are supplied with the F3 camera head ex works and press the center camera head button, the User Menu will not appear. Proceed as follows to ensure you can call up the User Menu by pressing the center camera head button:

- Call up the Options Menu by pressing the center camera head button for 7 seconds.
- Scroll to the menu item 'Setup Wizard (Camera Head)'. Select this menu item by pressing the center camera head button.
- The next screen will show the cursor positioned at the string 'Next'. Select this function by pressing the center camera head button.
- The next screen will again show the cursor positioned at the string 'Next'. Select this function by pressing the center camera head button. The screen will now show the heading 'Program User Menu'.
- Press the right camera head button several times until the cursor is positioned at line '1. <None>'. Select this line for editing by pressing the center camera head button.
- Press the right camera head button several times until you find the function to be positioned as the first item of the User Menu. This might be, for example, the function 'White Balance'. Confirm your selection by pressing the center camera head button.

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- Scroll to the second line ('2. <None>'). Now select the required function for the second line of the User Menu (e.g. Accessory 1).
- Then scroll to 'Save and Exit' and press the center camera head button to confirm.
- If you now press the center camera head button, the User Menu will appear with the two functions selected (e.g. White Balance and Accessory 1).

Connecting an endoscope

The Image 1™ F3 camera head is equipped with an integrated instrument coupling system to connect standard endoscopes.

To do this, rotate the outer ring of the coupler towards the tension spring, insert the DIN endoscope eyepiece and release the outer ring. The tension spring enables the instrument coupler to automatically click into place.

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Connecting a light cable

Connect the light cable to the endoscope eyepiece by tightening the screw on the light cable through a quarter turn.

Note: The entrance and exit faces of the light cable and endoscope must be kept clean if optional light transmission is to be maintained. Clean both faces with a cotton swab soaked in cleaning solution and then wipe with a cotton swab soaked in 70% isopropyl alcohol.

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Adjusting the video monitor**(using the integral color test pattern generator)**

(These adjustments can be performed only if the monitor is connected to the CCU's S-Video or BNC output).

1. Activate color bars by turning on the CCU without a camera head connected. Color bars can also be activated through head buttons or User Menu if programmed to these controls. For instructions on programming, please see "Setup Wizard" or "Options Menu" sections of this manual.
2. Adjust the video monitor's brightness and contrast controls until seven vertical bars declining in brightness from left to right become visible across the upper portion of the test pattern.
3. Adjust the video monitor's chrominance control until the bars appear in color, with no interference effects visible along their edges.

Note: Some monitors have menus which offer setup options to optimize the video image. If selected, these set-up options automatically adjust the monitor's brightness, contrast, chrominance, and hue features. If your monitor is configured with these automatic set-up options, adjust the settings according to the instructions provided in the monitor's operating manual.

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Camera head button basics

The Image 1™ HD/SD camera head has the following three buttons for feature/function access, menu control, and programming:

- ▲ scroll up: Scrolls up the menus, options, etc.
- menu/select: Displays the User Menu. Once the User Menu is displayed, this button selects the highlighted option (identical to ENTER on a computer keyboard).
- ▼ scroll down: Scrolls down the menus, options, etc.

The ▲ button and the ▼ button can also be programmed to operate one function each (i.e. white balance and enhancement). For programming instructions, see “Customizing the camera – Options Menu” section of this manual.

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Keyboard basics

A keyboard is available to program and control camera features and functions. The keys which operate the menus function identically to the camera head buttons:

- | | |
|-------|-------------|
| ↑ | scroll up |
| ENTER | menu/select |
| ↓ | scroll down |

The keyboard is also used to input patient information. When inputting patient information, all text keys are functional and specific control keys are available. For more information, see “Operating instructions – Patient Information input or editing” section of this manual.

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Menu basics

Software menus are provided to program and control the camera features/functions.

The User Menu is accessed by pressing the middle button ■ on the camera head or ENTER on the keyboard.

The Options Menu is accessed by holding down the SHIFT key and pressing ENTER on the keyboard.

The Setup Wizards are accessed through the Options Menu.

The symbols used in the menus include:

- ▶ indicates that options are available with that particular menu item. To access the options, highlight the menu item then press ENTER on the keyboard or the center button ■ on the camera head.
- indicates that there are sub-menus within that menu item. To display the sub-menu(s), highlight the menu item then press ENTER on the keyboard or the center button ■ on the camera head.
- ▼ indicates that more menu items are available below those displayed.
- ▲ indicates that more menu items are available above those displayed.
- () indicates a selected value within the parenthesis.

For complete instructions on menu operation, see “Operating instructions – User Menu” and “Customizing the Camera” sections of this manual.

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Description of features/options

Key to Symbols

-  Signifies that this feature/option is available in the Setup Wizard (CCU).
-  Signifies that this feature/option is available in the Setup Wizard (Camera Head).
-  Signifies that this feature/option is available in the Options Menu
-  Signifies that this feature/option can be programmed into the User Menu
-  Signifies that this feature/option can be programmed to the camera head buttons

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Camera Functions

Capture image: Saves the image currently displayed on the screen.

Video recording: Starts a video recording.

Automatic Exposure Control: Patented system automatically sets exposure depending on the available light.

Brightness: Selection of image brightness, including the settings *Low, Medium, High, Peak, Small Scope A, Small Scope B*. The setting 'Peak' can be used to reduce the glare from small, bright objects in an otherwise dark endoscopic image.

Button Hints: Turns Button Hints display On or Off. If On, displays settings for the camera head buttons.

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Button Settings: Determines the programming for both the ▼ and ▲ camera head buttons simultaneously depending on a specific procedure and/or camera head (i.e. dual white balance for stroboscopy).

(Note: Additional options for standard button programming are available under “Program Head Buttons” feature).

Color bars: Turns the SMPTE color bar test pattern On or Off.

Date and Time Setup: Allows entry of date and time and selects display.

Display Type: Allows selection of either Universal or PVM-20M2MDU (Sony model number) monitor types to optimize the image.

Dual White Balance: Performs white balance for two light sources when used intraoperatively for stroboscopy procedures.

Enhancement: Electronically controls the level of contrast, including Off, Low, High and Fiberscope Filter settings. The digital fiberscope filter (anti-moiré/anti-grid filter is switchable in two stages (A, B)). This new electronic filter interfering artefacts, such as the moiré effect or display of the fiberscope grid, to be eliminated. The fiberscope filter is selected via the same menu as “Enhancement”.

Light Mode: Toggles between Light Source 1 and Light Source 2 for stroboscopy procedures.

Program Head Buttons: Allows select features/options to be activated by the camera head buttons ▼ or ▲.

(Note: Additional options for button programming are available under “Button Settings” feature).

Shutter (exposure control): Selects the camera exposure control, including automatic and seven-teen manual shutter speeds ranging from 1/60 – 1/17,000 second.

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Split Screen Enhancement (to check the contrast enhancement with SD camera heads): Divides the image horizontally, with the bottom half displaying Enhancement Off, and the top half displaying Enhancement Low, Medium or High. This feature can be turned On or Off.

White Balance: Determines the camera system's color controls to conform to the color temperature of the light source in use.

Specific functions for the H3-M microscope camera head

Auto White Balance: Automatically regulates camera color control to match it to the color temperature of the light source used.

Manual White Balance: Allows the user to manually regulate camera color control to match it to the color temperature of the light source used

Exposure: Regulates the exposure mode, exposure level and brightness.

Load User Settings: Loads the current user settings and/or factory-set defaults.

Save User Settings: Saves the current user settings.

Reset User Settings: Resets the user settings selected in the User Menu (including those made there before switching on the CCU). Any settings made in the Options Menu are not reset. - Example: The setting 'Enhancement' was changed from 'Low' to 'High' in the User Menu before switching on the CCU. This change is now cleared (reset).

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Factory Defaults Reset: Restores all settings back to the factory defaults. This means that all changes made in the User Menu and the Options Menu are reset.

Example: The Options Menu (menu item Setup Wizard (CCU)) was used to change the language setting for the screen display from English to German. In the Options Menu (item Setup Wizard (Camera Head)) the first function of the User Menu was changed from 'Auto White Balance' to 'Accessory 1'. The setting 'Enhancement' was also changed from 'Low' to 'High' in the User Menu before or after switching on the CCU.

If 'Factory Defaults Reset' is now enabled, the two changes in the Options Menu and the individual change in the User Menu will be undone.

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Definition of features/options

Patient information

Patient Information Display: Turns patient information display Off or On. On enables the user to view, edit, and enter patient information using the options listed below.

Patient Information Page Select: Selects page for display (1-10).

Clear Patient Information: Deletes text from all patient information pages.

Edit Patient Information: Allows entering or editing of patient information.

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SCB

SCB Text (On/Off): Turns SCB text display On and Off with the exception of SCB Warning/Status messages. This command overrides SCB parameter display set in "Configure SCB Display" below.

Configure SCB Display: Displays up to eight SCB devices for on-screen monitoring (four on the top of the screen and four on the bottom).

Note: "SCB Text On/(Off)" command overrides this function.

SCB Warnings/Status Display: Turns SCB "warning" and "status" text On or Off ("alarms" and "errors" cannot be turned off, as these involve personal safety of the patient or physician).

Note: "SCB Text On/(Off)" command does not override this function.

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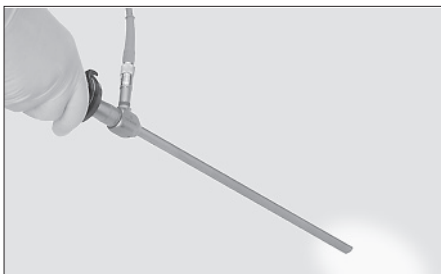
White Balance

The white balance determines the camera system's color controls based on the light source in use.

To perform this adjustment:

1. Turn on the light source and point the camera head (with attached endoscope) at a white surface, with no colored objects visible within the camera's field of view. The view should fill at least 70% of the area of the screen.
2. Activate the white balancing mode via the camera head buttons or keyboard (for complete instructions, refer to "Customizing the Camera" section of this manual). Once activated, "White balancing" appears on the monitor.
3. The white balance has been successfully performed when "White balance O.K." appears on the monitor. If white balancing has not been performed correctly, one of the following messages will appear:

"White balance fail – dark" Indicates that the field of view is too dark. Shorten the distance between the endoscope and the white surface. Activate white balance again.

"White balance fail – bright" Indicates that the field of view is too bright. Lengthen the distance between the endoscope and the white surface. Activate white balance again.

"White balance fail – color" Indicates that the color temperature exceeds the white balance control range.

Note: The last white balance setting remains stored even after the camera has been switched off. When the camera is next used, a new white balance should be performed in order to ensure optimal chrominance at all times.

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Dual White Balance

The dual white balance feature white balances two light sources when used intraoperatively (for stroboscopy). This feature can only be accessed from the head buttons when programmed via 'Button Settings' (for complete instructions, see 'Customizing the Camera – Options Menu' section of this manual). When the 'Button Settings' function (within the Options list) is set to 'Dual White Balance' one head button ▼ will perform white balance, the other head button ▲ will be assigned to light mode selection.

To perform the dual white balance function:

1. Turn on the light source and point the camera head (with attached endoscope) at a white surface, with no colored objects visible within the camera's field of view. The view should fill at least 70% of the area of the screen.
2. Press the head button ▲ assigned to 'light mode' selection. 'Light 1 Setting' or 'Light 2 Setting' will appear on-screen. To activate the white balance mode, press the other head button ▼. 'White balancing light 1 Setting' or 'White balancing light 2 Setting' will appear. After this white balance is completed, repeat the procedure to perform white balance on the other light source.
3. The white balance has been successfully performed when 'White balance O.K.' appears on the monitor. If white balancing has not been performed correctly, one of the following messages will appear:
'White balance fail – dark'
Indicates that the field of view is too dark. Shorten the distance between the endoscope and the white surface and activate white balance again.

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‘White balance fail – bright’

Indicates that the field of view is too bright. Lengthen the distance between the endoscope and the white surface and activate white balance again.

‘White balance fail – color’

Indicates the color temperature exceeds the white balance range.

Note: The last white balance setting remains stored even after the camera has been switched off. When the camera is next used, a new white balance should be performed if a different light source, light cable, or endoscope is used.

Note: After the two white balance settings have been stored in memory, pressing the assigned head button repeatedly will toggle between the two stored white balance settings.

Manual White Balance (H3-M)

The function ‘Manual White Balance’ is available in the User Menu of the H3-M microscope camera head. To execute this function, proceed as follows:

- Press the center camera head button so that the User Menu appears.
- Press the upper camera head button twice to scroll down so that the function ‘Manual White Balance’ is highlighted in white. Press the center camera head button.
- To adjust the red component, first press the center camera head button. The preset value for red is now highlighted in white (e.g., 320). Then press the upper camera head button or the lower camera head button (several times) to then select the value required (e.g., 400). Press the center camera head button to confirm.
- Then use the same procedure (as for red) to select the values required for green and blue.
- Finally highlight the line ‘Save and Exit’ in white and press the center camera head button to confirm.

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Shutter

(including automatic mode)

This camera features a patented auto exposure mode which controls the shutter automatically depending on the available light. The automatic exposure system may be overridden to select one of seventeen manual shutter speeds ranging from 1/60 to 1/17,000 second.

Caution: When using “Auto Exposure” mode, do not use an automatically controlled cold light source; this would cause control range overlap and thus a considerable reduction in image quality. Automatically controlled light sources may only be operated in the manual exposure mode.

Caution: Heat from the light source can cause chemical residue on the light cable and on the light entrance of the endoscope to burn. This can have a detrimental effect on the light conduction and hence the image accuracy.

Exposure settings can be changed through the “Shutter” camera function. This function can be programmed for control via the camera head buttons, User Menu or Options Menu. For information on programming, see “Customizing the camera” sections of this manual.

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Adjusting focus

Image sharpness can be adjusted by rotating the camera lens focus ring ⑳.

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Adjusting image size

Image size or focal length can be adjusted by rotating the zoom ring ㉑.

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Fokuseinstellung – H3-P Pendulum-Kamerakopf

siehe P1/P3

Adjusting focus – P1/P3 Pendulum camera head

Image sharpness can be adjusted by rotating the silver camera lens focusing ring ⑳.

Adjusting focus – H3-P Pendulum camera head

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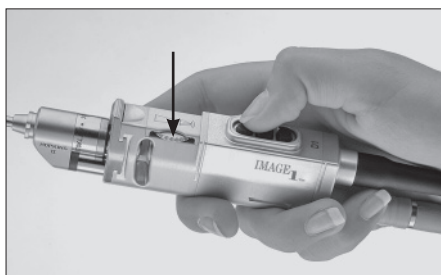
Adjusting focus – D1 camera head

Image sharpness can be adjusted by rotating the camera lens focusing ring ⑳ (see arrow).

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Patient information input or editing

To use the Patient Information pages a keyboard must be connected to the CCU.

The Patient Information screens include:

Patient Information Display:

The Patient Information screens should be On to enter, edit or view patient information. When turned On, two lines of patient information are available at the bottom of the screen.

Note: When Patient Information Display is On and contains text, the lower four KARL STORZ SCB parameters will not be visible on the monitor.

```
Patient Information
Exit
Patient Info. Display(On)/Off
Patient Info. Page Select ►
Clear All Pages
Edit Patient Information
```

Patient Information Page Select: To select one of ten pages available for patient information, scroll through the list of pages and select the desired page.

Clear Patient Information: Clears all stored text from patient information pages.

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Edit Patient Information: Patient Information can be entered or edited via the keyboard or its key assignments.

The INSERT key toggles between:

Overtyping – allows existing text to be typed over. Indicated by a flashing block cursor.

Insert – allows text to be inserted. Indicated by a flashing underscore cursor.

Note: When the cursor reaches the last character of the last line, it will remain in the same position. Select “next page” (see below) to insert additional text.

↑ or ↓ keys: Moves cursor between line 1, line 2, and the menu below.

In the Edit Patient Information screen, a menu is available at the bottom with options for:

Next - goes to next page.

Previous - returns to previous page.

Exit - saves information and returns to live image.

Use ↑ or ↓ on keyboard to highlight desired option and press ENTER to select.

Note: ↑ or ↓ buttons on the keyboard are not functional for menu operation while inputting or editing text.

Note: Patient Information pages are stored in the CCU and not the camera head. Therefore, if a new head is plugged into the CCU, the Patient Information text will remain.

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Setting Brightness or Enhancement

See 'Operating instructions – Description of Features/Options' section of this manual for a description of these functions.

For information on setting these functions, see 'Customizing the Camera' section of this instruction manual.

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User Menu (H3-Z/H3-ZI/H3-P)

The User Menu allows control of eight reprogrammable camera functions from the camera head buttons.

The User Menu is described here based on the example of the H3-Z camera head. The User Menus of other camera heads compatible with the CCU are similar. For example, the menus for the S3 camera head and the P3 camera head only additionally feature the function 'Split Screen Enhancement' and omit the Insufflator menu.

The User Menu for the H3-M microscope camera head is described separately from page 55 onwards.

To access this menu:

- from the camera head, press the center button ■ or
- from the keyboard, press ENTER

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```
User Menu
Exit
White Balance
Enhancement (Low) ▶
Brightness (Medium) ▶
Shutter (Auto) ▶
Button Hints On/(Off)
Program Head Buttons ▶
```

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This menu can be re-programmed to include eight of the following options:

Note: The following options apply to the H3-Z camera head.

Options list

- Camera functions
 - White Balance
 - Color Bars
 - Button Settings
 - Brightness
 - Enhancement
 - Shutter
 - Button Hints
 - Program Head Buttons
 - Light Source
 - Insufflator
 - Gas
 - Flow Control
 - Pressure Control
 - Volume Reset
 - Patient Information
 - Clear All Pages
 - Patient Information Display
 - Patient Information Page Select
 - Edit Patient Information
 - SCB
 - SCB Text
 - Configure SCB Display
 - SCB Warning/Status
 - Image Capture Menu
 - Capture Still
 - Capture Video
 - Print Now
 - <None>
- } Only available when enabled. See page 77.
- } Only available if a keyboard is plugged into the CCU.
- } Only available if ICM is present (USB output).

For full instructions as regards individual selection of the User Menu please see Chapter 'Setup Wizard (Camera Head)' from page 74.

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User Menu (H3-M)**(for H3-M camera head)**

Besides the function 'Exit', the User Menu contains a total of eight functions which can be executed with the H3-M camera head. The following list shows which eight functions are set on delivery.

User Menu

- Exit
- Auto White Balance
- Manual White Balance
- Exposure ►
- Enhancement (Low) ►
- Load User Settings (1)
- Save User Settings
- Program Head Buttons ►
- Color Bars On/(Off)

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List of options for User Menu (H3-M)

The list shows all options which are selectable/ programmable for the eight items of the User Menu (for procedure see next page).

- Camera functions
 - Auto White Balance
 - Manual White Balance
 - Color Bars
 - Exposure
 - Enhancement
 - Button Hints
 - Program Head Buttons
 - Load User Settings
 - Save User Settings
 - Reset User Settings
 - Patient Information
 - Clear All Pages
 - Patient Information Display
 - Patient Information Page Select
 - Edit Patient Information
 - SCB
 - SCB text
 - Configure SCB display
 - SCB warnings/status
 - Image Capture Menu
 - Capture Still
 - Capture Video
 - Print Now
 - None
- } Only available if keyboard is connected to CCU
- } Only available if ICM is present (USB outlet)

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Programming the User Menu (H3-M)

The User Menu provides for the set-up and control of eight camera functions using the camera head buttons.

The user can individually set the functions of the User Menu as required. For example, if he wishes to replace the second function 'Manual White Balance' with 'Accessory 1', he should proceed as follows:

- Scroll to 'Exit' in the User Menu and press the center camera head button to confirm.
- Hold down the center camera head button for 7 seconds to call up the Options Menu.
- Press the upper camera head button several times to scroll to 'Setup Wizard (Camera Head)' and press the center camera head button to confirm.
- The screen will now show the heading 'Program User Menu'.
- Press the upper camera head button several times to scroll to 'Manual White Balance'.
- Press the center camera head button to edit the item selected.
- Press the upper camera head button several times until the function 'Accessory 1' appears.
- Confirm this selection by pressing the center camera head button.
- Press the upper camera head button several times to scroll to 'Save and Exit'. Then confirm this function by pressing the center camera head button.
- Press the center camera head button briefly to call up the User Menu. The second function has now changed from 'Manual White Balance' to 'Accessory 1'.

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Default settings for the H3-M camera head of the operating microscope

For optimum monitor image quality, please take the following steps:

- Connect the camera control unit (CCU) of the Image 1 HUB™ HD with the HD monitor.
- Set the light intensity of the operating microscope to 70% of the maximum.
- Turn off the CCU of the Image 1 HUB™ HD.
- Connect the H3-M camera head with the CCU of the Image 1 HUB™ HD.
- Turn on the CCU of the Image 1 HUB™ HD.
- A message on the monitor indicates that the camera head has been recognized.

- Press the middle camera button briefly to display the user menu.
- Move the highlight bar to the menu item 'Exposure'.
- Press the middle camera button to select this item.
- Set the 'Mode' to 'Auto'.
- Set the value for 'Level' to '7'.
- Set the value for 'Brightness' to '2'.
- Save and exit.

- Press the middle camera button briefly to display the user menu.
- Move the highlight bar to the menu item 'Contrast enhancement'.
- Press the middle camera button to select this item.
- Set the 'Contrast enhancement' to OFF.

- Cover the beam path of the operating microscope with a white cloth.
- Press the middle camera button briefly to display the user menu.
- Move the highlight bar to the menu item 'Automatic white balance'.
- Press the middle camera button to select this menu item.

- Check the monitor for a message indicating that white balancing was successful.
- The monitor image quality should now be very good.

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ICM (Image Capture Module)

1. USB storage device and printer connection

NOTE: A USB storage device must be formatted to FAT 32 in order to store still images and/or video.

A USB storage device and/or a USB printer may be inserted/connected directly into the USB output on the front panel of the Image 1 CCU or the USB output on the back of the CCU at any time during a procedure. On-screen messages will temporarily appear on the surgical monitor indicating connection or disconnection. Only one storage device and one printer may be connected at the same time. If two USB storage devices or printers are connected at the same time, an error message will appear. If two printers are connected at the same time, both printers must be disconnected and one printer then reconnected in order to print.

NOTE: It is recommended to always have a USB storage device connected to ensure captured stills are not “lost” in the event of accidental powering off of the CCU prior to the completion of printing.

2. Starting a Capture Session

To start a capture session, power on the CCU, connect a camera head and a USB storage device and/or a USB printer. If a keyboard is attached, the patient information menu will appear.

NOTE: To start a capture session using a USB printer only, with no USB storage device connected, “Print During Capture” must be set to “On” in the Image Capture Menu. See “Print During Capture Feature” below.

3. Entering patient information (USB storage device and keyboard required)

A capture session may be started with or without first entering patient information. Entered patient information will be added to the folder name (stored to the USB storage device) that already includes the date and time of the capture session.

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After powering on the Image 1 CCU and inserting a USB storage device, if a keyboard is connected, the patient information menu will appear.

If a keyboard is not connected, this menu will not appear. Using a keyboard input the desired patient information. Use the Tab, Enter or Down/Up Arrow keys to move to the next field and hit Enter when “Exit” is highlighted to exit the menu.

4. Image Capture Menu

The Image Capture Menu provides access to the capture and print functions and to the Image Capture Setup menu.

To access the Image Capture Menu, hold the middle camera head button down for approximately 7 seconds, then use the down head button to scroll to “Image Capture Menu” and press the middle head button again to select. Alternatively, hit Shift+Enter on the keyboard and use the Down Arrow key to scroll down to “Image Capture Menu”, then hit Enter. Once in the Image Capture Menu, use the head buttons or keyboard Down/Up Arrows to scroll through the functions and press the middle head button or the Enter key to activate the desired function or to access the Image Capture Setup menu.

- **Capture Still:** Activates a still image capture. See “Capturing Still Images” section below for more information.
- **Capture Video:** Activates and deactivates recording of standard definition video. See “Recording Video” section below for more information.
- **Print Now:** Instantly prints all images in the print queue. The print queue size is based upon the selected number of prints per page (see below) in the Image Capture Setup menu.

NOTE: When the print queue is full and the next still is captured, the previous still images in the queue are cleared and are no longer available for printing. For example, if the prints per page is set to 4, and the 5th still is captured, only the 5th still will print as it is the only image now in the queue.

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- Image Capture Setup:

Before beginning a capture session, check the settings to insure that the system is set to the desired preferences. The Image Capture Setup menu allows selection of the following settings:

- Print During Capture: Selects the ability to print still images immediately after they are captured, and the selected prints per page layout is full. When set to “On”, this feature also enables still image capture with only a USB printer (no USB storage device) connected. See “Print During Capture Feature” below.
- Prints Per Page: Selects 2, 4, or 8 images to print on one page.
- Print Copies: Selects 1, 2 or 3 copies of each printed page.
- Paper Size: Selects paper size.
- Counter Display: Selects the display of the Print Counter and Video Counter below. The display options are Off (never display), Temporary (temporarily display upon each still image capture and video capture), or Permanent (always display).
- Print Counter: Displays the number of captured stills currently in the print queue based on the selected prints per page (above) in the lower left corner of the surgical monitor. Options are On or Off.
- Print Icon: Displays a graphic representation of the Print Counter (above) based on the selected prints per page (above) in the lower left corner of the surgical monitor. Options are On or Off.
- Video Counter: Displays the remaining minutes of video that will fit onto the connected USB storage device in the lower right corner of the surgical monitor. Options are On or Off.
- Video Marker: When recording 4:3 aspect ratio Standard Definition video using a High Definition camera head, an on-screen video marker may be set to display. The display acts as a “sight” to show the 4:3 video being recorded from the 16:9 surgical image.

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The user selectable options are Off, Corner-Transparent, Corner-Solid, Frame-Transparent or Frame-Solid.

NOTE: Messages generated by the Image Capture system could interfere with SCB messages that are displayed in the same location on the surgical monitor.

5. Capturing Still Images

The Image Capture Module captures high definition still images when used with a high definition camera head and captures standard definition still images when using a standard definition camera head. To capture still images, a USB storage device or a USB printer must be connected to one of the USB outputs on the Image 1 CCU. A USB storage device and a USB printer may also be connected at the same time.

NOTE: If using a printer only, with no USB storage device connected, “Print During Capture” must be set to “On” in the Image Capture Menu. See “Print During Capture Feature” under “Printing Still Images” below.

Still images may be “captured” via a camera head button programmed to capture still images (see the “Programming Head Buttons” section of this manual or the Image 1 Camera Head Instruction Card for more information on how to program camera head buttons). Still Images may also be captured via a keyboard or camera head buttons using the “Capture Still” command in the Image Capture Menu as described in the section above.

Once the desired image is on screen, press the programmed camera head button or use the “Capture Still” command in the Image Capture Menu to capture the image. The captured image will freeze momentarily on the surgical monitor. If “Print During Capture” is set to “On”, a graphic showing the status of the printed page/print queue will momentarily be displayed on the bottom left of the monitor. Still images may be captured while video is being recorded.

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6. Storing still images to a USB storage device

To store captured still images, a USB storage device must be connected to one of the USB outputs on the Image 1 CCU. A printer is not required to store images to a USB device.

All still images are stored to a USB storage device in .jpeg format. Captured still images may then be viewed on, and saved to, a computer.

NOTE: Only one USB storage device may be connected at one time.

7. Printing Still Images

To capture and print still images, a USB printer must be connected to one of the USB outputs on the Image 1 CCU. A USB storage device is not required to only capture and print (not store) still images.

Still images may be automatically printed as they are captured (See “Print During Capture Feature” below), or manually printed via a programmed head button or via a keyboard or camera head buttons using the “Print Now” command in the Image Capture Menu as described above. The number of prints per page and the 1, 2 or 3 copies of each page can be set in the Image Capture Setup menu as described above.

NOTE: The print queue is erased when the CCU is powered off. Be sure that all prints have been printed prior to powering off the CCU. It is recommended to always have a USB storage device connected to ensure captured stills are not “lost” in the event of accidental powerpowering off of the CCU prior to the completion of printing.

8. Print During Capture Feature

If “Print During Capture” is set to “On”, captured images will print immediately once a page is full (“full” is based on the selection of images per page set in the Prints Per Page option in the Image Capture Setup menu). Any printed pages which are not full when capturing is completed (and therefore have not printed) will print automatically upon removing the USB storage device. Setting “Print During Capture” to “On” also enables still image capture with only a USB printer (no USB storage device) connected.

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9. Capturing Video

The Image Capture system records Standard Definition video in 4:3 aspect ratio when using both High Definition and Standard Definition camera heads. To capture video, a USB storage device must be connected to one of the USB outputs on the Image 1 CCU. Long video captures are stored as separate 12 minute video files in mpeg4 format.

Video capturing may be started and stopped via a camera head button programmed to record video (See the “Programming Head Buttons” section of this manual or the Image 1 Camera Head Instruction Card for more information on how to program camera head buttons).

Video capturing may also be activated via a keyboard or camera head buttons using the “Capture Video” command in the Image Capture Menu as described in the section above.

When the desired scene is on screen, press the programmed camera head button or use the “Capture Video” command in the Image Capture Menu to start and stop video capturing. When video capturing is activated, a green circle icon will display on the bottom right of the screen. The green circle icon will remain on-screen until video capturing is stopped.

When using a High Definition camera head, a “Video Marker” may be set to display to act as a “sight” to show the 4:3 video being captured from the 16:9 surgical image. The Video Marker may be set in the Image Capture Setup menu as described in the section above.

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10. Capturing still images or video with optional footswitch

Still images or video may be captured via an optional Karl Storz single-pedal footswitch connected to the Acc 1 port on the CCU back panel. The footswitch must be configured to activate a still capture or to start/stop video capture in the CCU Setup Wizard. Use of a keyboard is recommended:

1. Go to the Setup Wizard (CCU).
2. Use the down arrow to highlight Next and press Enter to go to the CCU Defaults screen.
3. Use the down arrow to highlight Accessory Display Messages and press Enter.
4. Accessory 1 will be highlighted. Use the up arrow or down arrow to scroll through the available choices.
5. Acc 1-Still will configure the footswitch for still capture.
6. Acc 1-Video will configure the footswitch for video capture start/stop.

Please see the "Customizing the camera – Setup Wizards" section of this manual for more information.

NOTE: If the CCU Acc 1 output is connected to a Karl Storz AIDA image and video capture system via an accessory cable, the Acc 1-Still and Acc 1-Video will only activate still capture on the AIDA. No still image will be captured by the CCU Image Capture system.

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11. Ending a Capture Session

To end a capture session, power off the CCU, remove the camera head or remove the USB storage device. Disconnecting a USB printer will not end a capture session.

NOTE: The print queue is erased when the CCU is powered off. Be sure that all prints have been printed prior to powering off the CCU. It is recommended to always have a USB storage device connected to ensure captured stills are not “lost” in the event of accidental powering off of the CCU prior to the completion of printing.

12. Low USB Memory Warnings

Warning messages will be displayed at the bottom of the surgical monitor when a connected USB storage device has remaining memory space for less than 5 minutes of new video or less than 100 new still image captures. Both the video and still warning messages will display a countdown until there is no remaining memory space on the connected USB storage device, displaying the message, “USB Memory Full”. At that time, no additional video or still images can be stored to the USB Storage device.

NOTE: If the USB storage device memory is “Full”, and Print During Capture is set to “On”, still images may still be captured, but they will be sent to the connected USB printer only; they will not be stored on the USB storage device.

13. Reviewing Video and Still Images on a Computer

To view video or captured images on a computer, insert the USB storage device into the computer’s USB drive. Go to My Computer to access the USB drive. Open the “Karl Storz” folder to access the capture session(s) stored on the USB storage device.

The “Temp” file is stored on the USB storage device at the completion of a capture session. This is used by the system to end one capture session and set the system to begin another capture session. The “Temp” file contains no capture session data and may be safely deleted.

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Customizing Options

There are four options for customizing the camera:

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Setup Wizard (CCU)

The Setup Wizard (CCU) leads the user through a series of 3 screens to set the language used, CCU default settings (for text messages) and Date/Time. These settings will remain with the CCU when it is turned off and on again or when a different camera head is used.

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Setup Wizard (Camera Head)

The Setup Wizard (Camera Head) guides the user through different screens to set the camera head defaults (brightness and enhancement), to program the User Menu and, with the H3 and H3-Z camera heads, to enable/configure the insufflator functions. These settings will remain with the camera head when the CCU is turned off and on again or when a different CCU is used.

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User Menu

The User Menu offers eight programmable options which can be accessed from the head buttons. These options are programmed through the Setup Wizard (Camera Head).

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Options Menu

The Options Menu allows the user to activate or set camera functions for a specific surgical case. If the camera is turned off and on again, the default settings (as determined by user via Setup Wizards) will be restored. The Options Menu also allows access to the Patient Information sub-menu, the SCB sub-menu, and the Setup Wizard menus (used to set certain camera defaults).

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Camera Head Buttons

The ▲ and ▼ head buttons can be programmed to perform one function each (via the User Menu or the Options Menu).

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Setup Wizards Setup Wizard Access

Note: The following explanations for the Setup Wizard apply to all camera heads. The following menus apply to the H3-Z camera head.

Your sales representative is available to assist in using the Setup Wizards. Preferences for specific menu options, camera image settings and other features can be programmed within the Setup Wizards and will remain as defaults each time the camera is turned on. In many cases, the Setup Wizards will need to be set only once, with individual user adjustments made through the Options Menu or the User Menu.

To access the Setup Wizards, hold down the keyboard SHIFT key and press ENTER to open the Options Menu:

```
Options Menu
  Exit
  Camera Functions...
  Patient Information
  SCB...
  Setup Wizard (CCU) ▶
  Setup Wizard (Camera Head) ▶
  Image Capture Module *
  *only available if ICM module is present
```

Using the ↑ or ↓ key, highlight **Setup Wizard (CCU) or Setup Wizard (Camera Head)** and press ENTER. Each Setup Wizard will run through a series of screens to re-program the camera defaults.

At the bottom of each screen, options are provided as follows:

Next: saves what was selected on the screen and displays the next screen. This option is highlighted by default so user can quickly scroll through the screens.

Save and Exit: saves what was selected on all screens and exits back to image.

Quit without saving: exits back to image without saving selections.

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Setup Wizard (CCU)

The screen diagrams show default settings (i.e. in Language Selection screen, the default setting is "English"). The following menus apply to the H3-Z camera head.

Language Selection (screen display/keyboard switchover)

To change the language of the on-screen display and to identify the keyboard language:

- Call up the "Options Menu" (System Menu). For this, hold down the SHIFT key on the keyboard and press ENTER at the same time.
- Use the ↑ or ↓ keys to select the setup wizard CCU and activate with the ENTER key.
- Use the ↑ or ↓ key to highlight the word Display.
- Press ENTER to select.
- The highlighted area will move to the right (Language). Scroll through the list of languages using the ↑ or ↓ key (English, Deutsch, Français, Español, Italiano, Portugues, svenska, русский). When the required language is on-screen, press ENTER. The highlighted area will move back to the menu on the left (Display/...).
- To display the language used for the keyboard, highlight the word Keyboard and press ENTER.
- Select language, as in 5 above.
- To save, select "Save and Exit" and confirm with ENTER.*

* To proceed to the next screen, highlight Next and press ENTER.

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CCU Defaults

To change a setting:

1. Highlight the option to re-set. Press ENTER.
2. A value to the right of the selected option is now highlighted. Scroll through the additional values using the ↑ or ↓ key. The available values for each option include:

Button Hints: On, Off.

SCB Text: On, Off

(Note: The function SCB-Text: On, Off have priority over other displays with the exception of the SCB-Warning/-Status displays).

Display Type: Universal, PVM 20M2MDU.

Sesam Display: The Sesam™ function provides for speech control of the OR1^M (example: The spoken command 'Start recorder' will start the recorder). The function 'Sesam Display' enables/disables overlay of the speech tree on the Image 1 HUB™.

HD Text mode: 4:3 or 16:9

Note: For a complete description of these options please see the Chapter 'Description of features/options' in this manual (p. 41)

3. When the desired value is on-screen, press ENTER. The highlighted area will move back to the menu on the left for additional changes.

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Date and Time Setup

To program the date/time:

1. Select a category (Year, Month, Day, Hour, or Minute) using the ↑ or ↓ keys. Press ENTER.
2. The ↑ or ↓ keys will now allow you to scroll through options for each category (i.e. "Month" will have values from 1-12). Once the desired value is on-screen, press ENTER. The highlight will now return to the list of categories for further changes.

To change the time/date Display:

1. Highlight "Display" and press ENTER.

The ↑ or ↓ keys can now be used to scroll through the following options:

None
Time HH:MM
Date YYYY-MM-DD
Date MM-DD-YYYY

2. Using the ↑ or ↓ keys, choose the desired display and press ENTER.

This is the last screen in the Setup Wizard (CCU).

To make additional changes, highlight **Next** and press ENTER to scroll through Setup Wizard (CCU) screens.

To exit, highlight **Save and Exit** (or **Quit without Saving**) and press ENTER. All menus will disappear from the screen and the camera image will be restored.

Date and Time Setup	
Program Date/Time	
Year	(2007)
Month	(1)
Day	(1)
Hour	(1)
Minute	(00)
Display	(None)
Next	
Save and Exit	
Quit Without Saving	3/3

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Setup Wizard (Camera Head)

Note: The Setup Wizard (Camera Head) is only accessible if a camera head is connected to the CCU.

Note: The information given under the heading 'Setup Wizard (Camera Head)' applies to all camera heads except for the H3-M. The information for the H3-M can be found on p. 55-58. The insufflator function only applies to the H3-Z-, H3-ZI and the H3-P camera heads.

Camera Head Defaults

To change a setting:

1. Highlight the option to re-set. Press ENTER.
2. A value to the right of the selected option is now highlighted. Scroll through the additional values using the ↑ or ↓ key. The options have the following settings:
Brightness: Low, Medium, High, Peak, Small Scope A, Small Scope B.
Enhancement: Off, Low, High, Filter A or B.
Note: For a complete description of these options please see the Section 'Description of features/options' on p. 74.
3. When the desired value is on-screen, press ENTER. The highlighted area will move back to the menu on the left for additional changes.

To proceed to the next screen, highlight Next and press ENTER.

```
Camera Head Defaults
Brightness (Medium)
Enhancement (Low)

Next
Save and Exit
Quit Without Saving (1/3)
```

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Program User Menu

To change the User Menu (the control of features from the head buttons):

1. Highlight the item number to re-program. Press ENTER.

Program User Menu

1. White Balance
2. Enhancement
3. Brightness
4. Capture image
5. Video recording
6. Shutter
7. Button Hints
8. Program Head Buttons

Next

Save and Exit

Quit Without Saving (2/3)

2. The ↑ or ↓ keys can now be used to scroll through the following options:

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Note: For a complete definition of these options, please see 'Operating Instructions – Description of Features/Options' section of this manual.

3. Once the desired option appears on-screen, press ENTER. Other User Menu items can be reset using the same method.

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Configuring Insufflator menu

```

Configuring Insufflator menu
Gas (On/Off)          Enabled
Flow Control          Expanded (L/M/H)
Pressure Control      Expand. (8/12/15/18)
Volume Reset         Enabled

Next
Save and Exit
Quit without Saving  (3/3)
    
```

Activation = Precondition for display in Options List

Note: The above four items of the Insufflator menu only appear in the Options List (see previous page) if they have been enabled. To enable them, perform the following steps:

- Select Options Menu (to do so hold down center camera head button for 7 seconds).
- Select Setup Wizard (Camera Head).
- Press 'Next' twice.
- Select 'Gas (On/Off)'.
- Press center camera head button.
- The cursor is now also positioned at 'Disabled'. Set the field to 'Enabled' by pressing the lower camera head button and then press the center camera head button to confirm.
- Follow the same procedure to enable the functions 'Flow Control', 'Pressure Control' and 'Volume Reset'
- Select the field 'Save and Exit' and press the center camera head button.

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Setting options in Insufflator menu

Gas (On/Off)
Enabled
Disabled
Flow Control
Basic (L/H)
Expanded (L/M/H)
Disabled
Pressure Control
Basic 8/15 mmHg
Expanded 8/12/15/18 mmHg
Expanded 5/8/10/12 mmHg
Disabled
Volume Reset
Enabled
Disabled

If a function has been disabled, it will no longer be available for selection in all menus.

This is the last screen in the Setup Wizard (Camera Head). To make additional changes, highlight Next and press ENTER to scroll through Setup Wizard (Camera Head) screens.
To exit, highlight **Save and Exit** (or **Quit without Saving**) and press ENTER. All menus will disappear from the screen and the camera image will be restored.

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Options Menu Access

Once the Setup Wizards functions have been set to the desired defaults, the Options Menu can be accessed to easily adjust or activate certain camera functions for a specific surgical case. If the camera is turned off and on again, the selected default settings from the Setup Wizards will be restored. The Options Menu also allows access to the Patient Information sub-menu and the SCB sub-menu.

To bring up the Options Menu, hold down the SHIFT key and press ENTER on the keyboard. The following menu will appear:

```
Options Menu
Exit
Camera Functions...
Patient Information...
SCB...
Setup Wizard (CCU) ▶
Setup Wizard (Camera Head) ▶
Image Capture Module*
*only available if ICM module is present
```

To select an option, use the ↑ or ↓ keys to highlight this option and press ENTER.

Below is a description of each option and its operation.

Options Menu

Camera Functions

When selecting Camera Functions, the following menu will appear:

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Camera Functions
Exit
White Balance
Color Bars On/(Off)
Head Buttons sett. (Standard) ▶
Brightness (Medium) ▶
Enhancement (Low) ▶
Shutter (Auto) ▶
Button Hints On/(Off)
Program Head Buttons ▶
Light source
Insufflator

1. Highlight a function from the following list:

White Balance: performs White Balance.

Color Bars: turns the color bars On or Off.

Button Settings sets both ▲ and ▼ head buttons
simultaneously as follows:

Default: ▲ (value) ▼ (value)

Dual White Balance: ▲ (light mode)
▼ (White Balance)

Brightness: selects Low, Medium, High, Peak,
Small Scope A, Small Scope B.

Enhancement: selects Off, Low, High, Filter A or
B.

Shutter: selects auto exposure or speeds from
1/60 - 1/17,000 second.

Button Hints: turns head button labels On or Off.
(To select labels for accessory devices, please see
'Setup Wizard (CCU) – CCU Defaults' section of
this manual.)

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Program Head Buttons: sets the ▲ (Up button) and ▼ (Down button) on the camera head to control the following functions:

Light source: If a light source is connected via SCB, this light source can be controlled.

Insufflator: Used to control insufflator (Gas, Flow, Pressure, Volume Reset)

Camera Functions

- White Balance
- Color Bars
- Button Settings
- Brightness
- Enhancement
- Shutter
- Button Hints
- Program Head Buttons
- DV (with integrated DV module)
- Split Screen Enhancement
(SD camera head only)
- Light Source
- Insufflator
- Gas
- Flow Control
- Pressure Control
- Volume Reset
- Patient Information
- Clear All Pages
- Patient Information Display
- Patient Information Page Select
- Edit Patient Information
- SCB
- SCB Text
- Configure SCB Display
- SCB Warnings/Status
- Image Capture Menu
- Capture Still
- Capture Video
- Print Now
- <None>

Only available when enabled (activation see page 77).

Only available if a keyboard is plugged into the CCU.

Only available if ICM is present (USB outputs).

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Split Screen Enhancement

(only available with SD camera heads)

allows image to be split horizontally to display Low or High Enhancement settings on top and no Enhancement on bottom.

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2. Press ENTER to select.
For functions followed by a ►, use the ↑ or ↓ to scroll through available options.
Upon pressing ENTER, the menu will disappear and the camera image will reappear on the monitor.

Note: Patient Information option is not available unless a keyboard is plugged into the CCU.

Patient Information

When selecting Patient Information, the following menu will appear:

```
Patient Information
Exit
Patient Information Display
      ON/ (Off)
Patient Information
      Page Select (1) ►
Clear All Pages
Edit Patient Information
```

1. Highlight a function from the following list:
Patient Information Display:
turns the display On or Off.
Patient Information Page Select:
selects page from 1 – 10.
Clear All Pages:
clears all patient information text.
Edit Patient Information:
allows text input or editing.

Note: Patient Information is stored in the CCU and not the camera head.

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2. Press ENTER to select. For functions followed by a ►, use the ↑ or ↓ to scroll through available options. Upon pressing ENTER, the menu will disappear and the camera image will reappear on the monitor.

SCB

Note: The SCB function for the H3-M camera head will be included in a subsequent release. The SCB function is integrated in all camera heads compatible with the Image 1 HUB™ HD, except for the H3-M camera head.

Note: SCB information is saved in the CCU and not the camera head.

When selecting SCB, the following sub-menu will appear:

```
SCB
Exit
SCB Text (On)/Off
  Configure SCB Display ►
SCB Warnings/Status
      (On)/Off ►
```

SCB text

The SCB Text option turns the on-screen text On or Off. This function overrides the “Configure SCB Display” option but not the “SCB Warnings/ Status Display”.

I.e., if the SCB text “Off” and SCB Warnings/ Status “On” settings are selected, SCB Warnings/ Status messages are displayed on the screen, but no equipment displays from the “Configure SCB Display” sub-menu.

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Configure SCB display

The Configure SCB Display option allows selection of up to eight SCB devices/settings for display (i.e. insufflator, gain, etc).

To select the device parameters for display:

1. Highlight the Configure SCB Display option and select by pressing ENTER on the keyboard or the center button on the camera head. The devices connected to the SCB system will be displayed on-screen, for example:

```
SCB
  Exit
  SCB Text (On)/Off
  Configure SCB Display ►
  SCB Warnings/Status
      (On)/Off ►
```

SCB Display (Devices)

```
Image 1
Insufflator
Light Source
(Up to 8 devices specified)
Exit
```

The display will only include the units connected in the SCB network and switched on.

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2. Select the device to be displayed. A sub-menu of the device parameters will appear on-screen. For example, if Image 1™ is selected...

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SCB Display (Devices)

Image 1
Insufflator
Light Source
(Up to 8 devices specified)
Exit

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SCB Display (Image 1)

Shutter Speed (Off?1-8)
Brightness (Off?1-8)
Enhancement (Off?1-8)
Devices...
Exit

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3. As an example, to display the Brightness setting on the monitor, select Brightness by highlighting and then pressing ENTER on the keyboard or the center button on the camera head. The cursor will move to the options on the right.

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SCB Display (Image 1)

Brightness	(Off, 1-8)
Enhancement	(Off, 1-8)
Devices...	
Exit	

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Use the ↑ or ↓ to scroll through the following list of options: Off, 1, 2, 3, 4, 5, 6, 7, 8. The Off option turns the Brightness display off, and the options 1 – 8 represent the following regions on-screen:

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Note: If the Patient Information Display was turned On and contains text (see “Patient Information” in “Customizing the Camera – Options Menu” section of this manual for more information), only four SCB regions can be displayed (regions 1, 2, 3 and 4).

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4. Use the ↑ or ↓ to scroll through the list and select the desired option/region by pressing ENTER on the keyboard or the center button on the camera head.

Note: If a device parameter is already displayed in the selected region, that parameter will be automatically forwarded to a blank region.

5. Once selected, the highlighted area will move back to the sub-menu on the left side to continue configuring the display for that device. After completing the display options for that device, select Devices... to return to the previous screen (SCB Display (Devices)) to set the display for other devices. The menu item Exit saves the settings; the menu disappears and the camera image appears on the monitor again.

Note: The setting from “SCB Text On/Off” overrides the “Configure SCB Display” menu. I.e., if the “Off” setting is selected for SCB Text, no device displays appear on the screen.

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SCB Warnings/Status Display

The SCB Warnings/Status Display option on the SCB menu turns this on-screen Warnings/Status display On or Off.

```
SCB
  Exit
  SCB Text (On)/Off
  Configure SCB Display ►
  SCB Warnings/Status
      (On)/Off
```

- To change this option from its current status (indicated in parenthesis on-screen):
1. Highlight SCB Warnings/Status Display.
 2. Press ENTER to change status. A message will appear on-screen to confirm your selection.

Note: “SCB Text On/(Off)” command does not override this function.

3. Once confirmed, the image will return to live.

Exit

Exit removes the menu and restores the camera image to the monitor.

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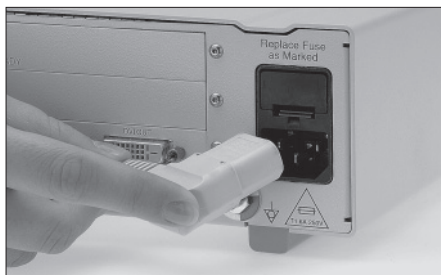
Fuse replacement

Switch off the unit and remove the power plug from the electrical outlet.

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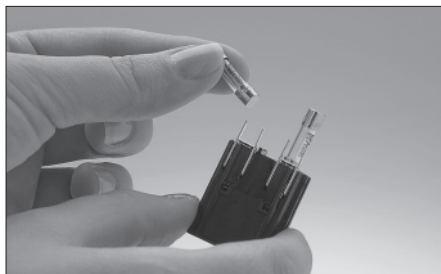
Remove the power fuse holder with a screwdriver or other tool.

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Insert new fuses of the appropriate rating.

Caution: Only use fuses of the correct rating.

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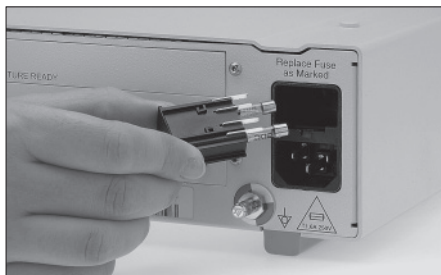
Refit the power fuse holder.
Reconnect the power cord.
Test the unit for proper operation.

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General care

Warning: Always pull out power plug before cleaning!

Store the camera control unit (CCU) and camera head in their protective case. Do not coil the camera head cable into tight loops or cause the camera head cable to kink while storing.

Do not store the CCU and the camera head in direct sunlight or excessive heat.

When returning the camera head and/or optical coupler for service, the user must ensure the equipment being returned is clean and disinfected.

The light entrances and exit faces of the light cable and the endoscope must always be kept clean (please refer to the corresponding instruction manual) to ensure optimal reproduction of the endoscopic image.

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Caution: Chemical residue on the card-edge connector can cause camera failure.

Caution: Heat from the light source can cause chemical residue on the light cable and on the light entrance of the endoscope to burn. This can damage the light fibers and have a detrimental effect on the lighting power and the picture quality.

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Care and handling

To maximize the life of your camera, please follow the guidelines given below for proper handling and care of the camera head.

DO NOT PLUG THE CAMERA HEAD CABLE INTO THE PROCESSOR RECEPTACLE IF IT IS WET. Always ensure the connector is completely dry and free of debris. Wipe off any moisture with a lint-free disposable cloth before plugging it in.

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DO NOT PICK UP THE CAMERA HEAD SET BY ITS CABLE. Always pick up the set by the camera head first, followed by the cable and connector. The cable contains tiny electronic conductors and should be treated gently and carefully.

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PROTECT THE CAMERA SYSTEM FROM DIRECT SUNLIGHT OR EXCESSIVE HEAT.

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HANDLE THE SYSTEM WITH CARE, avoiding extreme impacts to device. When transporting, use the original shipping box to prevent damage to the device.

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DO NOT STORE THE CAMERA WITH CABLE COILED TIGHTLY OR GATHERED INTO FOLDS. Always arrange the cable in loose coils of 6 inches (15 cm) or more in diameter.

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DO NOT RAPIDLY PULL THE CABLE WHEN UNCOILING. Harmful "kinks" can result. Always allow the cable to uncoil gently and loosely.

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DO NOT AUTOCLAVE THE CAMERA HEAD unless it is an autoclavable camera head.

The camera head may only be sterilized using a low-temperature method with ETO/FO gas at a temperature not exceeding 60°C (140°F) or using the STERIS® or STERRAD® method, according to the manufacturer's instructions.

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Caution: The H3-M head and its cable must not undergo preparation or sterilization. Wipe-down disinfection only! (p.93)

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WHEN DRYING, DO NOT PULL THE CABLE, AS THIS MAY RESULT IN BUNCHING OF THE CABLE JACKET.

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Note: Use of the term 'camera head assembly' means either a camera head assembly with integrated optical adaptor (lens), a screw mount camera head with or without optical adaptor, or an optical adaptor by itself.

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Note: We strongly recommend drying using sterile compressed air following manual preparation and, if required, after machine preparation.

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Maintenance**Cleaning, disinfection and sterilization**

Caution: Inspect the camera cable for breaks and cracks before and after preparation and before re-use. Breaks or cracks will allow fluids to enter the camera cable and cause damage. Camera heads with damaged cables should not be prepared but returned to KARL STORZ for repair.

Warning: The KARL STORZ camera head and lens should be thoroughly cleaned and disinfected/sterilized according to validated preparation procedures prior to use and subsequent reuse.

Caution: The H3-M camera head should only be subjected to wipe-down disinfection. Preparation and sterilization are forbidden (p. 93). In addition, please observe the instructions for the sterile covers at the bottom of page 141.

Warning: Any deviations from the recommended procedures for cleaning and disinfection/sterilization should be validated by the user.

Warning: The following procedures alone do not guarantee decontamination or sterility. Decontamination or sterility can only be determined by the staff when preparing recognized and validated disinfection and sterilization methods.

Caution: Neither the camera nor the optical coupler assembly are to be treated with steam and/or with pressure (exception: Image 1™ camera heads A1 and A3).

Note: The use of other disinfection/sterilization methods than those specified in the sections above may cause adverse effects on the materials of the camera and instrument coupling. Resulting damage will not be covered by the warranty.

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(Preliminary) cleaning

Place the camera head, lens and camera head cable in a plastic basin filled with a cleaning solution (according to the chemical manufacturer's instructions) immediately after use to prevent contaminants from coagulating and drying onto the surface of the instruments.

Water quality requirements

Tap water can be used for preparing the cleaning and disinfectant solution; however, completely demineralized water is preferred.

Microbiologically pure/sterile water must be used to rinse (final rinsing) after exposure.

Wipe-down disinfection

This can be performed using a lint-free disposable cloth moistened with disinfectant (no solutions predominantly containing alcohol).

Caution: The CCU and H3-M camera head should only be wipe-down disinfected (see table on page 93).

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Manual preparation

Warning: When cleaning and disinfecting contaminated instruments, the local guidelines of the Employer's Liability Insurance Association and equivalent organizations are always to be observed.

Cleaning the camera control unit

Caution: Always remove the power plug before cleaning the camera control unit.

The camera control unit must be cleaned by wiping it down with a lint-free disposable cloth moistened with disinfectant (no solutions predominantly containing alcohol).

Caution: Fluid must not be allowed to enter the camera control unit.

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**Cleaning and disinfecting the camera
head and lens**

Caution: The H3-M camera head should only be wipe-down disinfected (see p. 93)!

1. Completely immerse the camera head (including cable) and lens in a cleaning solution (according to the chemical manufacturers' instructions).
2. Remove any soiling from the camera head and lens with a disinfected soft brush while taking care to avoid splattering. Cleaning accessories which have been designed for the materials and sensitive surfaces are available from KARL STORZ (see catalog).
3. Rinse the camera head (including cable) and lens, and inspect them for residual dirt. Ensure that the camera plug is thoroughly clean.
4. Clean and disinfect any cleaning accessories (brushes etc.) after manual preparation.
5. Place the camera head (including cable) and lens in a suitable basin with disinfectant (concentration and exposure time according to manufacturer's instructions), making sure that they are completely free of air bubbles. Use plastic basins so as to avoid scratching the instruments and to prevent electrolytic corrosion. After exposure, rinse with microbiologically pure/sterile water. Use unsoiled, sterile water for each rinse. Rinse the camera head and lens several times thoroughly to remove residue which, among other things, could effect sterilization.
6. Dry carefully with sterile compressed air (max. pressure 0.5 bar). Clean the glass windows with a cotton swab moistened with 70% isopropyl alcohol to remove smears and residual moisture.
7. After preparation, inspect the camera head and/or lens and camera head cable for cleanliness and any damage.

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Maintenance**Cleaning, disinfection and
sterilization**

Caution: Inspect the camera head cable for breaks or cracks. A break or crack in the cable will allow liquid to enter the camera cable and cause damage. Camera heads with damaged cables should not be prepared but returned to KARL STORZ for repair.

8. Before sterilization, coil the camera head cable into loops at least 15 cm in diameter.
Do not kink or twist the cable.
We recommend using the camera tray 39301 ACTS (autoclavable sterilization container) for the A1/A3/S1/S3 camera heads resp. 39301 Z3TS for the H3-Z camera head.

Note: To ensure the longest possible camera life, we recommend using sterile covers.

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Machine preparation of autoclavable camera heads

Caution: Of the camera heads compatible with the Image 1 HUB™ HD, only the A1 and A3 are autoclavable (p. 93).

If possible, machine thermal disinfection (with temperature and cleaner) should be carried out, followed by steam sterilization at 134°C. The use of a machine for preparation offers the advantage of reducing the risks to personnel and standardizing the cleaning process (processes which can be validated or which are validated are required).

Note: As there is sometimes strong movement of the water, it is necessary to secure all parts in holding devices to avoid damage.

Note: It is not advisable to use lubricants etc. in the machine because they may cause deposits to form on the instruments and may damage plastics. Selective manual care and lubrication of joints etc. following the preparation and prior to sterilization is safer.

Use a cleaning/disinfecting agent approved by KARL STORZ for machine preparation (see Appendix, page 129, 130).

Note: Due to the possible cosmetic changes to the camera head coating, the use of enzymatic, neutral cleaners is recommended.

Warning: Only KARL STORZ camera heads which are designated as autoclavable can be prepared in devices for semi or fully automatic cleaning and disinfection. Preliminary cleaning as described above must also be carried out for machine preparation. The choice of rinsing programs must be agreed with the machine manufacturer. In most cases, special programs for endoscopic instruments are available which are suitable for cleaning without damaging the surface. It should be noted that, during machine preparation, autoclavable camera heads must not be heated above 93 °C (device-specific).

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Warning: Before the camera heads are placed in the cleaner/disinfector, all parts should be manually pretreated according to the cleaning instructions, regardless of whether or not they were used.

Warning: The cleaner/disinfector must fulfill all the demands made of it in terms of process validation, cleaning performance, disinfection and material compatibility.

Caution: Constant changing between various preparation methods is especially hard on the materials and must therefore be avoided. However, a one-off change of method, due to procurement of new cleaning equipment, is possible.

Caution: Always check the camera head and cable for external damage prior to cleaning in semi- and fully automatic machines. Lay the camera head carefully in the tray of the cleaner/disinfector. Use the rinsing program for endoscopic instruments according to the manufacturer's instructions.

Caution: Once the program has ended, the camera heads are subject to a high degree of corrosion due to the ambient conditions in the cleaning machine. Preferably, remove the camera heads from the cleaner/disinfector immediately after the program has finished.

Caution: Following machine cleaning, clean the distal lens, eyepiece and camera connector with a cotton tip applicator soaked in lens cleaner or 70% isopropyl alcohol. Always wipe the distal lens clean using circular movements.

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Sterilization instructions for non-autoclavable camera heads

Caution: The H3-M camera head and its cable must not undergo preparation or sterilization. Only wipe-down disinfection is permitted (see p. 93)!

Routine ethylene oxide (EtO) sterilization is recommended for initial and subsequent sterilization of all non-autoclavable camera head units. They may also be STERIS® or STERRAD® sterilized. To achieve the desired sterility assurance level (SAL) of 10⁶, KARL STORZ recommends the following EtO, STERIS® and STERRAD® sterilization methods.

Caution: Do **not** steam sterilize any part of the camera head or optical adaptor; irreparable damage will occur.

Caution: Sterilization can only succeed on a cleaned and disinfected camera head unit.

Caution: The recommended sterilization parameters are only valid with sterilization equipment that is properly maintained and calibrated.

If the optical adaptor is detachable (i.e., C-mount), remove it from the camera head prior to cleaning, disinfection, or sterilization.

Gas sterilization with ethylene oxide (EtO)

1. Clean the camera head unit as described in the preparation section of this instruction manual. Place the camera head unit in a sterilization tray.
Do not place any instruments on top of the camera.
2. All sterilization cycles must include a pre-conditioning cycle:
Temperature: 54 ± 2°C/
Time: 30 minutes
Humidity: ≥ 70% (relative humidity)
Vacuum: 1.3 psia

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3. KARL STORZ has validated EtO sterilization using the following parameters:
- | | |
|----------------------|---------------------------|
| Gas mixture: | 100% EtO |
| Temperature: | Set point of 55°C |
| Relative humidity: | ≥ 70% RH |
| Humidity dwell time: | 30-45 minutes |
| Exposure time: | 180 minutes, (full cycle) |
| EtO concentration: | 735 ± 30 mg/l |
4. Airing can be accomplished in an airing chamber at a temperature of 50 - 54°C for 12 hours.
- Maximum permissible levels of residue following EtO sterilization (according to ANSI/AAMI/ISO 10993-7):
- | | |
|------------------------|--------------|
| Ethylene oxide: | 4 mg (< 24h) |
| Ethylene chlorohydrin: | 9 mg (< 24h) |

NOTE: Sterilization methods which deviate from the validated processes described must be validated individually. The country-specific and sterilization device-specific requirements and outline conditions concerning the effectiveness of the sterilization must be taken into account, as must, where applicable, the necessary airing times.

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Gas sterilization

Sterilization with the System STERRAD® 100S

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Sterilization with STERRAD® 100S system

The STERRAD® 100S Sterilization System utilizes a synergism between hydrogen peroxide and low temperature gas plasma to produce a rapid (75 minutes), low temperature (10 - 60°C [50 - 140°F]), low moisture inactivation of microorganisms.

Caution: STERRAD® sterilization may cause cosmetic changes to the camera head assembly that do not necessarily impact the camera's functionality.

Caution: The camera head assembly must be thoroughly dried before loading into the STERRAD® sterilization chamber. Loads containing moisture may cause a cycle cancellation.

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Caution: Any deviations from the recommended STERRAD® 100S system sterilization parameters must be validated by the user.

1. Clean and prepare the camera head unit as described in the preparation section of this instruction manual. Be sure that the camera head assembly is completely dry.
2. Load the camera head assembly into the STERRAD® sterilizer, arranging it so that the hydrogen plasma can surround it. Do not allow the camera head assembly to touch the wall of the sterilizer.
3. Press the START button. The door will automatically close and the preset automatic cycle will begin. The complete cycle runs in approximately 75 minutes.
4. On completion of the cycle press the OPEN DOOR button and remove the camera head assembly. The camera head assembly is ready for use immediately.
5. Please consult the STERRAD® 100S Sterilization System operator's manual for detailed instructions for use. Please contact Advanced Sterilization Products (ASP) for the most up to date information regarding sterilization with the STERRAD®100S Sterilization System.

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**Sterilization with the
AMSCO® V-PRO™ 1 (from STERIS®)**

The Amsco® V-PRO™ 1 low temperature sterilization system is only intended for the sterilization of goods with hydrogen peroxide steam under under-pressure conditions. The sterilization process is fully automatic.

The V-Pro™ 1 unit has touchscreen control with a backward counter and completes a validated single cycle. The user can see at a glance when the cycle will be completed.

The V-Pro™ 1 unit completes a 55 minute standard cycle composed of conditioning, sterilization and venting phases. At the end of the cycle, the prepared articles are dry, sterile, cooled down and are available immediately for use or storage as required.

Caution: The Amsco® V-PRO™ 1 low temperature sterilization system is currently only approved for the H3-Z, H3-ZI and H3-P camera heads.

Warning: If the units are not carefully cleaned and positioned so that all surfaces come into contact with the sterilization medium, or if the container is overloaded, this can lead to an ineffective sterilization process or cause damage to the units.

Warning: Only use STERIS® containers to ensure correct sterilization of the units.

Warning: Once the cycle is complete, you must always check that the sterilization container is empty. If the sterilization container is not empty, the load cannot be considered sterile.

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Warning: If a sterilization cycle is cancelled, the units are neither properly sterilized nor rinsed. The procedure must be repeated under all circumstances (check the cause of the error!)

1. Ensure that all instruments/units are fully cleaned according to the recommended methods given in the preparation section of this instruction manual.
2. Perform a visual check of the instruments/units before preparation in the STERIS® unit and check them for correct functioning.

Observe the detailed descriptions pertaining to the Amsco® V-PRO™ 1 which can be found at www.steris.com.

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**Sterilisation with the
STERRAD® 100NX™ system**

The STERRAD® 100NX™ system uses a combination of hydrogen peroxide steam and low-temperature gas plasma to rapidly sterilize a wide variety of medical instruments and materials.

There are two sterilization cycles available on the STERRAD® 100NX™ system The STANDARD cycle time is approximately 47 minutes. The FLEX cycle time is approximately 42 minutes.

Caution: The STERRAD® 100NX™ system is currently only approved for the H3-Z, H3-ZI and H3-P camera heads.

Caution: STERRAD sterilization may cause cosmetic changes to the camera head unit that do not usually impact the camera's functionality.

Caution: The camera head unit must be thoroughly dried before loading into the STERRAD® sterilization chamber. Loads containing moisture may cause a cycle cancellation.

Caution: Use only polypropylene sterilization wraps and polyethylene pouches. Do not use paper pouches or sterilization wraps containing wood pulp or cotton.

Caution: Any deviations from the recommended STERRAD® 100NX system sterilization parameters must be validated by the user.

1. Clean and prepare the camera head unit as described in the preparation section of this instruction manual. It must be completely dry.
2. Place the camera head unit in the STERRAD® sterilizer so that the hydrogen peroxide can flow around it. Do not allow the camera head unit to touch the wall of the sterilizer.

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3. Press the START button. The door closes automatically and the first of the preset, automatic cycles is started. The two cycles last approx. 89 minutes in total.
4. Once the cycles are completed, press the OPEN DOOR button and remove the camera head unit. It is ready for use immediately.
5. Detailed instructions can be found in the instruction manual for the STERRAD® 100NX sterilization system. The latest information on sterilization with the STERRAD® 100NX system is available from Advanced Sterilization Products (ASP) (www.aspii.com).

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**Sterilization with
STERIS SYSTEM1® processor**

The STERIS® process is a sterile processing method for immersible surgical and diagnostic devices. The STERIS SYSTEM1® utilizes a liquid chemical process (primarily buffered peracetic acid) for the rapid, low temperature destruction of microorganisms on the surfaces of surgical instruments and devices. Please consult the STERIS® operator's manual for safe handling instructions for the STERIS® sterilant.

Warning: Failure to thoroughly clean and properly position devices so that all surfaces will be exposed to the sterilant or overloading of the processing container may result in an ineffective sterile process and/or damage to the devices.

Warning: Use only STERIS® processing containers to ensure proper sterilization of the devices. Do **not** use the STERIS® system for devices that cannot be immersed in liquid.

Warning: Always verify that the sterilant container is empty after the cycle is complete. If the sterilant container is not empty, then the load cannot be considered sterile.

Warning: Devices are not sterilized and/or adequately rinsed when a sterile processing cycle is cancelled.

1. Make sure that all instruments/devices have been completely cleaned in accordance with the methods recommended in the preparation section of this instruction manual.
2. Visually inspect and test the devices to ensure that they are working properly before processing in the STERIS® processor.

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Maintenance

Cleaning, disinfection and sterilization

3. Place water resistant caps on cameras and other devices as required according to manufacturers' instructions. Failure to properly place the caps may result in damage to the camera or device.
4. Position the device in the STERIS® tray to ensure that all surfaces will be exposed to the sterilant. All devices should be disassembled to the smallest components possible.
5. Place the lid on the processing container.
6. Place the processing container in the processing tray. Align the fluid port on the bottom of the processing container over the fluid port of the processing tray.
7. Add the STERIS® sterilant to the processing tray and insert the aspirator probe into the sterilant container. Be sure that the aspirator probe assembly is in the proper position.
8. Close the lid. **Do not force the lid to close.** If the lid does not close easily, open the lid and check the positioning of the processing tray/ container and the aspirator assembly.
9. Press the START button. The sterilization cycle will be complete in approximately 30 minutes.
10. On completion of the sterilization cycle, five short beeps will be sounded and the light for completed cycle will be illuminated.
11. Remove the empty sterilant container and discard. Please consult the STERIS® operator's manual for proper disposal instructions.
12. The equipment is ready to use immediately on completion of the STERIS® cycle.
13. Please consult the STERIS® operator's manual for more detailed instructions for use of the STERIS® processor. Please contact STERIS Corporation for the most up to date information regarding processing of cameras in the STERIS SYSTEM1® Processor.

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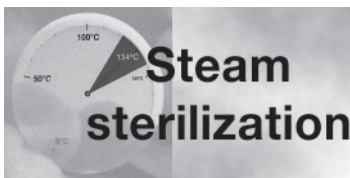
Sterilization instructions – autoclavable camera heads

Routine steam sterilization with the fractionated vacuum method is recommended for initial and subsequent sterilization of autoclavable camera head units. These camera head assemblies may also be ethylene oxide (EtO), STERIS® or STERRAD® sterilized. To achieve the desired sterility assurance level (SAL) of 10⁶, KARL STORZ recommends the following steam (or EtO, STERIS® or STERRAD®) sterilization methods:

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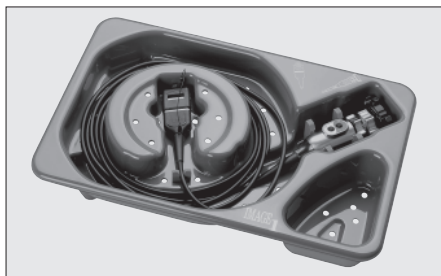
Steam sterilization

- Clean the camera head unit designated as autoclavable as described in the preparation section of this instruction manual. Place camera head unit in a suitable sterilization container (e.g., 39301 ACTS).
- KARL STORZ has validated the following steam sterilization methods:
 - Gravity Displacement:
Temperature: 121 - 122°C (250° - 252°F)
Exposure Time: 30 minutes
 - Gravity Displacement (Flash):
Temperature: 134 °C (+ 3 °C) (270° - 272°F)
Exposure Time: 10 minutes
 - Prevacuum:
Temperature: 134 °C (+ 3 °C) (270° - 272°F)
Exposure time: min. 4 minutes
- Trays should be positioned in the sterilizer so that there is adequate circulation and penetration of steam, air removal and condensate drainage. A loosely loaded sterilizer ensures the best penetration of sterilant.
- The recommended drying time for camera heads is at least 4 minutes to ensure that the card-edge connector is completely dry before it is plugged in.*
- On completion of the steam sterilization cycle, all camera heads should remain untouched until adequately cooled.

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Maintenance

Cleaning, disinfection and sterilization

Caution: Do not soak the camera head in cold liquid after removing it from the steam sterilizer. This may lead to a 'temperature shock' which can cause damage to the camera lens, and to rendering the warranty void.

6. Camera heads may be wrapped or unwrapped.

* For steam sterilizers which do not have an integrated drying cycle, the card-edge connector may be dried with a lint-free cloth, once cooled down. Prior to insertion all moisture must be removed from the card edge.

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Use of sterile covers

1. Clean and prepare the camera head unit as described in the preparation section of this instruction manual.
2. Follow the instructions provided by the manufacturer when using sterile covers.

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- Gruendemann, B.J. and Meeker, M.H. Alexander's Care of the Patient in Surgery, 7th edition. The C.V. Mosby Company, St. Louis, Mo. 1983.
- Association for the Advancement of Medical Instrumentation. Designing, Testing and Labeling Reusable Medical Devices for Reprocessing in Health Care Facilities: A Guide for Device Manufacturers. AAMI TIR No. 12-1994.
- The Difficulty of Reprocessing Reusable Rigid Laparoscopic Forceps and other Endoscopic Accessories: Are Disposables the Answer? Health Devices, Vol. 23, Nos. 1-2, pp. 57-58, 1994.
- Descoteaux, J-G, Poulin, E.C., Julein, M. and Guidoin, R. Residual Organic Debris on Processed Surgical Instruments. AORN Journal, Vol. 62, No. 1, pp. 23-29, 1995.

References for cleaning and sterilization

- Gruendemann, B.J. and Meeker, M.H. Alexander's Care of the Patient in Surgery, 7th edition. The C.V. Mosby Company, St. Louis, Mo. 1983.
- Association for the Advancement of Medical Instrumentation. Designing, Testing and Labeling Reusable Medical Devices for Reprocessing in Health Care Facilities: A Guide for Device Manufacturers. AAMI TIR No. 12-1994.
- The Difficulty of Reprocessing Reusable Rigid Laparoscopic Forceps and other Endoscopic Accessories: Are Disposables the Answer? Health Devices, Vol. 23, Nos. 1-2, pp. 57-58, 1994.
- Descoteaux, J-G, Poulin, E.C., Julein, M. and Guidoin, R. Residual Organic Debris on Processed Surgical Instruments. AORN Journal Vol. 62, No. 1, pp. 23 - 29, 1995.

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- Gruendemann, B.J. y Meeker, M.H. Alexander's Care of the Patient in Surgery, 7ª ed. La C.V. Mosby Company, St. Louis, Mo. 1983.
- Association for the Advancement of Medical Instrumentation. Designing, Testing and Labeling Reusable Medical Devices for Reprocessing in Health Care Facilities: A Guide for Device Manufacturers. AAMI TIR Nº 12-1994.
- The Difficulty of Reprocessing Reusable Rigid Laparoscopic Forceps and other Endoscopic Accessories: Are Disposables the Answer? Health Devices, Vol. 23, núm.s 1-2, págs. 57-58, 1994.
- Descoteaux, J-G, Poulin, E.C., Julein, M. y Guidoin, R. Residual Organic Debris on Processed Surgical Instruments. AORN Journal Vol. 62, Nº 1, págs. 23-29, 1995.

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Maintenance

Regular maintenance can contribute to identifying potential problems before they become serious, enhancing the device's reliability and extending its useful operating life. Maintenance services can be obtained from your local representative or from the manufacturer.

Maintenance service (functional and safety test) is recommended once a year, regardless of the accident or risk prevention regulations and/or testing intervals for medical instruments prescribed in different countries. For detailed instructions please refer to the version of the Service Manual currently valid.

Note: To ensure safe operation of the unit, you should perform the following procedures every 12 months:

1. Check leakage current from chassis <math>< 100 \mu\text{A}</math>.
2. Check ground impedance <math>< 0.1 \text{ ohms}</math>.
3. Check power consumption rated power.

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Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ original KARL STORZ parts only.

Caution: Do not attempt to open the camera head or lens. Opening these sealed assemblies affects the camera's soakability and voids all warranties.

Do not remove the covers on the camera processor. Warranty is voided unless removed by authorized personnel.

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Disposal

This unit has been marked in accordance with the European Directive 2002/96/EC on waste electrical and electronic equipment (WEEE).

At the end of its useful operating life, dispose of the unit as electronic scrap.

Please ask either KARL STORZ GmbH & Co. KG, a KARL STORZ subsidiary or your specialist dealer for information on your local collection point.

Within the scope of application of this Directive, KARL STORZ GmbH & Co. KG is responsible for the proper disposal of this unit.

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Repair program

Usually to bridge the repair period, you will receive a unit on loan which you then return to KARL STORZ as soon as you receive the repaired unit.

In Germany you can refer repairs direct to

KARL STORZ GmbH & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Str. 34
78532 Tuttlingen

In other countries please contact your local KARL STORZ branch or authorized dealer.

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Important information

In order to prevent infection, it is strictly forbidden to ship contaminated medical devices. All medical devices must be decontaminated on site, to avoid contact and aerogenous infections (among staff). We reserve the right to return contaminated instruments/equipment to the sender.

Repairs, changes or expansions which are not carried out by KARL STORZ or by experts authorized by KARL STORZ will invalidate all guarantee rights. KARL STORZ gives no guarantee on the correct functioning of equipment or instruments which have been repaired by unauthorized third parties.

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Limitation of liability

KARL STORZ shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the following conditions:

- all assembly operations, system expansions, readjustments, modifications, or repairs have been performed by a person or persons duly authorized by KARL STORZ;
- all electrical installations at the location of use meet applicable national and local electrical codes, and
- the unit has been used in accordance with its operating instructions at all times.

Manufacturer's warranty

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In so doing we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Please fill out the attached warranty card on the last page and return it as soon as possible to:

KARL STORZ GmbH & Co. KG
Postfach 230
78503 Tuttlingen, Germany

Note: The unit must always be sent to the following address (also during the warranty period; where applicable, with warranty card):

KARL STORZ GmbH & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Str. 34
78532 Tuttlingen, Germany

Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall relieve us of any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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- Unit will not switch on.

Possible causes:

- No power from power line.
- Defective power fuse.
- Power cord is not properly connected to power cord receptacle.

Remedy:

- Check that there is electricity to the wall outlet.
- Exchange fuses.
- Push power cord firmly into power cord receptacle.

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- No picture.

Possible causes

- No connection between video-out of CCU and monitor.
- Defect in camera electronics. Defective monitor.

Remedy:

- Check video cable connection, if necessary, change cables.
- Check monitor.
- Send camera head/CCU in for repair.

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- Kamerakabel (richtig) einstecken.

Symptom:

- Test image (remains displayed on the monitor).

Possible causes:

- Camera head cable connector is not correctly (or not at all) inserted into connection socket.

Remedy:

- (Correctly) insert camera head cable

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- Inaccurate color reproduction.

Possible causes:

- White balance adjustment faulty
- Color adjuster on monitor is not adjusted properly.
- Defective monitor or camera.

Remedy:

- Repeat white balance adjustment.
- Readjust color adjuster/inspect monitor using instruction manual.
- Send unit in for repair.

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Camera head cable is broken.

Remedy:

- Send unit in for repair to have camera head cable replaced.
- Change H3-M camera cable.

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Symptom:

- Image dropout, cloudy picture, streaks, smears, etc.

Possible causes:

- Endoscope telescope, camera connector or camera lens soiled.
- Cable not secured firmly.

Remedy:

- Clean with cotton swab and alcohol solution.
- Secure cable firmly.

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Troubleshooting

Warning: Always unplug the unit before doing any maintenance work on it.

Symptom:

- Interference on screen when HF instruments are used simultaneously.

Possible causes:

- Camera and HF instruments are connected to the same circuit.
- HF connecting cord is too close to camera cable or video cable.
- Too little light due to
 - soiled lens
 - defective light cable
 - defective light conductor of the telescope (Signal amplifier of the camera also amplifies HF interference signal when there is too little light).

Remedy:

- Connect units to different circuits.
- Lay HF cable at a distance from camera cable.
- Clean the telescope.
- Exchange light cable or telescope.

Note: The Image 1™ camera heads warm up automatically when in operation (to max. body temperature). This process does not represent a malfunction!

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Technical data**Camera control unit**

Image 1 HUB™ HD
PAL and NTSC, CE according to MDD
Ordering no. 22201020-1xx
Instrument coupling:
per DIN 58105 (standard)
CCU inputs: Keyboard input (5 pin DIN)
Serial port I/O: RJ-11 (2 x)
Signal system:
EIA standards, NTSC color system
CCIR-B standards, PAL color system
AGC: +18 dB
Video output (PAL):
Composite signal to BNC socket,
1 Vpp to 75 ohm, sync neg.
S-Video (Y/C) signal to 4-pin-Mini-DIN-socket
(Y): 1.0 Vpp to 75 ohm; (C): 0.33 Vpp to 75 ohm
RGB signal to 15pin-HD-D-Sub-sockets:
0.714 Vpp to 75 ohm; HD-DVI (2)
Video output (NTSC):
NTSC 3.58 signal to BNC socket,
1.0 Vpp to 75 ohm, sync neg.
S-Video (Y/C) signal to S-Video socket
(Y): 1.0 Vpp to 75 ohm; (C): 0.286 Vpp to 75 ohm
RGB signal to 15pin-HD-D-Sub-sockets:
0.714 Vpp to 75 ohm, HD-DVI (2)
SCB: 6 pin Mini-DIN-socket for connection to a
SCB interface
Keyboard input:
5-pin-socket, AT compatible
Power consumption: 84...96 VA
Power fuses: 2 x T1.6 AL250V
Power voltage: 100...240 VAC
Power frequency: 50/60 Hz
Dimensions:
CCU: 305 mm x 89 mm x 335 mm (wxhxd)
Weight: approx. 3.35 kg (CCU)
Operating temperature: +10...+40 °C, 50...104 °F
Storage conditions:
Ambient temperature -40...+70 °C (-40...158 °F)
Humidity 10...100% (non-condensing)
Atmospheric pressure 50 kPa...106 kPa

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Technical data

Camera control unit

IMAGE 1 Pure HD
PAL and NTSC, CE according to MDD
Ordering no. 22201020-1xx
Instrument coupling:
per DIN 58105 (standard)
CCU inputs: Keyboard input (5 pin DIN)
Serial port I/O: RJ-11 (2 x)
Signal system:
EIA standards, NTSC color system
CCIR-B standards, PAL color system
AGC: +18 dB
SCB: 6 pin Mini-DIN-socket for connection to a SCB interface
Keyboard input:
5-pin-socket, AT compatible
Power consumption: 84...96 VA
Power fuses: 2 x T1.6 AL250V
Power voltage: 100...240 VAC
Power frequency: 50/60 Hz
Dimensions:
CCU: 305 mm x 89 mm x 335 mm (wxhxd)
Weight: approx. 3.35 kg (CCU)
Operating temperature: +10...+35 °C
Storage conditions:
Ambient temperature -40...+70 °C (-40...158 °F)
Humidity 10...100% (non-condensing)
Atmospheric pressure 50 kPa...106 kPa

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Camera heads in use with

Image 1 HUB™ HD / IMAGE 1 Pure HD

Three-chip CCD HD Camera Head H3-ZI,
Parfocal zoom

Cat. no. 22220053-3 H3-ZI (Z=Zoom)

Image sensor: 3 x 1/3"

Aspect ratio: 16:9

Picture elements: 1920x1080 (2,073,600) pixels

Scan method: progressive

Image refresh rate: 50 Hz

* Internal lens: Parfocal 2:1 optical zoom

* Focal length: 14 ... 30 mm

Three Chip CCD HD Camera Head H3-M

Cat. no. 22220054-3

All other data: as for camera head H3;

The H3-M microscope camera head does not
have a lens system, but a C-mount interface.

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Camera heads in use with

Image 1 HUB™ HD / IMAGE 1 Pure HD

Three chip CCD HD camera head H3-Z,
Parfocal zoom

Cat. no. 22220055-3 H3-Z* (Z=Zoom)

Image sensor: 3 x 1/3"

Aspect ratio: 16:9

Picture elements: 1920 x 1080 (2,073,600) pixels

Scan method: progressive

Image refresh rate: 50 Hz / 60 Hz
(automatic switchover)

* Internal lens: Parfocal 2:1 optical zoom

* Focal length of the lens: 15-32 mm

Three Chip CCD HD Camera Head H3-M

Cat. no. 22220154-3

All other data: as for camera head H3;

The H3-M microscope camera head does not
have a lens system, but a C-mount interface.

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Camera heads in use with

Image 1 HUB™ HD / IMAGE 1 Pure HD

Three-chip CCD HD Pendulum camera head H3-P,
Parfocal zoom

Cat. no. 22220056-3 H3-P)

Image sensor: 3 x 1/3"

Aspect ratio: 16:9

Picture elements: 1920 x 1080 (2,073,600) pixels

Scan method: progressive

Image refresh rate: 50 H / 60 Hz
(automatic switchover)

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Camera heads in use with Image 1 HUB™ HD

Single Chip CCD Camera Head,
Parfocal zoom, color system PAL
Cat. no. 22210030-3 S1*/22210040-3 A1*
22210031-3 P1 (f = 14 mm),
22210032-3 P1 (f = 16.8 mm)
22260031-3 D1 (f = 16 mm)

Image sensor: 1/4"
Picture elements: 440 k
Resolution (horizontal): > 500 lines
Signal-to-noise-ratio: 54 dB
Auto exposure:
Microprocessor controlled
White balance: 1900 ... 6900 K
Min. sensitivity: 3 lux
* Internal lens: Parfocal 2:1 optical zoom
* Focal length: 14 ... 28 mm

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Image sensor: 1/4"
Picture elements: 380 k
Resolution (horizontal): > 500 lines
Signal-to-noise-ratio: 54 dB
Auto exposure:
Microprocessor controlled
White balance: 1900 ... 6900 K
Min. sensitivity: 3 lux
* Internal lens: Parfocal 2:1 optical zoom
* Focal length: 14 ... 28 mm

Cabezal de la cámara CCD de un chip,
Zoom Parfocal, sistema de color NTSC
Nº art. 22210130-3 S1*/22210140-3 A1*
22210131-3 P1 (f = 14 mm),
22210132-3 P1 (f = 16,8 mm)
22200131-3 D1 (f = 16 mm)

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Camera heads in use with Image 1 HUB™ HD

Three Chip CCD Camera Head,
Parfocal zoom, color system PAL
Cat. no. 222200 30-3 S3*/222200 40-3 A3*
222200 31-3 P3 (f = 14 mm),
222200 32-3 P3 (f = 16.8 mm)
Image sensor: 3 x 1/4"
Picture elements: 440 k
Resolution (horizontal): > 700 lines
Signal-to-noise-ratio: 60 dB
Auto exposure:
Microprocessor controlled
White balance: 1900 ... 8700 K
Min. sensitivity: 1.3 lux
* Internal lens: Parfocal 2:1 optical zoom
* Focal length: 14 ... 28 mm

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Three Chip CCD Camera Head,
Parfocal zoom, color system NTSC
Cat. no. 222201 30-3 S3*/222201 40-3 A3*
222201 31-3 P3 (f = 14 mm),
222201 32-3 P3 (f = 16.8 mm)
Image sensor: 3 x 1/4"
Picture elements: 380 k
Resolution (horizontal): > 700 lines
Signal-to-noise-ratio: 60 dB
Auto exposure:
Microprocessor controlled
White balance: 1900 ... 8700 K
Min. sensitivity: 1.3 lux
* Internal lens: Parfocal 2:1 optical zoom
* Focal length: 14 ... 28 mm

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Standard compliance (for 22201020-1xx / 22202020-yyy with camera heads 222x0x.xx-3)

According to IEC 60601-1, IEC 60601-2-18, UL 2601.1, CSA C22.2 No. 601.1-M90:

- Type of protection against electric shocks: Protection Class I
- Degree of protection against electric shocks: Applied part of type CF, defibrillation proof 

IEC 60601-1-2 :2001:

Please observe the information on electromagnetic compatibility in the Appendix (p. 119).

Directive compliance (for 22201020-1xx with camera heads 222x0x3x-3)

According to Medical Device Directive (MDD):

This medical product belongs to Class IIb

This medical product bears the CE mark in accordance with MDD 93/42/EEC. A code number after the CE mark indicates the responsible notified body.

Technical documentation

On request, the manufacturer will provide those circuit diagrams, itemized parts listings, descriptions, sets of adjustment instructions, and other items of available documentation to suitably qualified user personnel duly authorized by the manufacturer for their use in repairing those components of the unit that have been designated as repairable by their respective manufacturers.

Supply of such technical documentation relating to the unit shall not be construed as constituting manufacturer's authorization of user's personnel, regardless of their levels of technical training, to open or repair the unit. Explicitly exempted herefrom are those maintenance and repair operations described in this manual.

We reserve the right to make engineering modifications in the interest of promoting technological progress and generating performance improvements without obligation on our part to submit prior notice thereof.

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Software User Rights

1. With respect to KARL STORZ products containing software components, the Customer has a non-exclusive License, which is limited by the meaning of the following terms and conditions, to use the programmed logic, computer programs and/or software supplied by KARL STORZ and incorporated in KARL STORZ products (“Software”).
2. This License applies for such products sold to the Customer only to the form delivered to Customer and for the sole purpose of operating such products in accordance with written instructions by KARL STORZ. Use of the Software in connection with other products bought by the Customer from KARL STORZ or from third parties is prohibited.
3. A transfer of this License by the Customer is permissible only if the Customer provides the buyer’s name to KARL STORZ, and said buyer agrees to comply with the terms and conditions regarding the usage of the software as set forth in this License. In the case of a transfer, the Customer’s License shall terminate.
4. All copyrights to the Software, as well as all modifications, enhancements, and upgrades thereto, will remain the property of KARL STORZ. In particular, the Customer may not duplicate, copy, reverse-engineer, decompile, or disassemble the Software or in any way modify the same. The Customer has no right to create derivatives of the software, and the Customer may not copy, decipher or re-create the source code of the Software. § 69g paragraph 2 of the Copyright remains unaffected by this. [This refers to German Copyright Law]
5. Any modification or enhancement of the Software by the Customer in violation of this Agreement is considered a violation of this License; KARL STORZ immediately shall acquire all property rights and copyrights to such modifications or enhancements. The Customer hereby in advance transfers an exclusive, free, and unlimited license to use such modifications and enhancements to KARL STORZ.
6. KARL STORZ does not recommend use of the Software in connection with other software not supplied by KARL STORZ for the respective products. KARL STORZ is not liable for software malfunctions resulting from the use of software not supplied or recommended by KARL STORZ in connection with products supplied by KARL STORZ.
7. In the event of a failure of the Customer, or its agents, employees or representatives, to comply with any terms and conditions of the License herein granted, the License will, without termination or any further action by KARL STORZ or any other party, immediately terminate.

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
General circuit diagram
IMAGE 1 HUB™ HD

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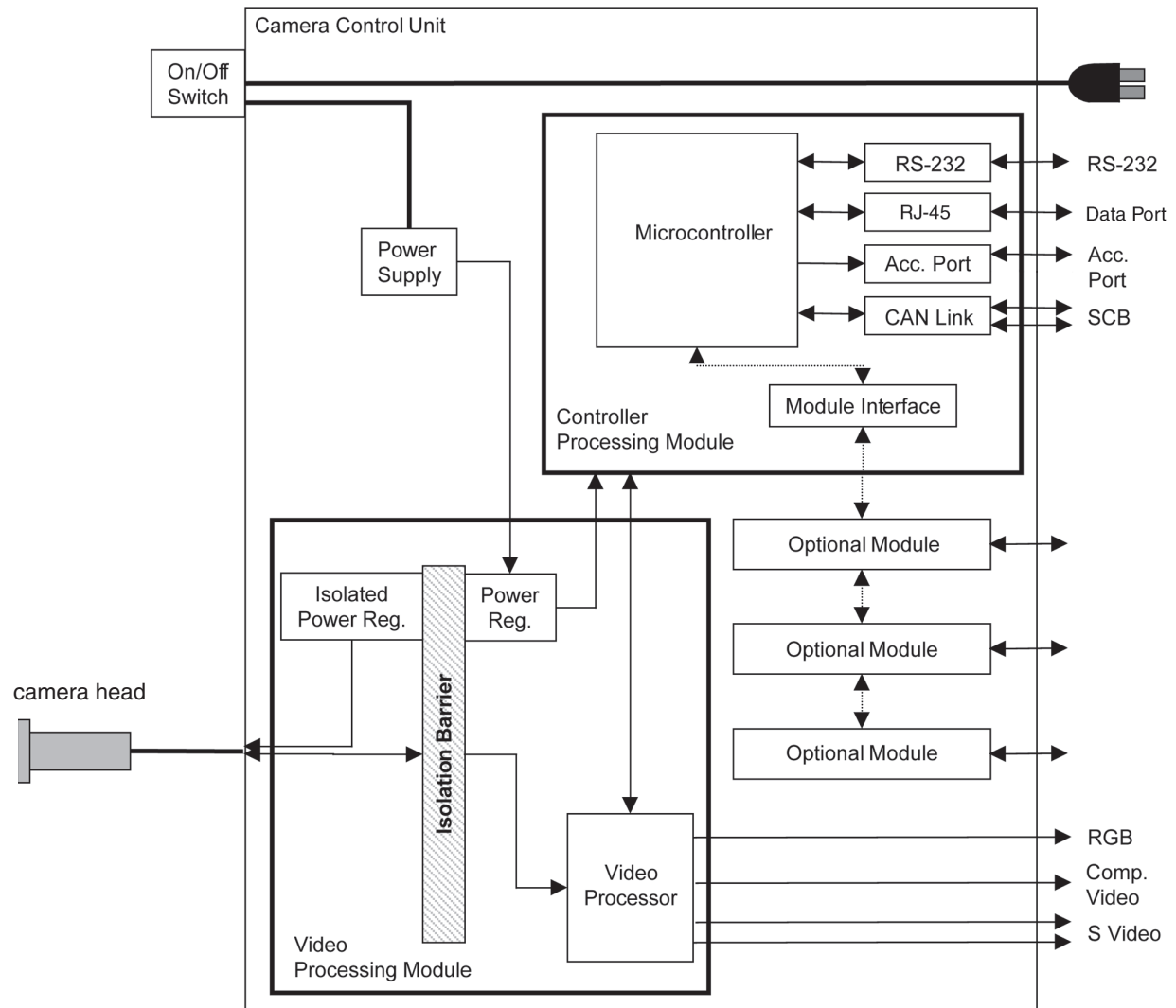
General circuit diagram
IMAGE 1 Pure HD

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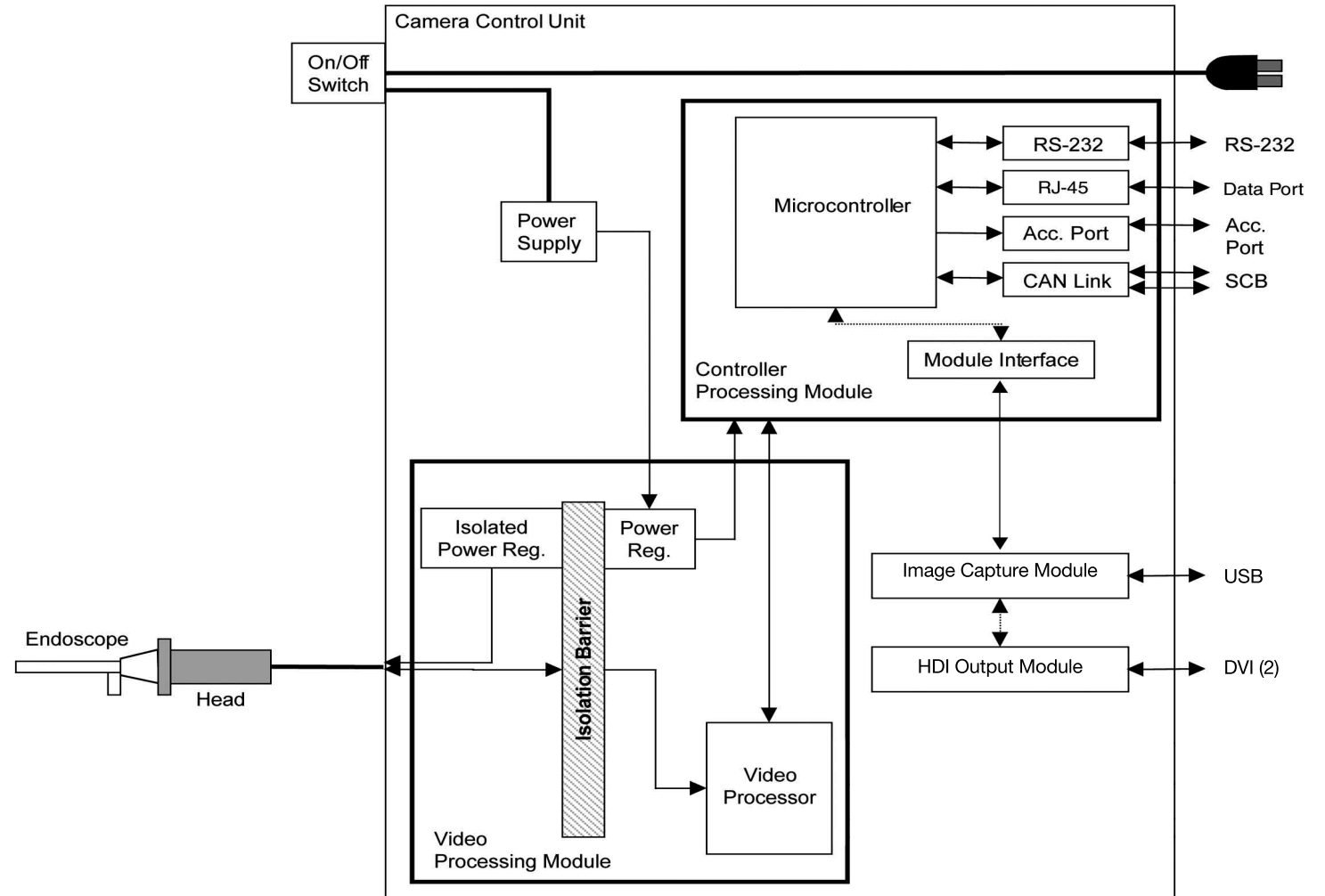Frequent questions concerning the menus:

1. How do I reach the menu item, where I can set my required language?

- Simultaneously press SHIFT + ENTER on the keyboard or hold down the center camera head button for at least 7 seconds.
 - (The Options Menu appears)
- Select the menu item “Setup Wizard CCU”.
- Now you can set your required language under “Display” and your required keyboard layout under “Keyboard”.

2. How do I make the insufflator functions available for use?

- Simultaneously press SHIFT + ENTER on the keyboard or hold down the center camera head button for at least 7 seconds.
- (The Options Menu appears)
- Select the menu item “Setup Wizard Camera Head”.
- Click on “Next” twice, so that the insufflator menu appears.
- Enable the required insufflator functions Gas, Flow, Pressure and Volume Reset.

User Menu

To access the User Menu press ENTER on the keyboard or briefly press the center camera head button.

Exit
White balance
Contrast enhancement
Luminance
Capture image
Video recording
Light source
Insufflator
Program Head Buttons

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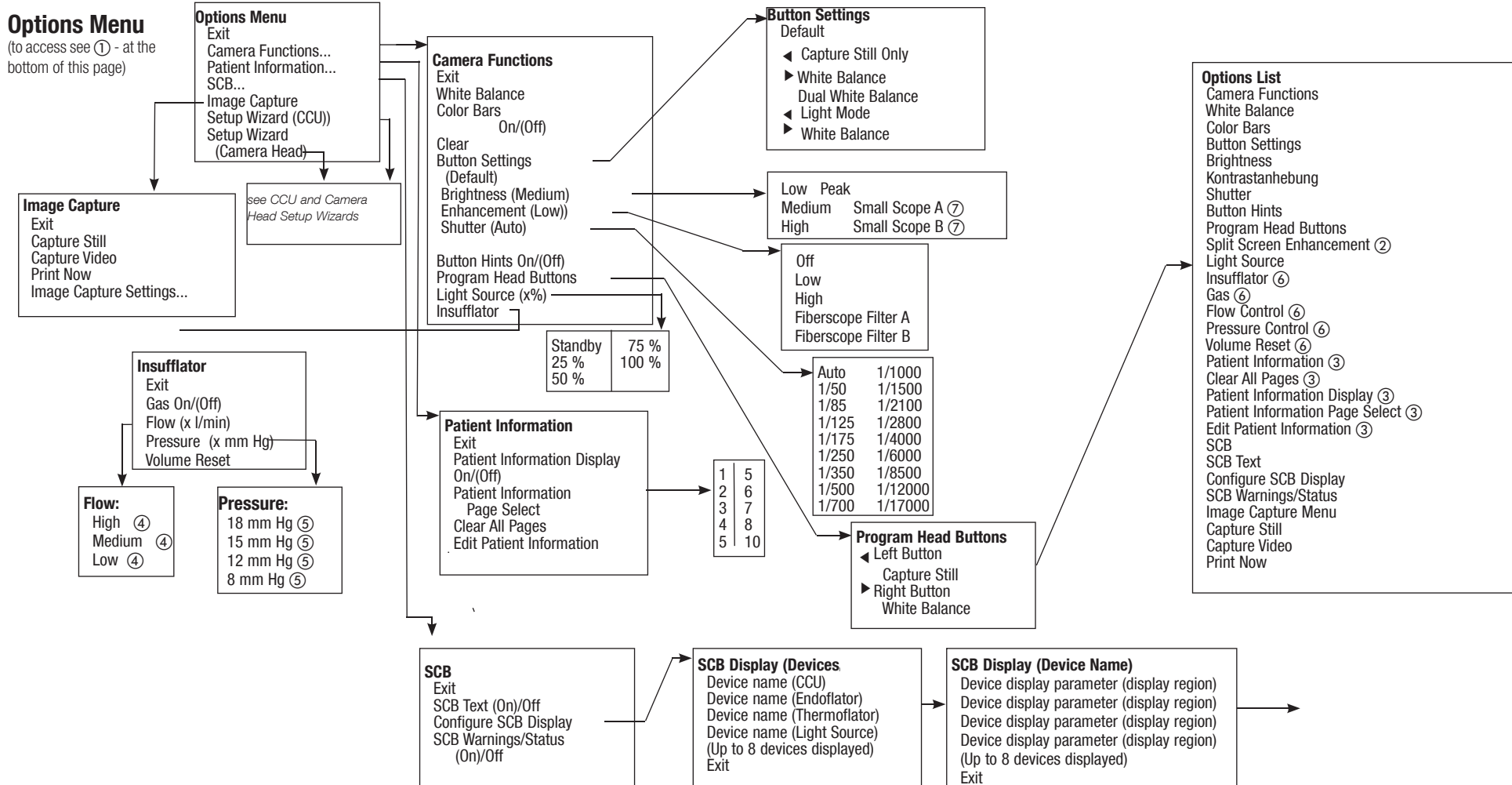
The following menu diagram applies to the H3-ZI HD camera head. The menus for the SD camera heads are similar. The key difference: The menus for the SD camera heads do not include the Insufflator menu.

① **Accessing the options menus** • either by simultaneously press SHIFT + ENTER on the keyboard • or by pressing and holding the center camera head button (at least 7 seconds)

② only available for connected SD camera head (Standard Definition) , not available for HD camera heads (High Definition)

③ only available if a keyboard is connected to the CCU (IMAGE 1 HUB HD / IMAGE 1 Pure HD)

④ • for Thermoflator®: High 30 l/min -- Medium 15 l/min -- Low 2 l/min • for Endoflator®: High 20 l/min -- Medium 10 l/min -- Low 2 l/min • for Endoflator®:

⑤ selectable values depend on camera head settings. To change the settings click on 'Setup Wizard Camera Head' in the Options Menu and then click on 'Next' twice. Now change the settings

⑥ only available when activated in the insufflator menu. To change the settings click on 'Setup Wizard Camera Head' in the Options Menu, then click on 'Next' twice. Now change the settings.

⑦ applies to the H3-Z, H3-ZI and H3-P camera heads

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

Setup Wizard (accessible by pressing **SHIFT+ENTER** on the keyboard or by pressing and holding down the center camera head button for a minimum of 7 seconds. Following this, select the Setup Wizard)

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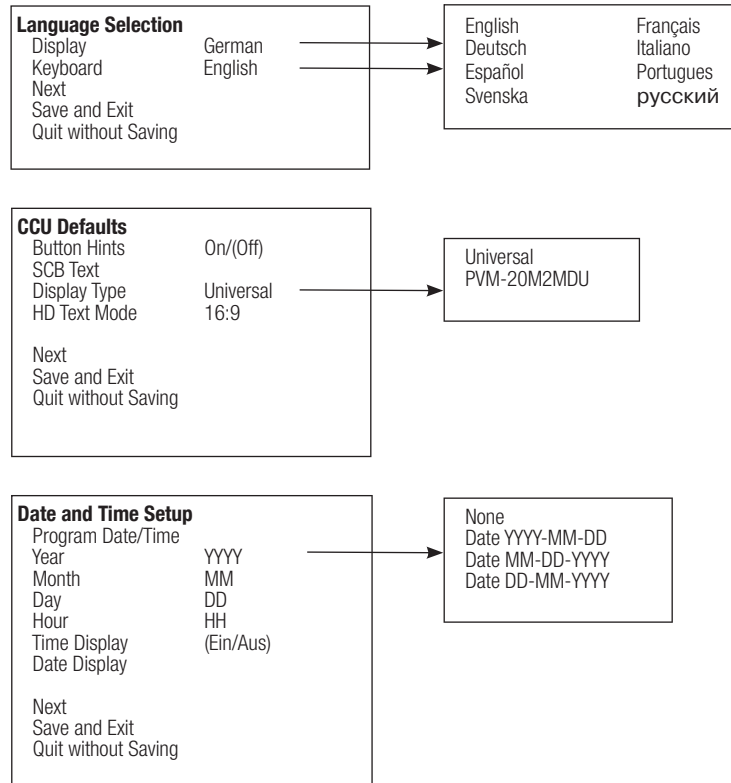Setup Wizard CCU

Setup Wizard Camera Head

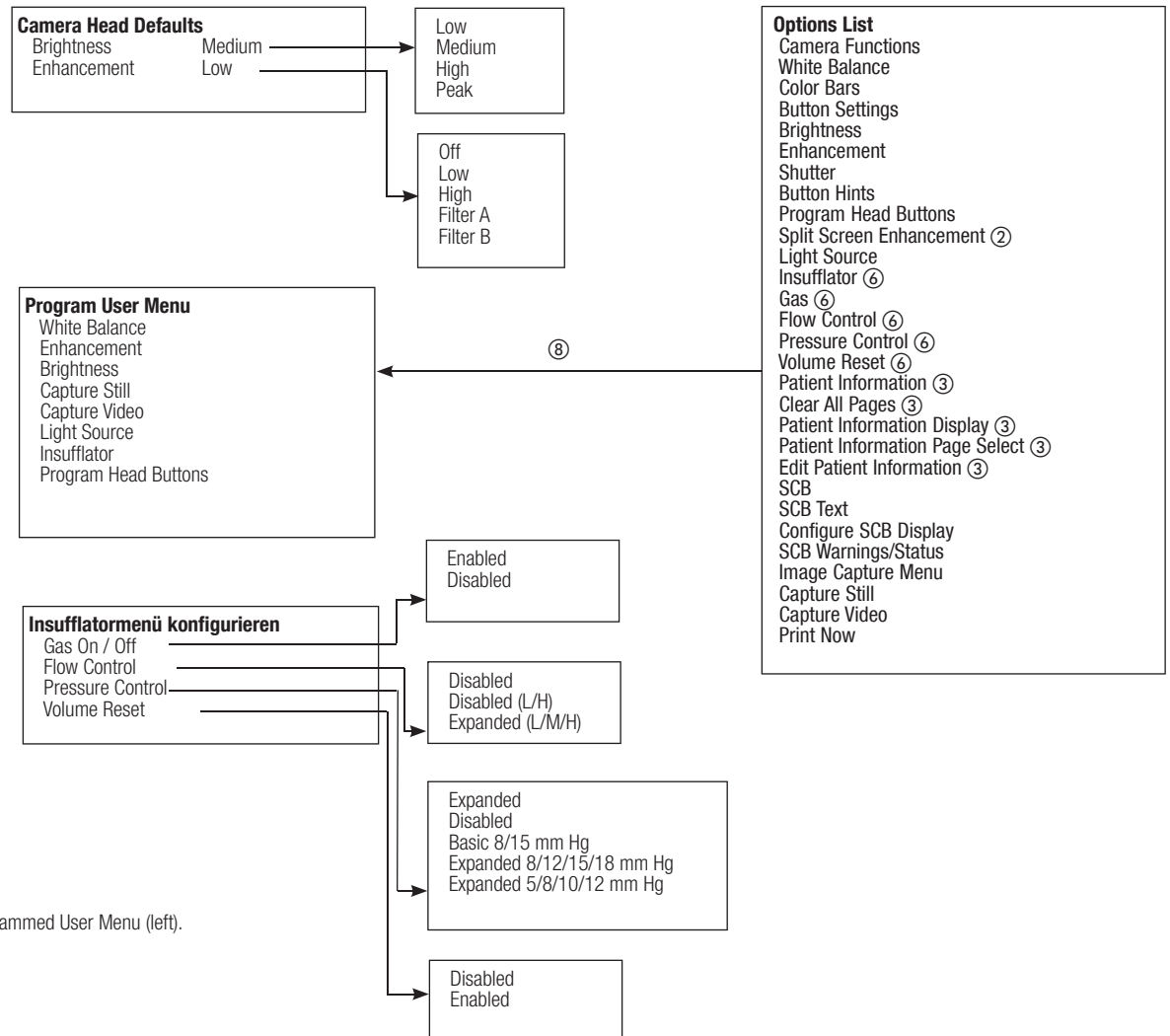
The explanation for numbers ②, ③ and ⑥ can be found on the previous page

From the options list (right), 8 menu entries can be selected, which then appear in the programmed User Menu (left).

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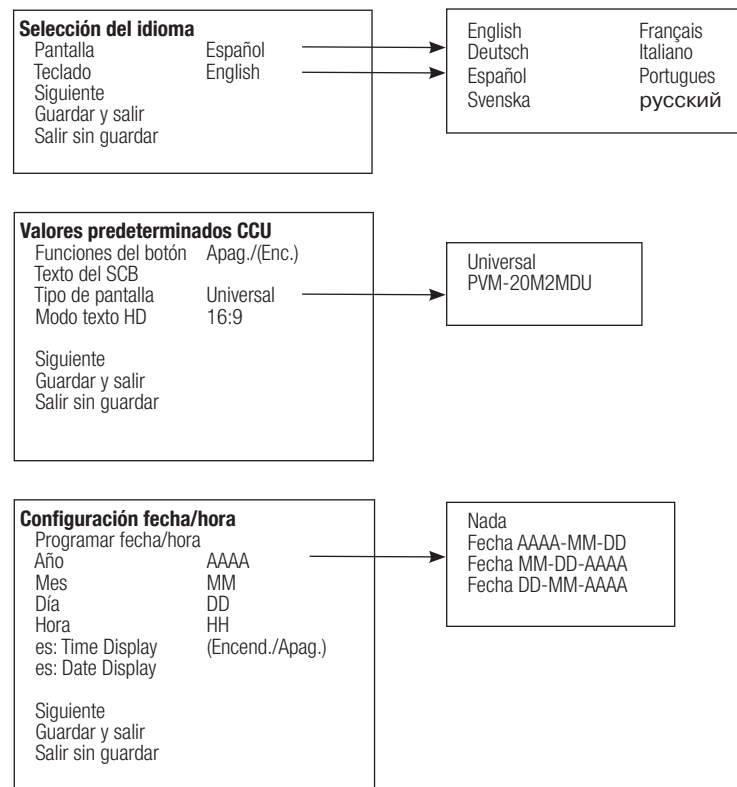
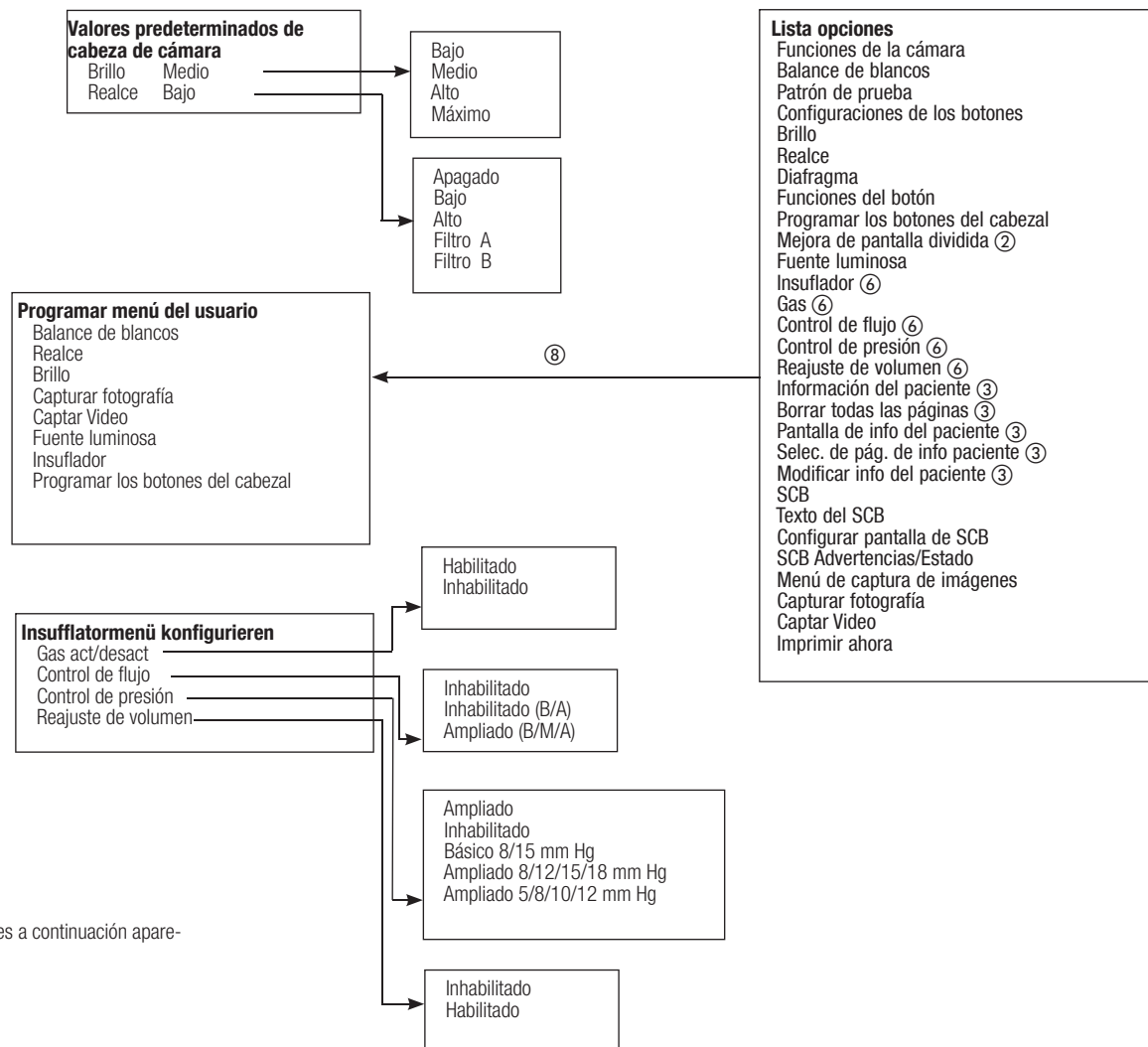
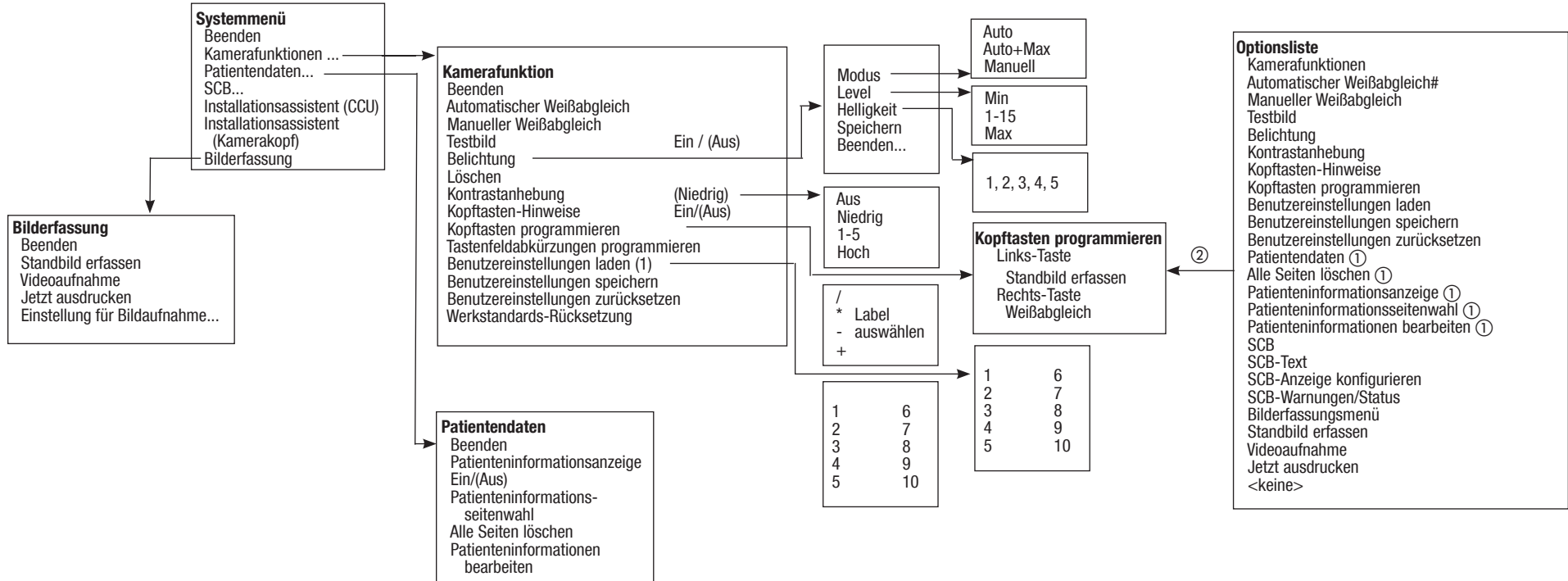
Options Menu for the H3-M microscope camera head

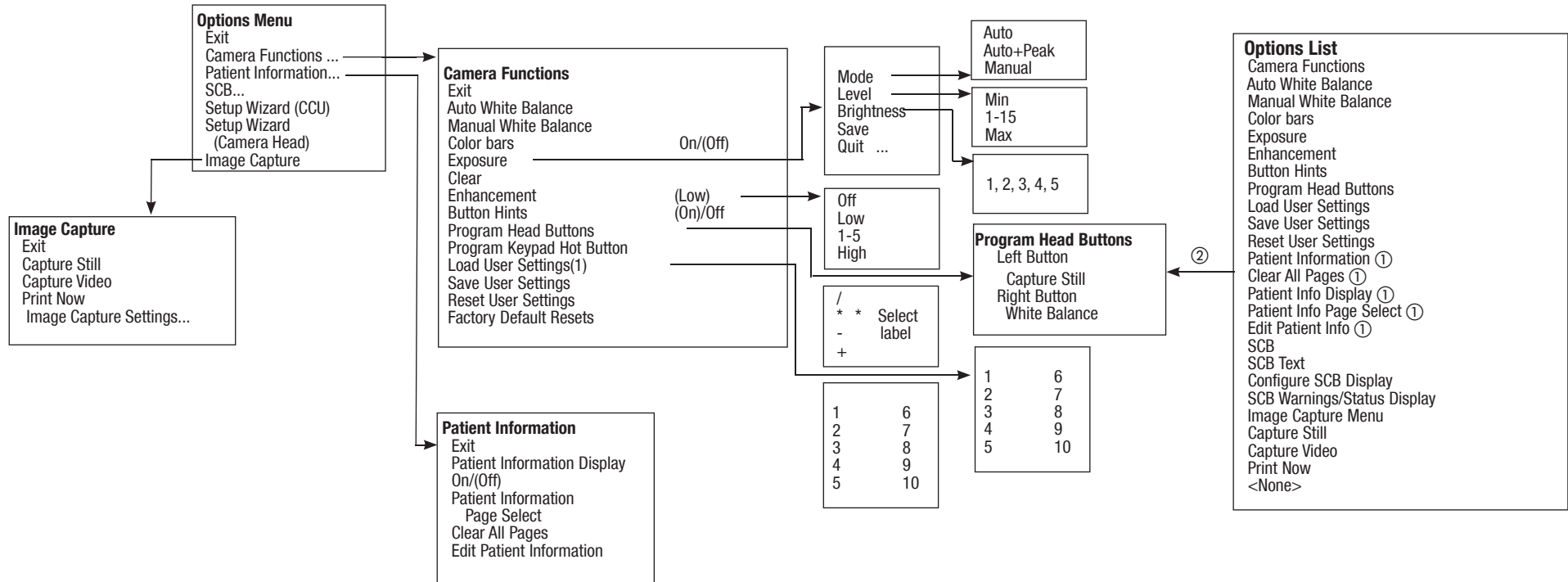Accessing the options menus • either by simultaneously pressing SHIFT + ENTER on the keyboard • or by pressing and holding the center camera head button (at least 7 seconds)


① only available if a keyboard is connected to the CCU (IMAGE 1 HUB HD / IMAGE 1 Pure HD)

② From the options list (right), 8 menu entries can be selected, which then appear in the programmed User Menu (left).

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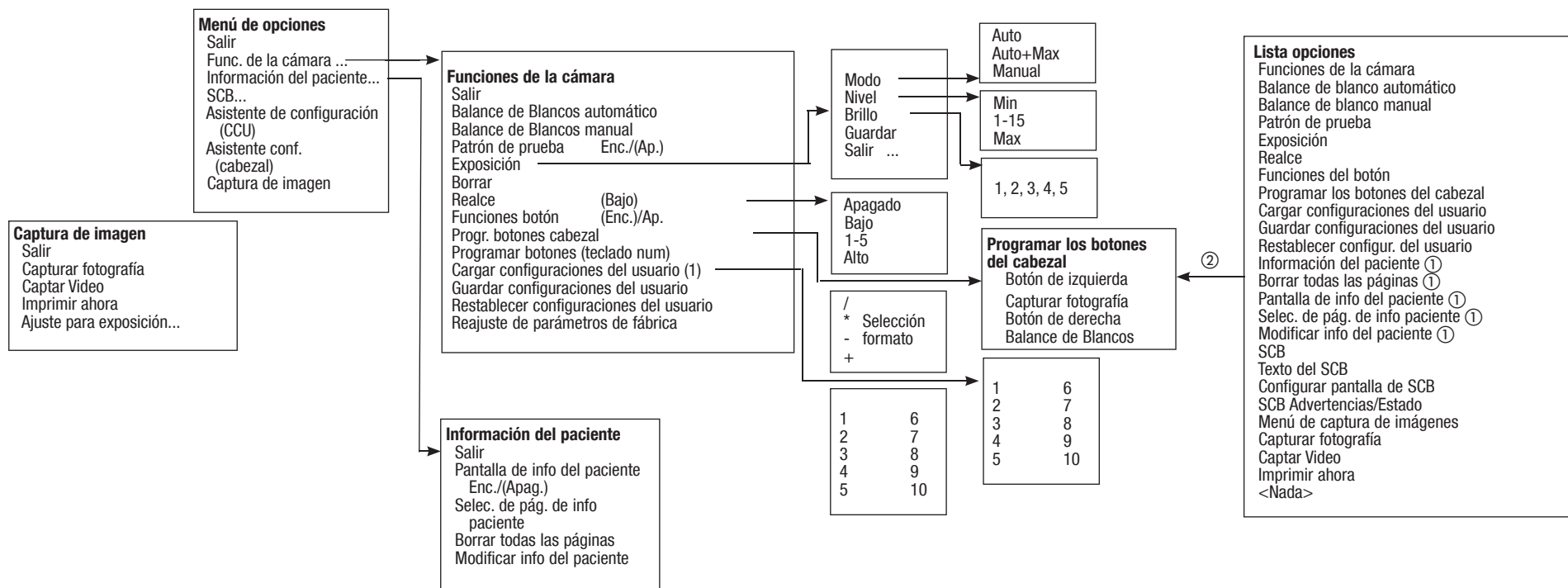* For safe and comfortable working with
KARL STORZ cameras we recommend the use of
sterile covers from
mtp medical technical promotion gmbh,
Take-Off GewerbePark 46
78579 Neuhausen ob Eck, Germany
Phone: +49 (0)7467 945 04-0
Fax: +49 (0)7467 945 04-99

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Type

- 1 = Cleaner
- 2 = Disinfectant
- 3 = Suitable for ultrasound baths
- A = Not suitable for flexible endoscopes (according to manufacturers' instructions)
- (A) = Only for the flexible endoscopes of the new generation (year of manufacture > 2005)

Note: You will also find the current list of approved products on our Web site (www.karlstorz.com).

Note: When agents other than those named are used, KARL STORZ assumes no liability for any damage which may occur.

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Extract from the list of approved chemicals for the preparation of telescopes and endoscopic instrument sets

Manual preparation

The following instruments and telescopes produced by KARL STORZ must not be fully immersed in a liquid: HAMOU® telescopes with ocular focusing, magnifiers with focusing, IMPERATOR drill handles, prismatic light deflectors. For motors and their handles the appropriate manual provides detailed information.

The following instruments produced by KARL STORZ must not be placed in an ultrasound cleaning bath: rigid telescopes, flexible telescopes, other instruments with optical components made of glass.

When preparing and using the solutions, follow the solution manufacturer's instructions, paying close attention to concentration and exposure times. Regarding the microbiological effectiveness please ask the manufacturer. The following cleaning and disinfectant solutions have been approved regarding material compatibility:

Ecolab GmbH & Co. OHG	Sekusept Plus	1, 2, 3
	Sekusept Pulver Classic	1, 2, 3
Esteer Pharma GmbH	Ultrasemit AF	1, 2, 3
	Ultrasseptin Aktiv	1, 2, 3
	Ultrasseptin Classic	1, 2, 3
Fresenius AG	Afid	1, 2
	Afid plus	2
Holifa Polska Sp. zo.o	Sporcid FF	1, 2
	Poisept Holifa	1, 2
Indeba Industria E Comercio LTDA	Letahdeido	2
	Darodor 4000 Liquido	1
Jose Collado S.A.	Darodor 9000	2
	Darodor Sinaldehyd 2000	2
	Aldasan 2000	1, 2
Lysoform Dr. Rosemann GmbH	Almyrol	1, 2
	Desoform	1, 2, 3
Medichem International	Lysoformin 3000	1, 2
	MedDis	2
	MediZyme	1
Merz Hygiene GmbH	Edisonite Super	1, 3
	Mucadont-IS	2, 3
	Mucadont-Zymaktiv	1, 3
	Mucocit-T neu	1, 2, 3
METASYS Medizintechnik GmbH (formerly LONZA)	ID 50	2
	A 20	1, 3, A
Promagent AB	Wavacide	1, 2
	Descoton Extra	1, 2
Schumacher, Dr. GmbH	Descoton	2
	Perfektan Endo	2, 3
	Perfektan Neu	1, 2
	Perfektan TB	1, 2, 3, A
Schülke & Mayr GmbH	Gigasept	1, 2
	Gigasept AF forte	2, 3
	Gigasept FF	1, 2
	Gigasept Instru AF	1, 2, 3, A
	Gigasept Med forte	1, 2
Steris	Gigazyme	1, 3
	Lysetol V	2
	EnzyCare 2	1, 3
Weigert, Dr. GmbH & Co.	neodisher LM 2	1, 3
	neodisher mediclean	1
	neodisher mediclean forte	1, 3
	neodisher medizym	1, 3
Whiteley Industries PTY.Ltd	neodisher SeptoClean	1, A
	neodisher SeptoPreClean	1, 2
Whiteley Industries PTY.Ltd	Aidal Plus	2
	Matrix	1

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*** Caution:** The use of CAUSTIC SODA can cause changes to the surface of aluminium parts (including coated parts), plastics and soldered connections, and may reduce the service life of the instrument set. For **flexible endoscopes**, only immerse the examination shaft, **not** the housing/handle part.

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Type

- 1 = Cleaner
- 2 = Disinfectant
- 3 = Suitable for ultrasound baths
- 4 = Neutralizer
- 5 = Rinse aid
- A = Not suitable for flexible endoscopes (according to manufacturers' instructions)

Note: You will also find the current list of approved products on our Web site (www.karlstorz.com).

Note: When agents other than those named are used, KARL STORZ assumes no liability for any damage which may occur.

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Machine preparation

The following instruments and telescopes produced by KARL STORZ must not be prepared in a cleaning and disinfecting machine: HAMOU® telescopes with ocular focusing, magnifiers with focusing, IMPERATOR drill handles, standard, tungsten carbide and diamond-tipped drills, prismatic light deflectors, and magnetic instrument rack. For motors and their handles the appropriate manual provides detailed information.

Selection of a method for cleaning and disinfection must be discussed with the machine manufacturer and the manufacturer of the chemical reagents. Only special methods must be used which have been verified for this purpose. Regarding the microbiological effectiveness please ask the manufacturer. Thermal disinfection is preferable. The following agents for machine cleaning and disinfection have been approved regarding material compatibility:

Schülke & Mayr GmbH	Thermosept alka clean	1
	Thermosept DK	2
	Thermosept ED	2
	Thermosept ER	1
	Thermosept RKN-zym	1
Steris	Instru-Klenz	1, 3
Weigert, Dr. GmbH & Co.	neodisher DuoClean	1, A
	neodisher FA	1
	neodisher FA forte	1, 3
	neodisher Oxivario	1
	neodisher mediclean	1
	neodisher mediclean forte	1, 3
	neodisher mediklar	5
	neodisher medizym	1
	neodisher SeptoClean	1, 2, A
	neodisher Septo DA	2
	neodisher Septo DN	2

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Appendix

Electromagnetic Compatibility (EMC) Information

WARNING: Medical electrical equipment needs special precautions regarding Electromagnetic Compatibility (EMC). Observe the EMC instructions in this appendix during installation and commissioning.

The Image 1 HUB™ HD model 22201020-1xx corresponds to EN/IEC 60601-1-2 :2001 [CISPR 11 Class A] and therefore meets the EMC requirements of the Medical Device Directive (MDD) 93/42/EEC.

These limits are designed to provide reasonable protection against the typical electromagnetic interference to be expected in a medical environment. The Image 1 HUB™ HD model 22201020-1xx is a Group 1 unit (as per CISPR 11).

Group 1 contains all the 'equipment and systems which generate or use RF energy only for their internal functioning'.

NOTE: The tables and guidelines that are included in this appendix provide information to the customer or user that is essential in determining the suitability of the equipment or system for the electromagnetic environment of use, and in managing the electromagnetic environment of use to permit the equipment or system to perform its intended use without disturbing other equipment and systems or non-medical electrical equipment. If this equipment does cause harmful interference with other devices, the user is encouraged to try to correct the interference by one or more of the following measures:

- reorient or relocate the receiving device
- increase the separation between the equipment
- Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.

If you have any further questions, please contact your local representative or our service department.

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Appendix**Electromagnetic Compatibility (EMC)
Information**

WARNING: The Image 1 HUB™ HD model 22201020-1xx should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the equipment or system should be observed to verify normal operation in the configuration in which it will be used.

WARNING: The Image 1 HUB™ HD model 22201020-1xx is intended for use by healthcare professionals only. This is a CISPR 11 Class A medical system (RF emissions) and is suitable for use in all establishments other than domestic and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes. In a domestic environment the Image 1 HUB™ HD model 22201020-1xx may cause EMC interference, in which case it may be necessary to take adequate mitigation measures, such as re-orienting, relocating, or shielding the Image 1 HUB™ HD model 22201020-1xx or filtering the connection to the public mains network.

WARNING: The use of portable and mobile RF equipment may have an impact on this and other pieces of medical equipment.

WARNING: The use of cables other than those specified in the KARL STORZ instruction manual may result in increased emissions or decreased immunity of the Image 1 HUB™ HD model 22201020-1xx. The cables listed below have been shown to comply with the requirements of EN/IEC 60601-1-2. When using cables other than those specified here, it is the responsibility of the user to ensure that they comply with EN/IEC 60601-1-2.

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Table 200				
Cables which have been shown to comply with EN/IEC 60601-1-2:				
Cable type	Shielded	Length [m]	Ferrite	Used for
S-Video [Y/C]	Yes	1.8	No	Connection of S-Video signals
BNC to BNC	Yes	1.8	No	Connection of composite or SDI video signals
BNC to RGB Video	Yes	1.8	No	Connection of RGB signals
4 Pin Mini DIN	Yes	1.8	No	Connection of SCB signals
1395 Firewire	Yes	4.5	No	Connection of DV Signals
AC Power Cord	No	3	No	Power supply

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Table 201		
Guidance and manufacturer's declaration – electromagnetic emissions		
IMAGE 1 HUB™ HD (22201020-1xx) and IMAGE 1 Pure HD (22202020-yyy) are intended for use in the electromagnetic environment specified below. The customer or user of the devices should ensure that they are used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidelines
RF emissions CISPR 11	Group 1	IMAGE 1 HUB™ HD and IMAGE 1 Pure HD uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	IMAGE 1 HUB™ HD and IMAGE 1 Pure HD are suitable for use in all establishments other than domestic and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

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<p align="center">Table 202 Guidance and manufacturer's declaration – electromagnetic immunity</p>			
<p>IMAGE 1 HUB™ HD (22201020-1xx) and IMAGE 1 Pure HD (22202020-yyy) are intended for use in the electromagnetic environment specified below. The customer or user of the devices should ensure that they are used in such an environment.</p>			
Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidelines
Electrostatic Discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	Complies ± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Complies ± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	Complies ± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% U_T^* (>95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles <5% U_T (>95% dip in U_T) for 5 sec	Complies <5% U_T^* (>95% dip in U_T) for 0.5 cycle Complies 40% U_T (60% dip in U_T) for 5 cycles Complies 70% U_T (30% dip in U_T) for 25 cycles Complies <5% U_T (>95% dip in U_T) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the equipment or system requires continued operation during power mains interruptions, it is recommended that the equipment or system be powered from an uninterruptible power supply.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	Complies 3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
<p>* Note: U_T is the a.c. mains voltage prior to application of the test level.</p>			

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Table 204

**Guidance and manufacturer's declaration – electromagnetic immunity –
for equipment and systems that are not life-supporting**

IMAGE 1 HUB™ HD (22201020-1xx) and IMAGE 1 Pure HD (22202020-yyy) are intended for use in the electromagnetic environment specified below. The customer or user of the IMAGE 1 HUB™ HD (22201020-1xx) and IMAGE 1 Pure HD (22202020-yyy) should ensure that they are used in such an environment.

Immunity test	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 V_{rms} 150 kHz to < 80 MHz</p> <p>3 V/m 80 MHz to 2.5 GHz</p>	<p>3 V_{rms}</p> <p>3 V/m</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the IMAGE 1 HUB™ HD/IMAGE 1 Pure HD, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> <p>$d = [3.5/3]\sqrt{P}$ 150 kHz to < 80 MHz</p> <p>$d = [3.5/3]\sqrt{P}$ 80 MHz to < 800 MHz</p> <p>$d = [7/3]\sqrt{P}$ 800 MHz to 2.5 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts [W] according to the transmitter manufacturer and d is the recommended separation in meters [m].</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection by structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio [cellular/cordless] telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the unit is used exceeds the applicable RF compliance level above, the unit should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the unit.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

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Table 206

**Recommended separation distances between portable and mobile RF communications equipment and the
IMAGE 1 HUB™ HD (22201020-1xx) or IMAGE 1 Pure HD (22202020-yyy)**

The IMAGE 1 HUB™ HD (22201020-1xx) and IMAGE 1 Pure HD (22202020-yyy) are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the units can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the units as recommended below, according to the maximum output power of the communications equipment.

Rated maximum out- put power of trans- mitter [W]	Separation distance d [m] according to frequency of transmitter		
	150 kHz to < 80 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	80 MHz to < 800 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{3} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance **d** in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where **P** is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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WARRANTY

For a period of two years after delivery to the end-user, we shall replace free of charge equipment that can be proven to have defects in material or workmanship. In doing so we cannot accept to bear the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Complete warranty card upon purchase/delivery and return immediately to:

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To be completed by the supplier/importer:
Company stamp/signature:

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Tipo del aparato: Número de serie:

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Firma/Fecha:

To be filled out by instrument owner:
Return address/company stamp:

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KARL STORZ Endoscopy South Africa, (Pty) Ltd.
8th Floor Convention Tower
Cnr Heerengracht & Coen Steytler, Foreshore
Cape Town 8001, South Africa
P.O.Box 6061, Roggebaai Cape Town,
8012, South Africa
Phone: +27 21 417 2600, Fax: +27 21 421 5103
E-Mail: info@karlstorz.co.za

TOO KARL STORZ ENDOSCOPY Kasachstan
Khodjanova 17
050060 Almaty, Kazakhstan
Phone/Fax: +7 72 72 49 43 63, +7 72 72 49 41 00
e-mail: KarlStorzKazakhstan@itte.kz

KARL STORZ Endoscopy India Private Ltd.
C-126, Okhla Industrial Area, Phase-1
New Delhi 110020, India
Phone: +91 11 26815445-51,
Fax: +91 11 268129 86
E-Mail: karlstorz@vsnl.com

KARL STORZ GmbH & Co. KG
Resident Representative Office
80/33 (44/19) Dang Van Ngu,
F.10-Q. Phu Nhuan
Ho Chi Minh City, Vietnam
Phone: +848 991 8442, Fax: +848 844 0320

KARL STORZ Endoscopy (S) Pte. Ltd.
#05 – 08 San Centre, 171 Chin Swee Road
Singapore 0316, Singapore
Phone: +65 6376-1066, Fax: +65 6376-1068
E-Mail: karlstorz@pacific.net.sg

Karl Storz Endoscopy Australia Pty Ltd
15 Orion Road Lane Cove NSW 2066
P O Box 50 Lane Cove NSW 1595
Phone: +61 (0)2 9490 6700, Fax: +61 (0)2 9420 0695
E-Mail: info@karlstorz.au

KARL STORZ Endoscopy China Ltd.
Hong Kong Representative Office
Unit 1601, Chinachem Exchange Square
1 Hoi Wan Street, Quarry Bay,
Hong Kong, People's Republic of China
Phone: +8 52 28 65 2411, Fax: +8 52 28 65 4114
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Beijing Representative Office
Room 610, China Life Tower
No. 6, Chaowai Street
Beijing, 100020, People's Republic of China
Phone: +86 10 8525 3725, Fax: +86 10 8525 3728
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Unit 3901-3904, Tower 1 Grand Gateway
No. 1 Hong Qiao Road
Shanghai, 200030, People's Republic of China
Phone: +86 21 6113-1188, Fax: +86 21 6113-1199
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Chengdu Representative Office
F-5, 24/F., Chuanxing Mansion,
No. 18 Renming Road South
Chengdu, Sichuan, 610016, People's Rep. of China
Phone: +86 28 8620-0175, Fax: +86 28 8620-0177
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy China Ltd.
Guangzhou Representative Office
Room 1119-20, Dongshan Plaza
69 Xianlie Road Middle, Dongshan District,
Guangzhou, Guangdong, 510095,
People's Rep. of China
Phone: +86 20 8732-1281, Fax: +86 20 8732-1286
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy Asia Marketing Pte Ltd
3791 Jalan Bukit Merah
06-11 e-Centre @ Redhill
Singapore 159471, Singapore
Tel. No. +65 63761066, Fax. No. +65 63761068
Email : infoasia@karlstorz.com.sg
Email : servicesasia@karlstorz.com.sg

KARL STORZ Endoscopy Singapore Sales Pte Ltd
3791 Jalan Bukit Merah
#06-07 e-Centre @ Redhill
Singapore 159471
Tel. No. +65 65325548, Fax No. +65 65323832
Email : infosp@karlstorz.com.sg
Email : servicesgp@karlstorz.com.sg

KARL STORZ Endoscopy Japan K. K.
Bois Hongo Building 6F, 3-42-5 Hongo
Bunkyo-ku, Tokyo 113-0033, Japan
Phone: +81 3 5802-3966, Fax: +81 3 5802-3988
E-Mail: info@karlstorz.co.jp
www.karlstorz.com

STORZ
KARL STORZ—ENDOSKOPE

KARL STORZ GmbH & Co. KG

Mittelstraße 8
78532 Tuttlingen

Postfach 230
78503 Tuttlingen
Germany

Telefon: +49 (0)7461 708-0
Telefax: +49 (0)7461 708-105
E-Mail: info@karlstorz.de
Web: www.karlstorz.com