



The ISR-Pro Dry Salt Aerosol Generator

Salt therapy on demand - anywhere - anytime - any space

User Manual

The ISR-Pro

A truly portable, self contained, go anywhere, salt therapy system.

Compact size, rechargeable battery and single use salt packets give one the freedom to enjoy healthier, clearer lungs, and freer breathing without being limited to salt room locations and hours of operation.

Create your own personal salt room experience, at your convenience whether you are at home, on the road, staying in a hotel, visiting relatives or camping in the great outdoors (where the allergens lurk).

Not intended to diagnose, treat, cure or prevent any medical condition. If you are unsure as to whether dry salt aerosol would be beneficial for you, please consult your physician.

What's In The Box

Machine (3 parts)


Charging cord

Cleaning Brush

Carrying bag

Quick Start Manual

The ISR-Pro is operable with either the power cord, or battery.
Please charge the machine 6 hours prior to first use.

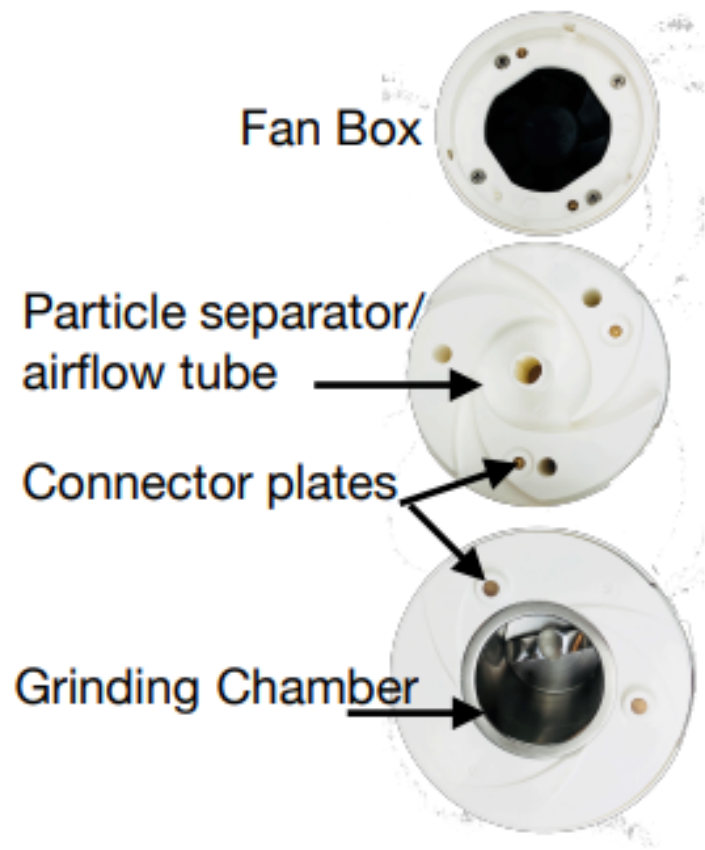

Instructions - Loading the Machine

If the machine has been transported or left in a cold or damp environment, open the machine and let it warm up to room temperature and humidity before using. Condensation will cause the salt to clump.

The two halves are held together magnetically. The top pops off with gentle pressure. Hold the machine firmly and push up on the grooved middle area with your thumbs. Once the magnetic connection is broken, remove the top and rest upside down on a table while loading the salt.

Dispense the contents of one full packet of ISR prepackaged salt into the grinding chamber.



To replace the top, align the logo over the button panel and it will snap together automatically.

Press the power button to begin the flow of dry salt aerosol.

The machines are shipped preset to run for 5 minutes and stop. This is more than enough time for single, at home sessions in an open space larger than 10 x 10 x 8.

Use a fresh packet of salt for each session. Salt remaining in the chamber will accumulate moisture and clog the machine.

Quick Start Guide

1. Please charge it for at least 6 hours but you can operate it while still charging after about 5 minutes. Thereafter, charge it after every 5th session or at least once per week **OVERNIGHT**. Always charge at least 6 hours. For Spa usage, charge it each night.
2. Notice the master switch on the bottom to turn on the power and we recommend that you keep the micro-switches on the factory setting for optimal use.



3. Remember to dump the salt residue from the grinding bowl and brush the "swooshes" in the upper assembly to keep proper air circulation after EVERY session. There should be about 4 grams of salt to dump after each session.
4. The fan assemble should be cleaned after every 5th session or at least one per week. Take note when you reassemble to match up the dots on the side of the fan with the middle unit.
5. Simply brush it out the middle section with the included brush.
6. Take a few extra seconds to run your finger over the contact points to remove any invisible layer of salt that might affect your connections.
7. **DEEP CLEAN THE ELECTRICAL CONNECTION AT LEAST WEEKLY TO MAINTAIN PROPER OPERATION. SEE REMAINDER OF USER MANUAL FOR DETAILS.**

Instructions - Loading the Machine

If the machine has been transported or left in a cold or damp environment, open the machine and let it warm up to room temperature and humidity before using. Condensation will cause the salt to clump.

The two halves are held together magnetically. The top pops off with gentle pressure. Hold the machine firmly and push up on the grooved middle area with your thumbs. Once the magnetic connection is broken, remove the top and rest upside down on a table while loading the salt.

Dispense the contents of one full packet of ISR prepackaged salt into the grinding chamber.



Cleaning the Machine

When the session has ended, open the machine and brush the leftover salt into a trash can or sink (do not get the machine wet). Gently brush away any salt grains, and salt dust from the grinding chamber, and under and around the blade of the bottom half.

Brush the salt dust from the air chamber, and clear the tube leading to the grinder.

The fan assembly is completely removable. Twist gently to unlock. brush all salt dust off of fan blades, inner and outer walls. **The fan does not need to be cleaned after each use, just when it has accumulated a good bit of salt, or at the end of the day when finished using.**



KEEP DRY

KEEP DRY

BRUSH

Make sure to remove all salt dust on the metal connectors. If they are not able to make clear contact between the two halves of the machine, the fan will not operate and a thick salt dust will come out the air intake vents along the side of the machine.

Replace fan assembly and connect top and bottom halves. Store in a dry area at room temperature.



One year warranty on machine. Void if machine is dropped, if it gets wet or if you use salt other than the ISR prepackaged salt

Deep Cleaning of all electrical connections is necessary to maintain proper operation of the ISRPRO.

Microscopic buildup of salt on the electric connections can cause the machine to fail to operate.

If your machine is not operating properly and to keep it operating properly, please do this deep cleaning process weekly.

First of all, on the bottom of the machine, switch the black master power switch to the off position.....push it toward the center of the machine

Also, make sure the two tiny switches are both in factory settings with both pushed toward the center of the machine.

While the machine is turned off and in factory setting, the microprocessor can reset while you perform the cleaning process below.

The ISRPRO operates via the metal contacts you can see in each section of the machine. These metal contacts need to be cleaned regularly to avoid a build-up of salt. If the salt is allowed to remain on the contacts and the humidity is not controlled, the humid salt can affect the unit's ability to transfer power to each component. If this happens, it appears like the master on/off button on the front is not working, but it is really an issue with the metal contacts. Here are 2 images of a unit we put through a high humidity test: This unit would not sense the fan nor turn on the motor.



If you have a brass brush, it is ideal for this clean-up. A brass brush will allow you to clean those contacts with a little more abrasive material than the supplied brush. If you don't have a brass brush then a stiff toothbrush will help to do this cleanup (it can be very slightly damp to aid in breaking up the salt)

After cleaning the flat contact, it will look like this (shiny brass color). Also, be sure to brush the retractable brass contacts opposite the metal contacts on each section of the machine.



I

This should keep your ISRPRO operating well for many years.


How to Use

The factory setting is the recommended setting to run the ISRPRO.

On the bottom of the machine there is a small depression containing the black on/off switch. This will permanently turn the power off. This is useful when traveling so the machine does not come on accidentally if the start button is bumped.

Next to the power switch there are two smaller switches. The machine will be shipped with them both switched down toward the center of the machine (both switches down)

This is the most efficient setting.

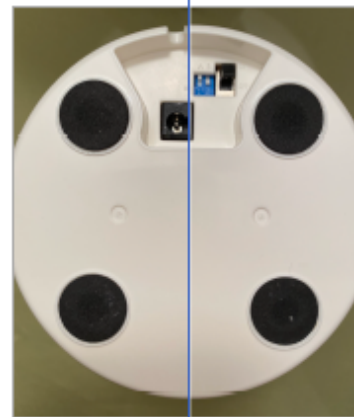

Bottom of machine

www.infinitysaltair.com

info@infinitysaltair.com

Charlotte, North Carolina