


ANYKIT

Otoscope User Manual

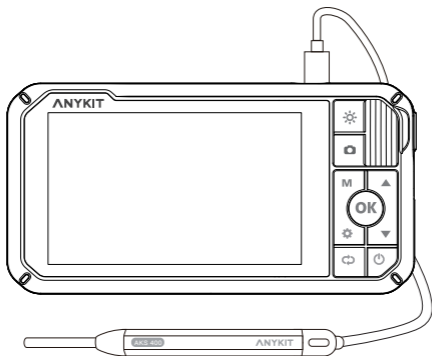

Model No.: AKS400

READ BEFORE USE

Please keep the user manual for future reference.

You can also scan this QR code to download the PDF user manual, view and enlarge on your device.

By using this product, you signify that you have read, understand, and accept the terms and conditions here.



Read the following documents before using.

- (1) Safety Guidelines
- (2) Quick Start Guide
- (3) User Manual

It is recommended to watch all tutorial videos on the official Anykit website and read safety guidelines before using for the first time.

Prepare for your first use by reviewing the quick start guide and refer to this user manual for more information. It is recommended to watch tutorial videos and notices before using for the first time.

Video Tutorials

scan the QR code to watch the AKS400 tutorial videos, which demonstrates how to use the AKS400 safely.



U.S. Local Customer Support

Tel: 877-888-7979 | Mon.-Fri. 8:00-5:00 PST

SAFETY WARNING

1. You must read the instruction manual carefully before using.
2. Keep out of reach of children. Children are not allowed to use this product.
3. You must put the protective cover onto the otoscope before using, then sanitize all the areas that contact the skin.
4. Please take great care when using ear spoon accessories to clean earwax. DO NOT insert them too far into the ear canal.
5. Don't clean your ear canal' excessively. Prolonged use may cause discomfort or damage to the ear canal.
6. Stop using the otoscope if it overheats.
7. When using this product be aware of people around you. Be careful that nobody bumps into your hand while holding the device in your ear.
8. Stop using this product immediately if you experience any discomfort.
9. This product can NOT replace a doctor or medical provider's diagnosis or treatment. Contact your doctor immediately if you experience any discomfort.T

TABLE OF CONTENTS

ABOUT ANYKIT-----	1
FUNCTIONS-----	2
SPECIFICATIONS-----	3
USING YOUR INSPECTION CAMERA-----	4
INSTRUCTIONS-----	5
LANGUAGE SETTINGS-----	6
NOTICE-----	7
USER INTERFACE-----	8
USER MODE-----	9
TF CARD INSTRUCTION-----	10
MAINTENANCE-----	11
CHARGING-----	12
TROUBLESHOOTING-----	13
WARRANTY-----	14

ABOUT ANYKIT



ANYKIT

Anykit is a trendy and practical tool brand. We focus on simplicity and ease of use, providing people with convenience in their everyday lives. Our products including otoscopes, endoscopes, and a variety of household tools to enable DIY enthusiasts to embrace creativity and convenience.







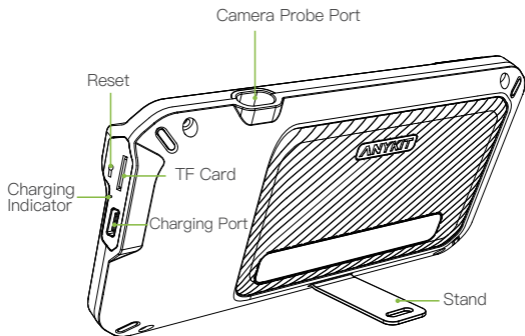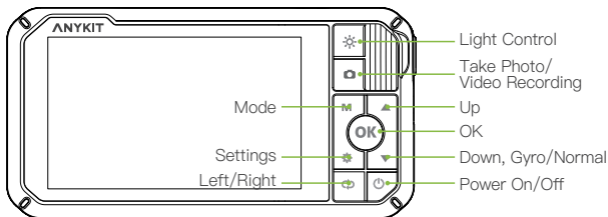



Follow Anykit on Amazon for More





FUNCTIONS



SPECIFICATIONS


			
Capacity	Input	Camera Dia.	Camera Resolution
2500mAh	5V = 1A	3.9mm	1.0 MP

			
View Angle	Depth of Field	Screen	Image Resolution
70°	10–20mm	4 Inch	1920*1080


			
Video Resolution	LED light	Operating Temperature	Operating Hours
1280*720	6 Adjustable LEDs	-4°F to 140°F (-20°C to 60°C)	About 4h

USING YOUR INSPECTION CAMERA

Remove the plastic screen cover before use.

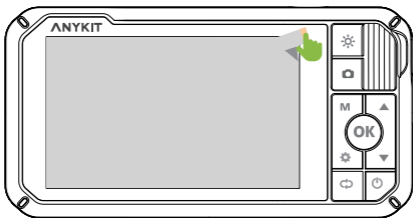

Connect the camera probe to the monitor (Make sure the camera is connected firmly to prevent the “no camera” error message.)

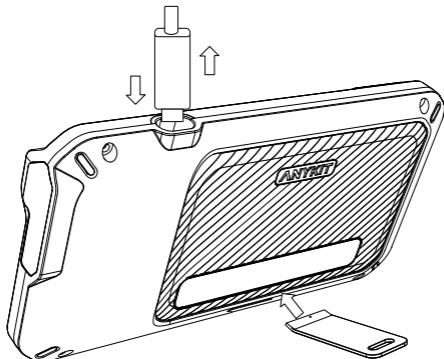

Stand Installation



INSTRUCTIONS

Disinfect the specula or ear spoon by wiping them with 75% alcohol solution before and after EVERY use.




For safer and cleaner ears, please install the speculum or ear spoon before use.



Tip on the speculum

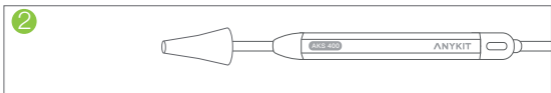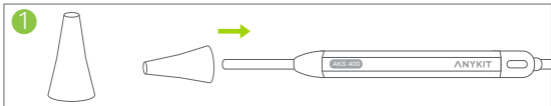

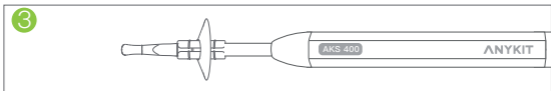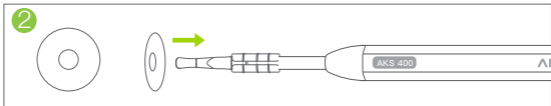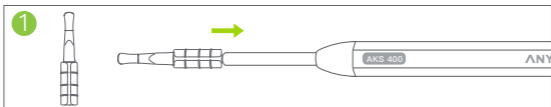

Tip on the ear spoon




SETTINGS


Press the “” button to enter settings page.

Press “” and “” to select the items you need.


Press “**OK**” to save your choice.

Settings

 Date/Time

 Format SD Card


 Restore Factory

 Language


 Version

NOTICE

1. Connect the camera probe before turning on the device. Restart the device after replacing the camera probe.
2. Please ensure that your USB-C camera probe is connected to the TOP of the monitor and not to the charging port on the BOTTOM.
3. When using the otoscope, don't hold the lens in a vertical, up and down. position or else the gyroscope may fail.
4. Disinfect the surface of the device and accessories before contacting skin.
5. Please keep the camera clean to avoid blurry image. (Refer to maintenance how to clean the camera).
6. Please keep a focal length of about 10mm–20mm (0.4–0.8 inch) to achieve the best quality image (placing the scope too far from your focal point may result in blurry images).

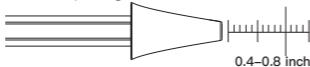

7. The ear canal is not perfectly straight, gently adjust the position of the otoscope and gently pull the top of your ear up and back to help view your eardrum easily.

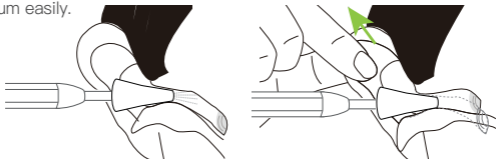

USER INTERFACE


Mode: Press “M” to switch the three modes:
“📷” photo taking/ “📹” video recording/ “🖼️” playback

TF card status: TF card inserted/no card

Left/Right

Gyro/Normal


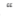

Battery

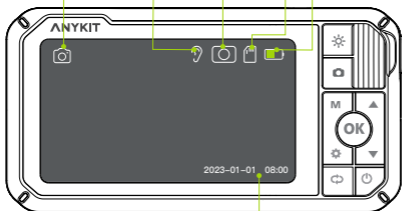

Date and time: modify the date and time in Settings menu



USER MODE

Press “  ” to take a photo or start recording.

At playback mode, press “   ” to select photo or video to view , and press “  ” to delete photo or video.



Take Photos



Press  to switch to the take photos, and press  to take a photo.


Video Recording




Press  to switch to video recording mode, press  to start recording.


Normal



Press  to switch to the take Gyro Mode


Gyro Mode

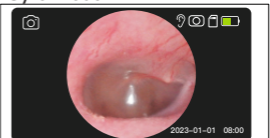

Press  to switch to the take Normal Mode

Left



Press  to switch to the take Right Mode


Right



Press  to switch to the take Left Mode


TF CARD INSTRUCTION

The device comes with a 32G TF card inserted in.

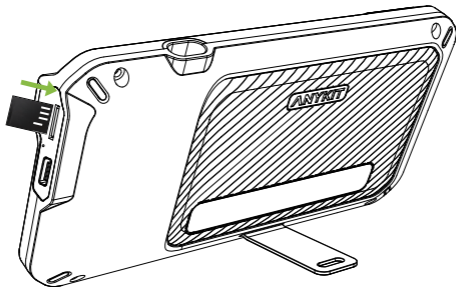

- ① Without the TF card, you can only preview the video stream.
- ② Power off the device before installing or removing the TF card.
- ③ To ensure the device works properly, please use a Class 10 high-speed TF card.

Copy photo and video files to computer

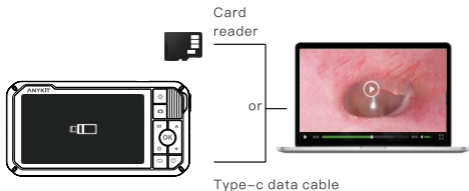

- ① Use a TF card reader to copy data directly.
- ② Connect the device to a computer with a Type-C data cable, power it on for U-disk mode.

MAINTENANCE

To extend battery life, please refer the following tips:

Please use a 5V household charger that meets the safety regulations to charge this device.


This device does not support PD charger.

This device can work while charging.


If you do not use the device for a long time, please ensure that it is charged at least once every 3 months to prevent irreparable damage caused by excessive battery discharge.

To optimize the image quality and maintain personal hygiene, please following these tips to clean the camera before and after EACH use:

1. Clean the lens carefully with alcohol pads.
2. Wipe the lens grooves with alcohol-soaked Q-tip.

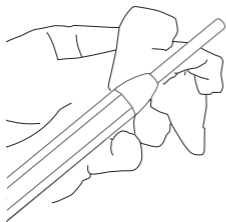

Alcohol

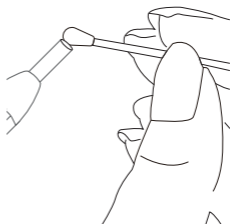


Q-tip

This can prevent ear wax from sticking to the tip of the camera and causing blurry images. You can apply pressure when cleaning the camera to remove stubborn ear wax completely.


CHARGING

Please use a 5V household charger that meets the safety regulations to charge this device.

This device does not support PD charger.

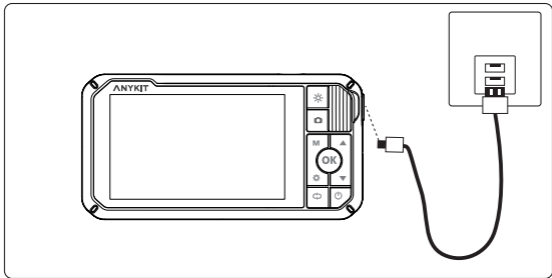

Please press the reset button if the screen freezes or the buttons do not respond.

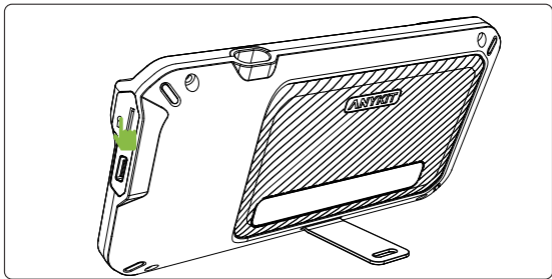

TROUBLESHOOTING

No camera

Please make sure you have plugged the camera probe in completely until you hear a snap. If it still does not work please feel free to respond via support@anykit.com or contact our support line at 877-899-8809, Mon.-Fri. 8:00-5:00 PST

Blurry image

The focal length is fixed at 10mm-20mm (0.4-0.8 inch), so please adjust the distance between the camera lens and your ear to get focused if you cannot see the image clearly.

Blurry image after several uses

Please clean the camera before and after every use to avoid blurry image. Blurred images after a period of use are caused by the earwax on the camera lens. (In general, earwax is oily and can cause glare, unclear image) .

Cannot thoroughly clean the lens

Please follow the steps:

- 1.Clean the lens carefully with alcohol pads.
- 2.Wipe the lens grooves with alcohol-soaked cotton swab.

WARRANTY

30-Day Guarantee

We will give you a full refund if you are not satisfied with the product.

One-Year Warranty

We offer a limited warranty that commences from the date of purchase of the product.



Visit anykit.com or scan the QR code for more details.

CUSTOMER SUPPORT

If you have any questions or concerns about your product,
please contact our Customer Support Team



1-877-888-7979
Mon.-Fri. 8:00-5:00 PST(US)



support@anykit.com



www.anykit.com



@anykit



@anykit



Facebook Messenger

