
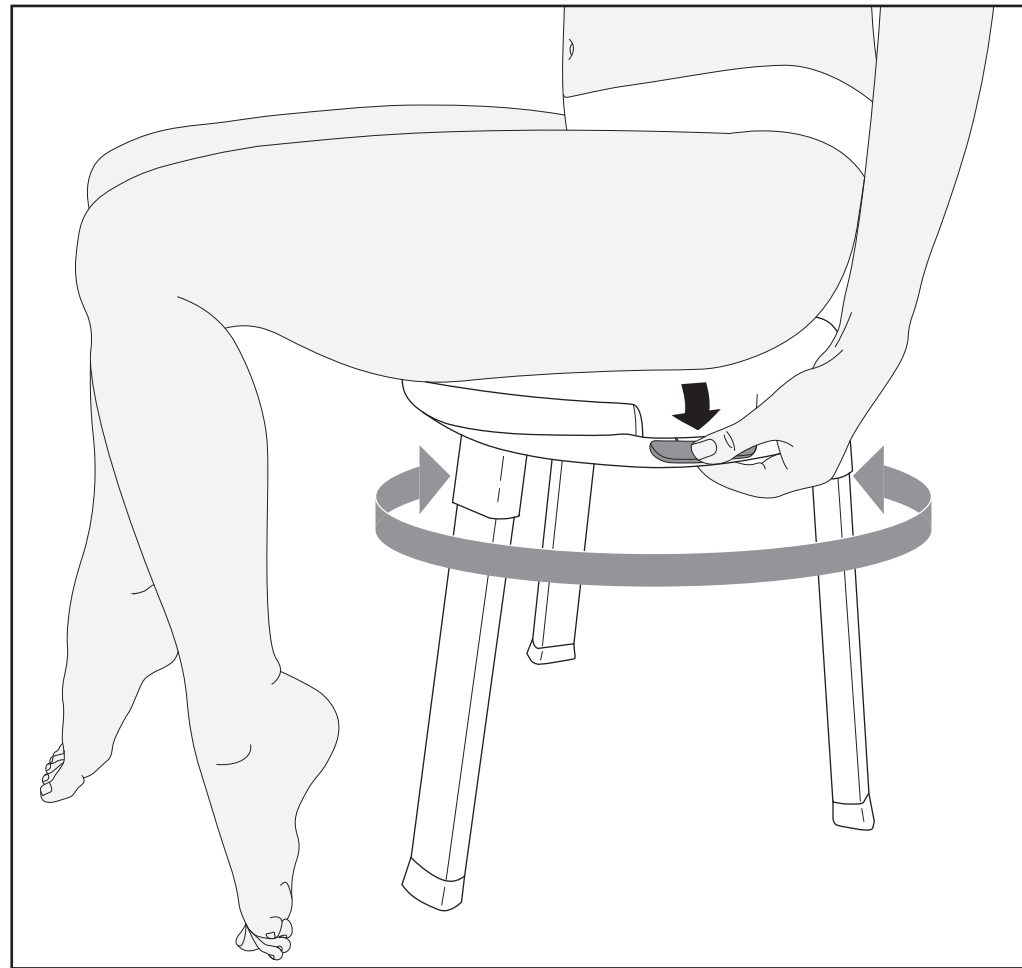
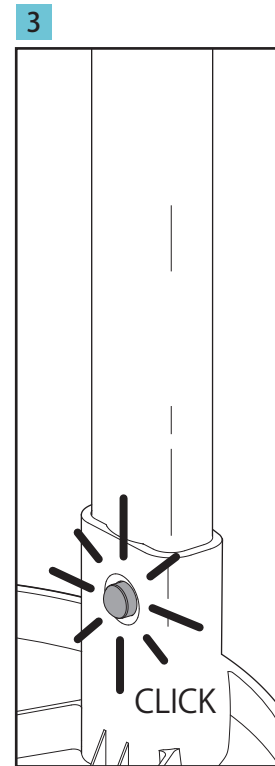
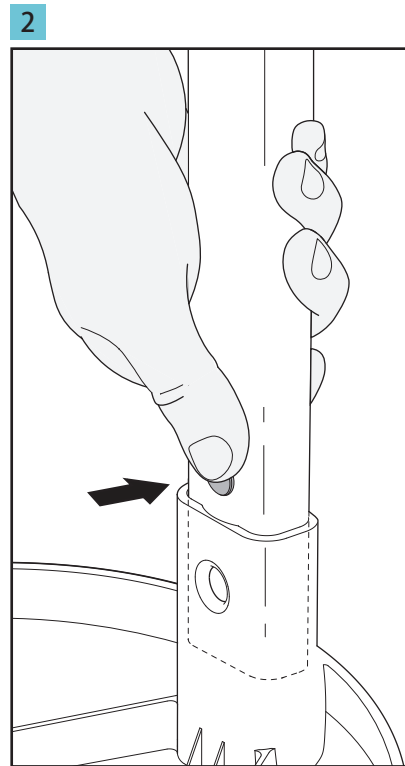
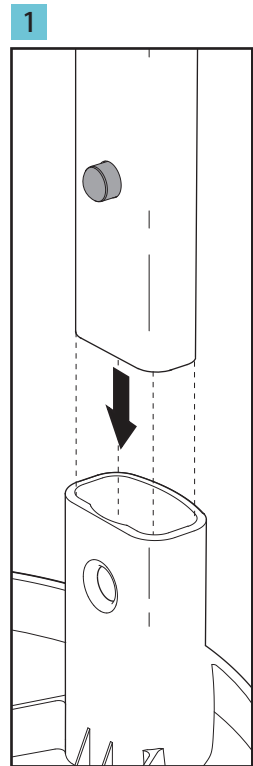
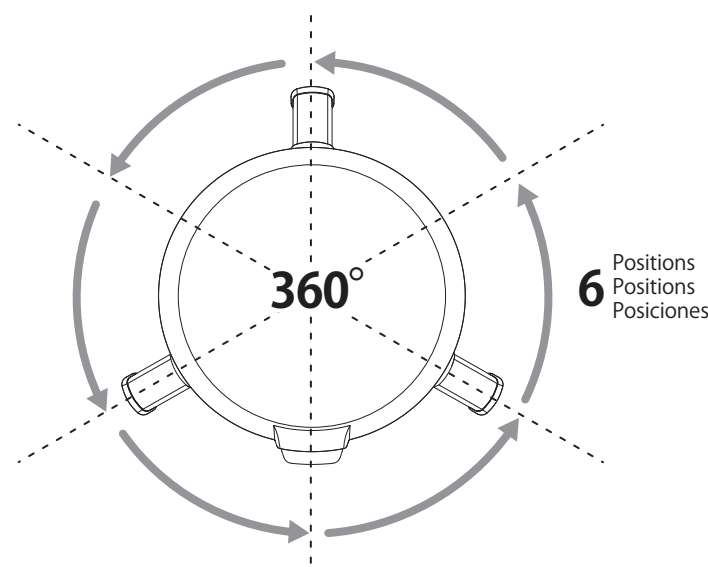
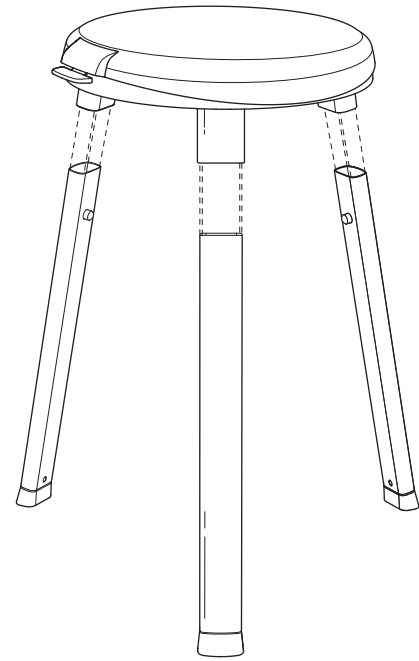
SWIVEL 360 Shower Seat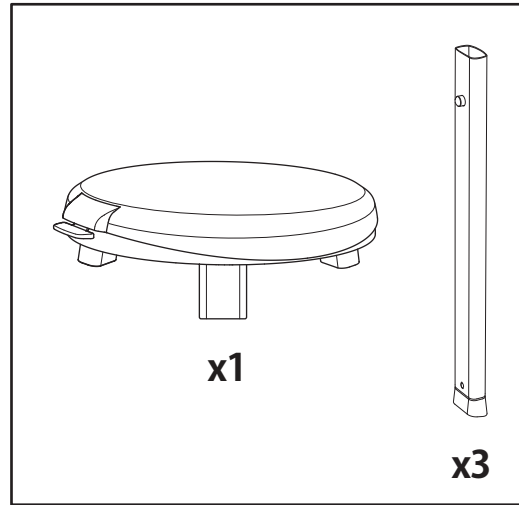

the Spa Seat™

SWIVEL 360 Shower Seat

Proper use:

Only use the stool in accordance with the instructions.
Do not use as a step, as training equipment, or as a climbing aid.
The stool is suitable for a maximum body weight of 300 lbs (136 kg).
Please do not use the stool if you weigh more than 136 kg.
Permanent damage may be caused to the stool if the maximum weight is exceeded.
Do not remove legs once assembled.
This product is intended for adults and not to be used by babies or small children, whether supervised or unsupervised.
This product should not be used to sit in the tub while holding a small child on your lap.

Cleaning:

Clean from time to time with a cloth.
You may use stain-removing cleaning agents.
Do not use cleaning agents containing alkali, acid, chlorine and abrasive additives.

Liability exclusion:

Better Living is not responsible for injuries and damage arising from the non-observance of the instructions and safety measures stated above.

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Do not remove legs once assembled.
This product is intended for adults and not to be used by babies or small children, whether supervised or unsupervised.
This product should not be used to sit in the tub while holding a small child on your lap.

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