


ACCESS DIAGNOSTIC TESTS UK

Do not open the foil wrapped kit until the time you want to proceed with the test. To perform a test, you will need a fresh urine sample, this should be cooled to room temperature, (approx. 20 minutes is adequate) and a watch or timer.

Dip the test into the sample for 10-15 seconds, do not dip beyond the max dip line marked on the strip.

CONDUCTING A TEST

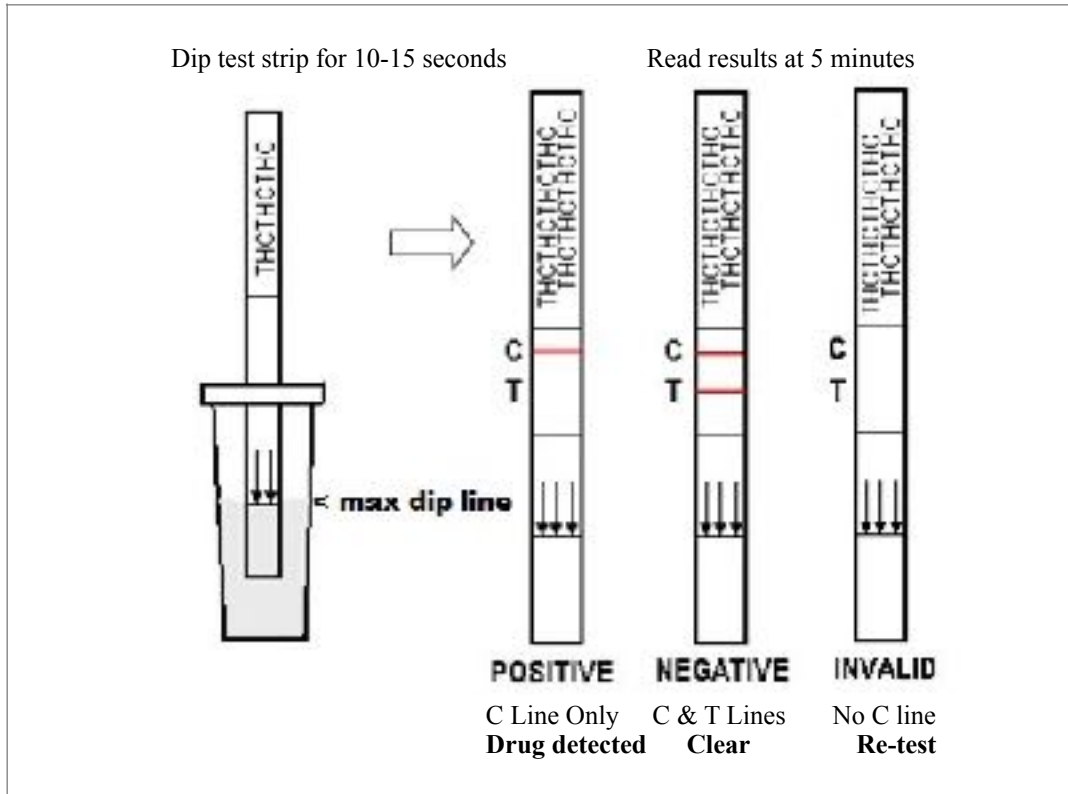

CODING ABBREVIATIONS ON PACKAGING AND STRIP HANDLES.

- THC = MARIJUANA & CANNABIS.
- MOP/MOR = MORPHINE (HEROIN, OPIATE)
- AMP = AMPHETAMINE
- MTD = METHADONE.
- MET = METHAMPHETAMINE.
- COC = COCAINE & CRACK.
- BZO = BENZODIAZEPINE.
- MDMA = ECSTASY
- KET = KETAMINE
- PGB = PREGABALIN
- LSD = LSD
- ETG = ALCOHOL.
- ZOL = ZOLPIDEM.
- COT = COTININE/NICOTINE
- K2 - K3 = SYNTHETIC CANNABIS
- FYL = FENTANYL
- TML = TRAMADOL
- MDPV = BATH SALTS
- OXY = OXYCODONE
- ZOP = ZOPICLONE

IMPORTANT. Do not dip the strip beyond the max dip line as shown above.

The result showing at 5 minutes is the accurate reading for the test performed.

Test lines in some tests may be VERY THIN & FAINT > ANY test line showing in the presence of a Control line must be read as a NEGATIVE test result.

For detection time guides and more information or to re-order visit www.ukdrugtesting.co.uk

All positive tests should be confirmed. If prescription medication is being taken or you need further information e-mail us at customerservices@accessdiagnostics.co.uk ©2017 Tel 01263 731168