


ACCESS DIAGNOSTIC TESTS UK

www.ukdrugtesting.co.uk

Thank you for purchasing a rapid drug test kit. Please read the following advice before proceeding with your tests.


Do not open the foil wrapped kit until the time you want to proceed with the test. To perform a test you will need a fresh urine sample, this should be cooled to room temperature, (approx. 20 minutes is adequate) and a watch or timer.

CONDUCTING A TEST
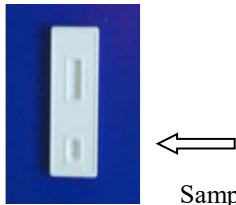
<p style="text-align: center;">SINGLE CASSETTES.</p> <ul style="list-style-type: none">• When the urine sample is ready, remove the cassette from its foil pouch.• In the pouch is the test cassette, and a pipette. (discard the absorbent pouch)• Place the cassette on a clean level surface• Using the pipette, add 3 full drops of urine into the sample hole. Allow 5 seconds between each drop.  <p style="text-align: center;">Sample hole.</p> <ul style="list-style-type: none">• Read the result at 5 minutes.	<p style="text-align: center;">CODING ABBREVIATIONS ON PACKAGING AND CASSETTE PANELS.</p> <ul style="list-style-type: none">• THC = MARIJUANA/CANNABIS.• MOP = Opiates 300ng (HEROIN).• AMP = AMPHETAMINE.• MTD = METHADONE.• MET = METHAMPHETAMINE.• COC = COCAINE & CRACK.• BZO = BENZODIAZEPINE.• MDMA = ECSTASY• PCP = PCP• COT= COTININE (NICOTINE) <p style="text-align: center;">IMPORTANT.</p> <ul style="list-style-type: none">• Do not attempt to interpret the results beyond 10 minutes from adding the sample. The result showing at 5 minutes is the accurate reading for the test performed.
--	---

READ THE RESULTS

- Each test kit has a control line and a test line. These are marked C, and T.

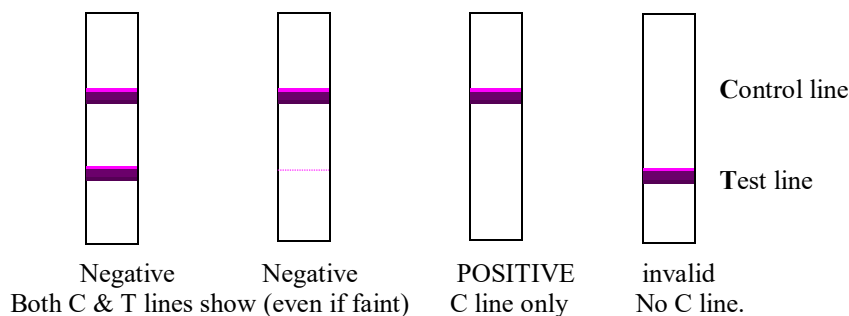

Negative tests can be read at 1-2 minutes. Allow a full five minutes development time for positive results. The line intensity may vary, but any line, no matter how faint in the test line zone, indicates a negative test. For the test to be valid, the control line must always show. All positive tests should be confirmed. If prescription medication is being taken or you need further information? E-mail us at customerservices@accessdiagnostics.co.uk ©2010 ADTUK Tel 01263 731168