

Warning!

Your vision and/or attention will be limited and/or impaired while using the Tru-Vu Omniscreen Viewhole Eyeset. Therefore, it is important to take extra caution if you are walking, running, training, etc.

Do NOT use the Tru-Vu Omniscreen Viewhole Eyeset while operating a motor vehicle, heavy equipment or while performing ANY hazardous tasks requiring attention.



Mind Alive Inc.

6716 75 St NW
Edmonton, Alberta
Canada T6E 6T9

Toll Free: 800.661.MIND
Phone: 780.465.MIND
www.mindalive.com
E-mail: info@mindalive.com

Tru-Vu Omniscreen Viewhole Eyeset



For DAVID Audio-Visual
Entrainment Devices



***Enhance your AVE experience
with eyes open***

Eyes Open Entrainment

Open Your Eyes

The viewhole eyeset allows you to use your DAVID while performing various tasks to enhance performance or training with your eyes open.

Activity Suggestions

Reading

Use with SMR for relaxed attention or reading session while reading to improve comprehension.*

Office/Computer Work

Use with SMR for relaxed attention or reading session while working at your computer.*

**If you feel “foggy” or tired while reading, a Beta or Energizer session may be more effective. Use to replace caffeine.*

Activity Suggestions

Sports

Use with SMR for relaxed attention or reading session while training in any sport.

Biofeedback/ Neurofeedback

Use a session with the frequency you are trying to uptrain or inhibit during your Neurofeedback session. Use with alpha or slow alpha sessions such as Schumann during temperature, EDR, HRV and EMG training.

Phobia Desensitization

Use with SMR and/or alpha sessions during desensitization training session.

More Information

For more information on AVE or the DAVID devices, please visit our website at www.mindalive.com

Why Eyes Open

Although we typically recommend you use your DAVID with your eyes closed, there are some instances when eyes open can be more effective.

How to Use

Connect the Eyeset

The viewhole eyeset connects to your DAVID device using the same 2.5mm patch cord as the standard eyeset.

Plug into the Omniscreen () jack of the DAVID device. Connect the cable to the jack located on the lower left side of the eyeset.

Cleaning and Care

Wipe with a clean, soft cloth to remove fingerprints or smudges from the Omniscreen Viewhole eyeset. Glasses cleaner or a mild detergent may be used on the sunglass lens to help remove oils. Alcohol swabs or wipes may be used to disinfect the eyeset between uses.

Disconnect the eyeset cable from the DAVID device and the eyeset before storage.