ElectraStim Flux Firmware History

VERSION	RELEASE DATE	COMMENTS
1.2	10th Jan 2022 NOTE: If your Flux has a serial number of 181238 or higher then it will have V1.2 pre- installed so do not carry out an update.	 Wave/Pulse pattern menus updated to include variable speed options to both synchronised and alternating patterns by utilising the accelerometer tilt functionality. The Sync./Alt. Slow patterns are now called Sync./Alt Fixed. The Sync./Alt. Fast patterns are now called Sync./Alt Variable. Tilting Axis to the left will slow down the pattern rate. Tilting to the right will increase the speed. Battery management algorithm improvement. Output control improvement.
1.1	25th Jan 2021 NOTE: If your Flux has a serial number of 180-950 or higher then it will have V1.1 pre- installed so do not carry out an update.	 Tilt Mode modified so as to utilise both X and Y planes of the integrated accelerometer to control the stimulation frequency between 20Hz and 200Hz and the pulse width from 0 to 160us respectively. A damping function also allows the user to change the response time of tilt motions to create long or short sustains for each plane independently. The line-in and microphone DSP filter settings modified to give a faster response.
1.0	25th Oct. 2019	Original firmware release - No Updates Available