EN Cholesterol Test

The cerascreen® Cholesterol Test is a send-in blood sampling kit to determine your cholesterol levels. The test is used to analyse the concentration of total cholesterol, HDL, LDL and triglycerides as well as the TC/HDL ratio in human capillary blood. The sample is analysed in vitro in our specialised cerascreen® partner diagnostics laboratory.

Your test kit contains

- A** 2 plaster **Č**., 1 swab **Č**., 1 alcohol wipe **Č**., **B** 1 foil pouch (blood collection device) with absorbent sachet **Č**.,

- C** 3 lancets **Č**., **D** 1 return envelope

- E** 1 test-ID card with sticker

Information about the test

Read the instructions for use carefully before you begin and follow the listed steps.


Warnings

- Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.
- Store this kit out of reach of children.
- The kit may be used only for adults aged 18 and over.
- If you are pregnant or are breastfeeding, use the kit only after consulting your therapist.
- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, use the kit only after consulting your therapist.
- Use the lancets only as described in the instructions for use.
- If you have problems with the sight of blood or blood sampling, take the sample only in the presence of another person or contact your therapist.

- We can only analyse your sample if you activate your test with the test-ID of your test-ID card! More details on this can be found in step 16 + 17.**
- This test requires a 12-hour fast (no food or drink, except water).**
- Do NOT open the foil pouch until you're ready to begin collecting your blood sample. The blood collection device must be used within 30 minutes after opening the foil pouch.**
- Unfortunately, it is not possible to analyze blood samples that contain less than the required amount of blood.**
- Do not take a blood sample immediately after strenuous physical activity.**
- This is a blood sampling kit for personal single use only!**

- We can only analyse your sample if you activate your test with the test-ID of your test-ID card! More details on this can be found in step 16 + 17.**
- This test requires a 12-hour fast (no food or drink, except water).**
- Do NOT open the foil pouch until you're ready to begin collecting your blood sample. The blood collection device must be used within 30 minutes after opening the foil pouch.**
- Unfortunately, it is not possible to analyze blood samples that contain less than the required amount of blood.**
- Do not take a blood sample immediately after strenuous physical activity.**
- This is a blood sampling kit for personal single use only!**

- We can only analyse your sample if you activate your test with the test-ID of your test-ID card! More details on this can be found in step 16 + 17.**
- This test requires a 12-hour fast (no food or drink, except water).**
- Do NOT open the foil pouch until you're ready to begin collecting your blood sample. The blood collection device must be used within 30 minutes after opening the foil pouch.**
- Unfortunately, it is not possible to analyze blood samples that contain less than the required amount of blood.**
- Do not take a blood sample immediately after strenuous physical activity.**
- This is a blood sampling kit for personal single use only!**

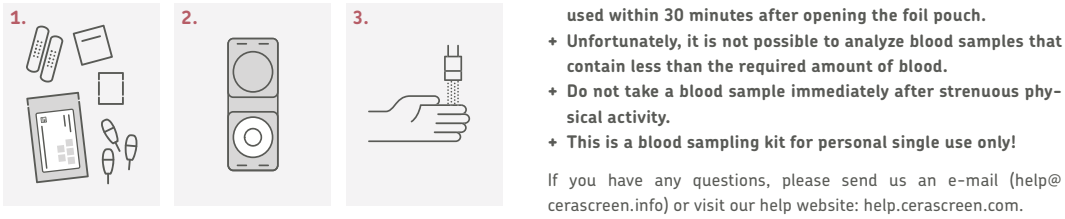- 1.** Place all contents of the test kit on a clean surface. Before you start, choose the sampling spot. **Tip:** the middle and ring finger of the left hand usually works best.

- 2.** Retrieve the blood collection device from the foil pouch. Place the device with collection side up, on a flat and level surface. Do NOT remove the clear cover.

- 3.** Wash your hands with soap and warm water. Hold your preferred hand under warm water for at least 15 seconds to stimulate blood circulation. Dry your hand carefully with a clean towel.

- 4.** Swing your preferred arm around in a circle 5 to 10 times, fully extended if possible, in large movements to stimulate blood circulation. Remove the alcohol wipe from its film and use it to disinfect the tip of the preferred finger. Wait until the finger is dry again (15 sec).

- 5.** Take the lancet. (1) Rotate the protective cover once around its axis. (2) Carefully remove the protective cover. **Caution:** Do not use a lancet with a loose or missing protective cover.

- 6.** Place your hand on a firm surface. Place the lancet diagonally on the tip of your finger as shown in the illustration. **Note:** Do NOT use the centre of the fingertip. Only use the side of the finger tip pad as the puncture site for less discomfort. Press the lancet firmly against the finger until it releases. **Caution:** The lancet can only be released once!

- 7.** Press the thumb of your other hand below the sampling spot and slightly pull back the skin. Encourage blood flow by getting up and holding your hand below the heart. Gently massage your finger from the palm of your hand to your fingertip until a drop of blood forms.

- 8.** Before you start taking the blood, take the swab and wipe off the first drop of blood. Wait until a big drop has formed again.

- 9.** Let the blood drop fall in the centre of the blood collection device.

- 10.** Repeat until you have collected 4 large drops of blood. Apply drops continuously. If blood flow stops, gently massage finger from your palm to the fingertip.

- 11.** After applying 4 drops, wait 3 minutes for blood drops to soak into the white membrane.

- 12.** After taking the blood, use the swab and patch to clean and cover the wound.

- 13.** After absorption, close the blood collection device firmly by folding over the top. Make sure the latches are secure.

- 14.** Remove the square code sticker from the test-ID card and stick it on the top portion of the blood collection device.

- 15.** Place the blood collection device in the return envelope with the address. Keep the test-ID card – you will need this to activate your test. Close the envelope carefully and send it immediately by post free of charge.

- 16.** If you are a new customer, please create an account at myce-

rascreen.com or with the **my cerascreen®** app. See our FAQs on help.cerascreen.com for further instructions.

- 17.** To create your result report, log in via the **my cerascreen®** app (Android, iOS) or on my.cerascreen.com. Insert your six-digit ID-number according to your test-ID card under 'activate test' and follow the instructions.

- 18.** Dispose of the used parts of the test kit in the transparent plastic bag in the household waste.

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