EN Food Reaction Test

The cerascreen® Food Reaction Test is a send-in sampling kit for a screening test that covers 56 foods when suspecting allergies and food intolerance. The test is used to determine specific IgE and IgG4 in human capillary blood. The **process** is pain-free and can be done easily from home. The sample is analysed in vitro in the specialised cerascreen partner diagnostic laboratory.

Your test kit contains

A 1 alcohol wipe **€€**

B 1 plaster **€€**, 1 swab **€€**

C 1 dried blood spot card

D 2  **€€ 0050**

E 1 return envelope

F 1 test-ID card with sticker

Food Reaction Test

Instructions for Use

- Návod k použití • Gebrauchsanweisung
- Brugsanvisning • Instrucciones de uso • Käyttöohjeet
- Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso
- Gebruiksaanwijzing • Brugsanvisning • Instrukcja użycia
- Instruções de Utilização • Användningsanvisningar

Information about the test

 Read the Instructions for Use carefully before you begin and follow the listed steps.

Warnings

- Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.
- Store this kit out of reach of children.
- The kit may be used only for adults aged 18 and over.
- If you are pregnant or are breastfeeding, use the kit only after consulting your therapist.
- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, use the kit only after consulting your therapist.
- The lancets are exclusively for your own use.
- If you have problems with the sight of blood or blood sampling, take the sample only in the presence of another person or contact your therapist or pharmacist.

Important Information

We can only analyse your sample if you activate your test with the test-ID of your test-ID card! More details on this can be found in step 17. Note that a meaningful and conclusive result can only be provided if you maintain a mixed diet (for around 2 weeks) before taking the sample. Food intolerance and allergies can only be detected if you have consumed those foods recently. Attention: Do not consume foods that are known to cause a strong reaction within your body (Anaphylactic shock may occur!). Also, note that certain medication (e.g. antihistamines) can affect the test result. More information is available in the FAQ section on our website.

➔ Read the Instructions for Use carefully before you begin and follow the listed steps.

• Do not take a blood sample immediately after strenuous physical activity.

• If you have any difficulties performing the blood sample or if the kit is incomplete, contact our customer service department, who will treat this matter confidentially.

➔ This is a blood sampling kit for **single** use self-testing!

• Do not touch the dried blood spot card within the markings and keep it dust free.

If you have any questions, please send us an e-mail (help@cerascreen.com) or visit our help website: help.cerascreen.com

cerascreen®

CRS3-LMR_INT_000
Version 02/2019

help@cerascreen.com
www.cerascreen.com

Directions for Use

- Place all contents of the test kit on a clean surface. Before you start, choose the sampling spot. **Tip:** the middle and ring finger of the left hand usually work best.
- Wash your hands with soap and warm water. Hold your preferred hand under warm water **for at least 15 to 30 seconds** to stimulate blood circulation. Dry your hand carefully with a clean towel.
- Swing your preferred arm around in a circle 5 to 10 times, fully extended if possible, in large movements to stimulate blood circulation.
- Remove the alcohol wipe from its film and use it to disinfect the tip of the preferred finger. Wait until the finger is dry again (15 sec).
- Take the lancet. (1) Rotate the protective cover around its axis. (2) Carefully remove the protective cover. **Caution:** Do not use a lancet with a loose or missing protective cover.
- Place your hand on a firm surface. Place the lancet diagonally on the tip of your finger as shown in the illustration. Press the lancet firmly against the finger until it releases. **Caution:** The lancet can only be released once!
- Press the thumb of your other hand below the sampling spot and slightly pull back the skin.
- Hold your hand horizontally at hip height (for instance while standing at the edge of a table) and massage your finger from the palm of the hand down to the fingertip until a blood droplet forms.
- Before you start taking blood, use the swab to remove the first blood drop. Wait for a new large drop of blood to form. Support this process by standing up and holding your hand below heart level.
- Let a large blood drop fall onto the marked circle on the dried blood spot card. In case the drop doesn't fall its own from your finger, hold the dried blood spot card carefully against the drop without touching the paper with your finger.
- For the best evaluation of the sample, the largest possible drop of blood is required. The blood drop should cover the circle inside the marking completely.
- Repeat this process until you have completed four circles.
- After taking the blood sample, press the swab against the location of bleeding. Once the bleeding slows down, apply the plaster to the wound.
- Remove the square code sticker from the test-ID card and stick it on the dried blood spot card in the area intended for date and name.
- Allow the dried blood spot card to dry for 1 to 2 hours at room temperature. Ensure to place it flat on a dry surface, protected from direct sunlight.
- Place the dried blood spot card in the return envelope with address. Keep the test-ID card – you will need this to activate your test. Close the envelope carefully and send it immediately by post free of charge.
- To create your result report, log in via the My Cerascreen® app (Android, iOS) or on my.cerascreen.com. Insert your six-digit ID-number

according to your test-ID card under 'activate test' and follow the instructions.

18. Dispose of the used parts of the test kit in the transparent plastic bag in the household waste.

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