
Vitamin B12 Test

Instructions for Use

- Návod k použití + Gebruiksaanwijzing
- + Brugsanvisning + Instrucciones de uso + Käyttöohjeet
- + Mode d'emploi + Oδηγίες χρήσης + Istruzioni per l'uso
- + Gebruiksaanwijzing + Bruksanvisning + Instrukcia užycia
- + Instruções de Utilização + Användaransvarings

Your test kit contains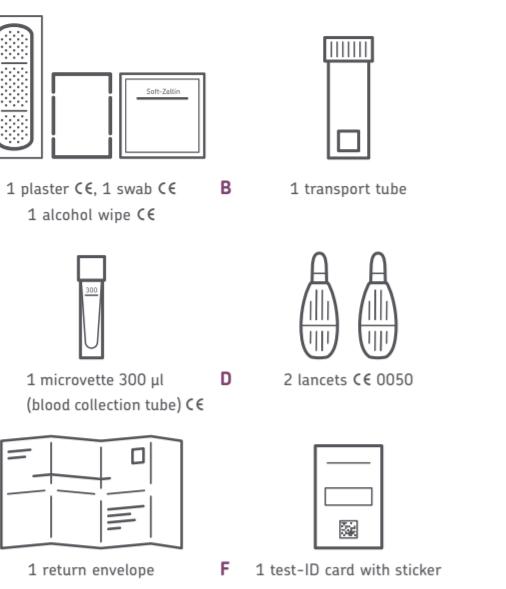

Information about the test

Read the Instructions for Use carefully before you begin and follow the listed steps.

Warnings

- Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.
- Store this kit out of reach of children.
- The kit may be used only for adults aged 18 and over.
- If you are pregnant or are breastfeeding, use the kit only after consulting your therapist.
- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, use the kit only after consulting your therapist.
- The lancets are exclusively for your own use.
- If you have problems with the sight of blood or blood sampling, take the sample only in the presence of another person or contact your therapist or pharmacist.

Important Information

- The sample should only be taken between Sunday and Tuesday (latest) and sent off on the same day. Please take into account any public holidays.
- We can only analyse your sample if you activate your test with the test-ID of your test-ID card! More details on this can be found in step 17.
- Unfortunately, it is not possible to analyse blood samples of volumes under 300 µl.
- Do not take a blood sample immediately after strenuous physical activity.
- If you have any difficulties performing the blood sample or if the kit is incomplete, contact our customer service department, who will treat this matter confidentially.
- This is a blood sampling kit for single use self-testing!
- Please note that in the event of vitamin B12 injections, you should wait for at least four weeks after the injection before conducting this test. Supplements taken orally will not affect your test result.

If you have any questions, please send us an e-mail (help@cerascreen.com) or visit our help website: help.cerascreen.com.

Directions for Use

- Place all contents of the test kit on a clean surface. Before you start, choose the sampling spot. TIP: the middle and ring finger of the left hand usually work best. Open the blood collection tube and place it on a flat surface. Make sure it cannot fall over.
- Wash your hands with soap and warm water. Hold your preferred hand under warm water **for at least 15 to 30 seconds** to stimulate blood circulation. Dry your hand carefully with a clean towel.
- Swing your preferred arm around in a circle 5 to 10 times, fully extended if possible, in large movements to stimulate blood circulation.
- Remove the alcohol wipe from its film and use it to disinfect the tip of the preferred finger. Wait until the finger is dry again (15 sec.).
- Take the lancet. (1) Rotate the protective cover once around its axis. (2) Carefully remove the protective cover. **Caution:** Do not use a lancet with a loose or missing protective cover.
- Place your hand on a firm surface. Place the lancet diagonally on the tip of your finger as shown in the illustration. Press the lancet firmly against the finger until it releases. **Caution:** The lancet can only be released once!
- Press the thumb of your other hand below the sampling spot and slightly pull back the skin.
- Hold your hand horizontally at hip height (for instance while standing at the edge of a table) and massage your finger from the palm of the hand down to the fingertip until a blood droplet forms.
- Before you start taking blood, use the swab to remove the first blood drop. Wait for a new large drop of blood to form. Support this process by standing up and holding your hand below heart level.
- Hold your finger diagonally at the edge of the upright blood collection tube so that the blood can run into it. Continue massaging your finger to keep the blood flow going. This process may take several minutes.
- Only finish taking the sample when the blood volume reaches the 300 µl mark. Otherwise, the laboratory can't analyse the sample. Tap the tube gently numerous times against the table so the blood runs downwards.
- After taking the blood sample, please press the swab against the location of bleeding. Once the bleeding slows down, apply the plaster to the wound.
- Close the tube by firmly pressing the cap onto the opening until it clicks into place. Shake the tube well.
- Remove the square code adhesive label from the test-ID card

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Help:

help.cerascreen.com
www.cerascreen.com

CE

CR53-V12_B12_INT_000



Version 01/2019

EN Vitamin B12 Test

The cerascreen® Vitamin B12 Test is a send-in sampling kit to test human capillary blood for suspected vitamin B12 deficiency. The test is used to determine the concentration of the protein complex holotranscobalamin (Holo-Tc), which binds to B12. The sample is analysed in vitro in the specialised cerascreen® partner diagnostic laboratory.

Your test kit contains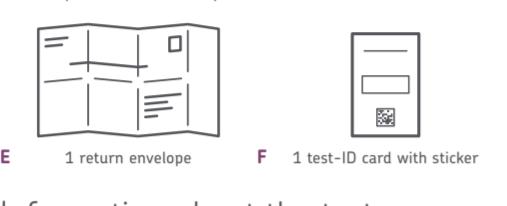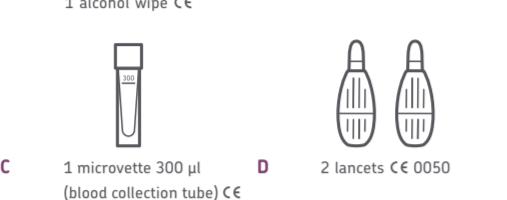

and stick it on the filled blood collection tube – NOT on the transport tube.

- Slide the filled blood collection tube into the larger transport tube without removing the absorbent felt. Seal the transport tube well.
- Place the transport tube containing the blood collection tube into the return envelope. Keep the test-ID card – you will need this to activate your test.
- To create your result report, log in via the My cerascreen® app (Android, iOS) or my.cerascreen.com. Insert your six-digit ID-number according to your test-ID card under 'activate test' and follow the instructions.
- Dispose of the used parts of the test kit in the transparent plastic bag in the household waste.

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