EN Vitamin D Test — 112

The cerascreen® Vitamin D Test is a send-in sampling kit to determine your vitamin D level. The test is used to analyse the concentration of 25-hydroxyvitamin D3 in human capillary blood. The sample is analysed in vitro in our specialised cerascreen® partner diagnostics laboratory.

Your test kit contains

A 1 alcohol wipe €€

B 1 plaster €€, 1 swab €€

C 1 dried blood spot card

D 2 lancets €€ 0050

E 1 return envelope

F 1 test-ID card with sticker

Instructions for Use

- Návod k použití • Gebrauchsanweisung
- Brugsanvisning • Instrucciones de uso • Käyttöohjeet
- Mode d'emploi • Οδηγίες χρήσης • Istruzioni per l'uso
- Gebrauchsanwizing • Bruksanvisning • Instrukcja użytkownika
- Instruções de Utilização • Användningsanvisning

Information about the test

- Read the Instructions for Use carefully before you begin and follow the listed steps.

Warnings

- Do not use this kit if you suffer from a blood-borne contagious illness such as hepatitis or HIV.

- Store this kit out of reach of children.

- The kit may only be used for adults aged 18 and over.

- If you are pregnant or are breastfeeding, use the kit only after consulting your therapist.

- If you suffer from a congenital or acquired blood-clotting disorder or are taking anticoagulant medication, use the kit only after consulting your therapist.

- Use the lancets only as described in the instructions for use.

- If you have problems with the sight of blood or blood sampling, take the sample only in the presence of another person or contact your therapist or pharmacist.

Important Information


- We can only analyse your sample if you activate your test with the test-ID or your test-ID-kort! More details on this can be found in ID 17.**

- Do not take a blood sample immediately after strenuous physical activity.**

- If you have any difficulties performing the blood sample or if the kit is incomplete, contact our customer service department via email (help@cerascreen.info), who will treat this matter confidentially.**

- This is a blood sampling kit for personal single use only!**

If you have any questions, please send us an e-mail (help@cerascreen.info) or visit our help website: help.cerascreen.com

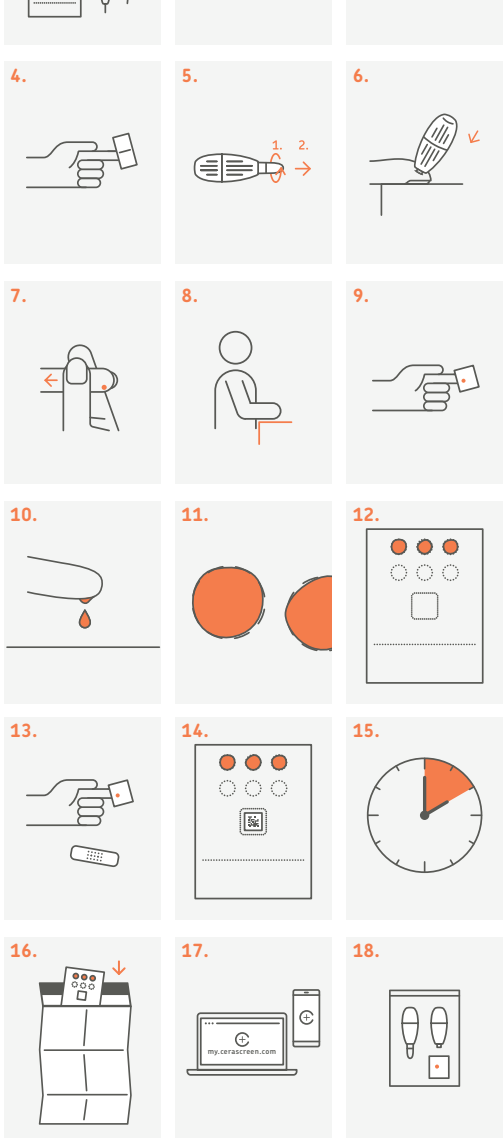1. Place all contents of the test kit on a clean surface. Before you start, choose the sampling spot. Tip: the middle and ring finger of the left hand usually work best.

2. Wash your hands with soap and warm water. Hold your preferred hand under warm water for at least 15 seconds to stimulate blood circulation. Dry your hand carefully with a clean towel.

3. Swing your preferred arm around in a circle 5 to 10 times, fully extended if possible, in large movements to stimulate blood circulation.

4. Remove the alcohol wipe from its film and use it to disinfect the tip of the preferred finger. Wait until the finger is dry again (15 sec).

5. Take the lancet. (1) Rotate the protective cover once around its axis. (2) Carefully remove the protective cover. Caution: Do not use a lancet with a loose or missing protective cover.

6. Place your hand on a firm surface. Place the lancet diagonally on the tip of your finger as shown in the illustration. Note: Do NOT use the centre of the fingertip. Only use the side of finger tip as per the puncture site to minimize discomfort. Press the lancet firmly against the finger until it releases. Caution: The lancet can only be released once!

7. Press the thumb of your other hand below the sampling spot and pull back on the skin slightly.

8. Encourage blood flow by getting up and holding your hand below the heart. Gently massage your finger from the palm of your hand to your fingertip until a drop of blood forms.

9. Before you start collecting the blood, take the swab (NOT the alcohol wipe) and wipe off the first drop of blood. Wait until a big drop has formed again.

10. Let a large blood drop fall onto the marked circle on the dried blood spot card. In case the drop doesn't fall on its own from your finger, hold the dried blood spot card carefully against the drop without touching the paper to your finger.

11. For the best evaluation of the sample, the largest possible drop of blood is required. The blood drop should cover the circle inside the marking completely.

12. Repeat this process until you have completed three circles.

13. After taking the blood, use the swab and patch to clean and cover the wound.

14. Remove the square code sticker from the test-ID card and stick it on the dried blood spot card in the area intended for the test-ID sticker.

15. Allow the dried blood spot card to dry for 2 to 3 hours at room temperature. Ensure to place it flat on a dry surface, protected from direct sunlight. Make sure the blood is completely dried before putting the dried blood spot card in the envelope.

16. Place the dried blood spot card in the return envelope with address. Keep the test-ID card – you will need this to activate your test. Close the envelope carefully and send it immediately by post free of charge.

17. If you are a new customer, please create an account at my.cerascreen.com or with the my cerascreen® app. See our FAQs on help.cerascreen.com for further instructions. To receive your result report, log in via the my cerascreen® app (Android, iOS) or on my.cerascreen.com. Insert your six-digit ID-number according to your test-ID card under 'activate test' and follow the instructions.

18. Read the instructions for use carefully before you begin and follow the listed steps.

instructions.

18. Dispose of the used parts of the test kit in the transparent plastic bag in the household waste.

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