Bovie® SharkSkin Pencil Adapter
Electrosurgical Pencil Adapter

REF SS95, SS95-1

20/Case

10 ft (3.05 m) tubing

EN


Indications For Use: Bovie® SharkSkin Pencil Adapter is an aspiration sheath that fits over an electrosurgical pencil body and leaves the tip or blade exposed. This device is considered an accessory to an electrosurgical unit (ESU). Bovie® SharkSkin Pencil Adapter removes surgical smoke during surgical procedures that uses an ESU for cutting and cauterizing tissue. This device is used in conjunction with a suction (vacuum) source.

Contraindication: This device is not to be used for microsurgery.

Precautions: For single use only. Do not reuse, reprocess, or re-sterilize. Reuse, reprocessing, or re-sterilization will compromise integrity of Bovie® SharkSkin Pencil Adapter and result in patient injury, illness, or death.

Instructions for Use: Using aseptic technique, open packaging and remove Bovie® SharkSkin Pencil Adapter onto the sterile field. Follow steps below.

Report complaints involving this device to complaint@symmetrysurgical.com or to your local distributor.

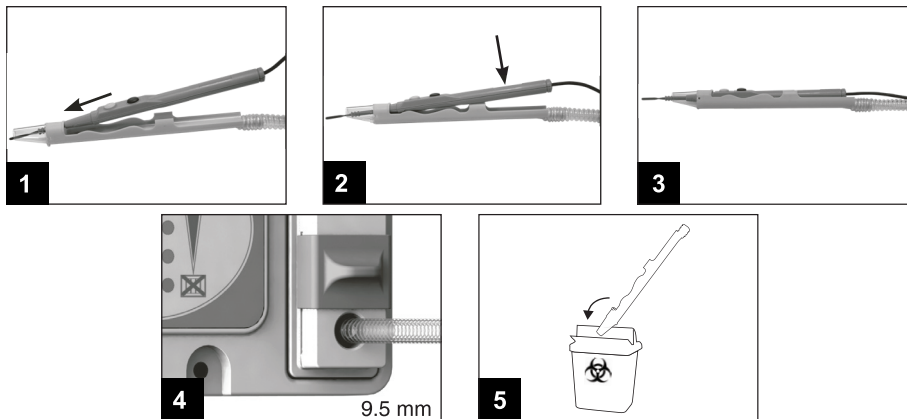		STERILE EO	Rx ONLY	AU SPONSOR: Dalcross Medical Equipment 3 Forge Pl Narellan, NSW 2567 Australia	 EC REP MDSS GmbH Schiffgraben 41 D-30175 Hannover Germany

Bovie

Bovie® products are exclusively distributed by:
 Symmetry Surgical, Inc.
 3034 Owen Drive, Antioch, TN 37013, USA
 USA & Canada: 800.251.3000 Fax: 800.342.3272
 Email: customerservice@symmetrysurgical.com
 For International Inquiries: +1 615.964.5532 Email: internationalorders@symmetrysurgical.com
 905293 Rev C 11/2020

CONMED Corporation
 525 French Road, Utica, NY 13502-5994
 Customer Service: 1-866-426-6633
 USA FAX: (727) 399-5256
 International FAX: +1 (727) 397-4540
 email: customerexperience@conmed.com
 www.conmed.com

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