

4-Wheel Folding Scooter

QUICK START GUIDE

Follow the instructions below to quickly and safely set up and start your scooter.

Issue: 2022-06 Rev2



1. Unpack the scooter, as shown above. You will have the following parts:

- 1 x scooter unit
- 1 x battery pack (pre-installed)
- 2 x keys
- 1 x remote fob
- 2 x armrest
- 2 x threaded knob for armrest
- 1 x charger and mains cable

2. Plug the mains lead into the charger, and then plug the round connector from the charger into the charging socket of the scooter.
Finally switch the charger on at the mains.



3. Insert the key into the ignition (as shown below) and switch the scooter on. Then press the button marked "A - Open" for 1 second. The scooter will then slowly beep and unfold. For safety, the unfold process will stop halfway. Press the button again to fully unfold the scooter. When fully unfolded the scooter will emit three quick beeps. Switch off the scooter.



4. Pull up the strap on the battery so that the plastic insulation strip can be removed, then refit the battery pack as shown below.



5. Flip the backrest into position.



PLEASE TURN OVER

6. Pull on the catch (arrowed) on the tiller to unlock. Then pull up the tiller until it is set at the correct height for the user. Push the catch to lock the tiller.



7. Adjust the tiller angle by pressing the lever (arrowed) whilst pulling or pushing the tiller to the correct angle. Release the lever to lock the tiller in position.



8. Insert the armrest into the receiver.
Note: The angle of the armrest may need adjusting.
Please refer to the procedure on page 19 of the manual.



9. Secure the armrest by screwing the threaded knob into the hole (arrowed).



10. Remove the protective film wrapping on each of the four tyres (not shown). *This only needs performing the first time the scooter is set up.*

11. Insert the key into the ignition and turn to start the scooter.



12. CHARGE BEFORE USE

Plug the mains lead into the charger, and then plug the round connector from the charger into the charging socket of the scooter. Finally switch the charger on at the mains. The scooter can be charged either unfolded or folded (refer to user manual for folding instructions).

