

Dual Wheel Folding Scooter Quick Start Guide

Issued 2018/10

1. Unpack the scooter and position on the ground as shown below.



2. The battery is already in position but requires connecting. Connect the two blue connectors together, and then connect the two black connectors together.



3. Press the button illuminated green for one second. The button is located at the rear right hand side of the scooter. (Note. If the button is not lit, initialise it by turning the scooter on and off with the key)



4. Push the cables in the small void behind the battery. Install the battery cover over the battery compartment (as shown above).



5. Turn the tiller adjustment knob anticlockwise to loosen. Pull up the tiller to the desired height. Turn the knob clockwise to secure.



6. Insert the key and turn clockwise to start the scooter.



**ALWAYS FULLY READ THE OWNER'S
HANDBOOK SUPPLIED WITH THE
SCOOTER BEFORE DRIVING**