

drive

DeVilbiss[®]
HEALTHCARE



Nitro Rollator

Instruction Manual

Please read these instructions carefully before assembling or using this product.
Ensure you are familiar with all of the safety and operating instructions.

CE

Parts List



- | | |
|-----------------------------------|------------------------|
| 1. Handgrip | 7. Seat |
| 2. Adjustable tubing handgrip | 8. Bag |
| 3. Rear reflector on handgrip | 9. Front wheel |
| 4. Brake lever | 10. Brake adjust screw |
| 5. Interlock button for handgrips | 11. Rear wheel |
| 6. Back Rest | |

Introduction

The Walking aid is suitable for a single user with limited mobility. The Nitro is ideal for indoor and outdoor use.

Nitro Includes:

1. Bag
2. Back Rest
3. Operating Instructions


Unfolding the Nitro

Position the Nitro on its wheels. To open, pull the handles apart (figure 1). press both pipes of the seat downwards until they interlock (figure 2).



Height Adjustment

Press the red release buttons for handgrips below, as shown in figure 3, and pull the handgrip upwards at the same time. As the handle moves, release the button. This automatically locks in the next possible height. Repeat this process until the desired height is reached.

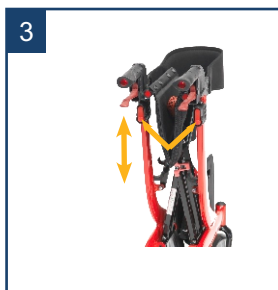

Adjustment of Brakes

To adjust the braking strength, adjust the brake by pressing the screw housing downwards (figure 9) and then turn left or right (figure 10).





Sitting

Position the Nitro on an even and solid surface. Lock the brakes (parking position). The Nitro must stand and must not slip. Sit straight up and do not lean backwards. (See figure 11).






Folding/Transport/Storage

To fold, pull seat upwards by the handle on the seat plate as shown in figure 12. The Nitro can now be folded complete by pushing the nitro together until the mechanism is locked under the seat. shown in figure 13 & 14.

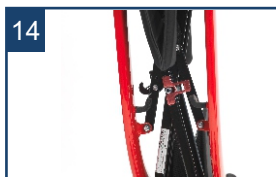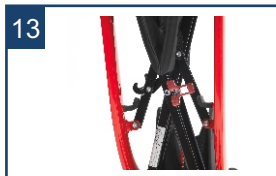

Attaching the Carry Bag

Fold the Nitro and push the sliding block from bag into guiding sleeve on the Nitro. Unfold the Nitro. (see figures 4-6).

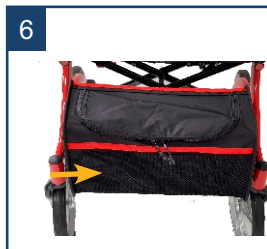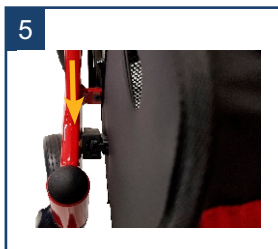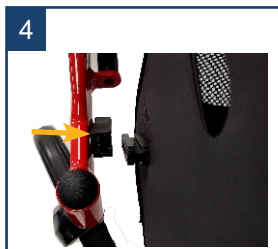

Braking/Locking

Pull both brake levers upwards to apply the brakes while the Nitro is moving (see figure 7). Always brake slowly and steady.

Press both brake levers down to apply the locks while the Nitro is stationary (see figure 8). Pull up the brake levers and release to release the locks.

The effect of the brakes can be affected by abrasion of the wheels.

Regularly test the brakes.



Safety Instructions

- Use only as a walking aid!
- Use only on even and solid surfaces.
- Maximum user weight: 135kg.
- Regularly check the screws and fasteners on the Nitro and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 5kg.
- Do not expose the product to temperatures above 40°C for an extended period of time, as this can cause the Nitro parts to overheat and burn the user.
- Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
- Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.
- Replace tyres when significantly worn or damaged.
- Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

Cleaning / Maintenance

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

Specification

REF: RTL10266 / RTL 10266BLK

Overall Length: 70.5cm (27.75")
Overall Width: 58.5cm (23")
Handle (Highest):97cm (38.25")
Handle (Lowest): 85cm (33.5")
Seat Depth: 20cm (8")
Seat Width: 46cm (18")
Seat Height: 52cm (20.5")

Wheel Diameter:

Rear: 20cm (8")
Front: 25cm (10")

Product Weight: 8kg (17.5lb)
Weight Limit: 135kg (21 stone)

REF: RTL10266-H

Overall Length: 70.5cm (27.75")
Overall Width: 58.5cm (23")
Handle (Highest):79cm (31")
Handle (Lowest): 71cm (28")
Seat Depth: 20cm (8")
Seat Width: 46cm (18")
Seat Height: 46cm (18")

Wheel Diameter:

Rear: 20cm (8")
Front: 25cm (10")

Product Weight: 8kg (17.5lb)
Weight Limit: 135kg (21 stone)

REF: 10266HD

Overall Length: 70.5cm (27.75")
Overall Width: 58.5cm (23")
Handle (Highest):97cm (38.25")
Handle (Lowest): 85cm (33.5")
Seat Depth: 20cm (8")
Seat Width: 46cm (18")
Seat Height: 52cm (20.5")

Wheel Diameter:

Rear: 20cm (8")
Front: 25cm (10")

Product Weight: 8kg (17.5lb)
Weight Limit: 200kg (32 stone)

REF: RTL10266-T

Overall Length: 70.5cm (27.75")
Overall Width: 58.5cm (23")
Handle (Highest):104cm (41")
Handle (Lowest): 91cm (36")
Seat Depth: 20cm (8")
Seat Width: 46cm (18")
Seat Height: 58cm (23")

Wheel Diameter:

Rear: 20cm (8")
Front: 25cm (10")

Product Weight: 8kg (17.5lb)
Weight Limit: 135kg (21 stone)

Warranty

Your Drive product is warranted to be free of defects in materials and workmanship for two years from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.

This does not affect your statutory rights.



Drive DeVilbiss Healthcare Ltd,
1 Heathfield Lane, Birkenshaw,
West Yorkshire, GB BD11 2HW
www.drivedevilbiss.co.uk