

IMPORTANT SAFEGUARDS

Always consult with your doctor before using a Nasal Irrigator as everyone's medical situation is different.

ONLY USE THE FOLLOWING TYPES OF WATER:

- Distilled or sterile water, which you can buy in stores. The label will state "distilled" or "sterile."
- Boiled and cooled tap water — boiled for 3 to 5 minutes, then cooled until it is lukewarm. Previously boiled water can be stored in a clean, closed container for use within 24 hours.
- Water passed through a filter designed to trap potentially infectious organisms.
- Temperature of water should be lukewarm or room temperature. Hot or cold water should not be used.

TAP WATER SHOULD NEVER BE USED

READ ALL INSTRUCTIONS BEFORE USING

When using this electrical product, the basic safety precautions below should always be followed.

DANGERS

- To reduce the risk of electrocution:
- Do not handle charger with wet hands.
- Do not immerse in water or other liquid.
- Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub, shower stall or sink.
- Do not reach for a product that is plugged in and has fallen into water: Unplug immediately.
- Check the charger cord for damage before the first use and during the life of the product.
- To reduce the risk of burns, electric shock, fire, serious injury or damaging the Nasal Irrigator:
- Do not plug this device into a voltage system that is different from the voltage system specified on the device or charger.
- Do not use this product if it has a broken cord or plug.


- Do not direct water flow into your ears or body parts other than your nose. Refer to operating instructions for correct usage.
- Use this product only as indicated in these instructions or suggested by your medical professional.
- Keep the charger and cord away from heated surfaces. For household use only. Do not use outdoors.
- Please keep out of Children's reach. Children should not be allowed to use this device unsupervised.
- If the unit is not working properly please call or email us for help (contact info at end of manual).

NASAL IRRIGATOR PARTS

Configuration


The nasal irrigator is composed of following four parts:

- ① Nasal irrigator unit
- ② Water tank
- ③ Nozzle
- ④ Charging cable

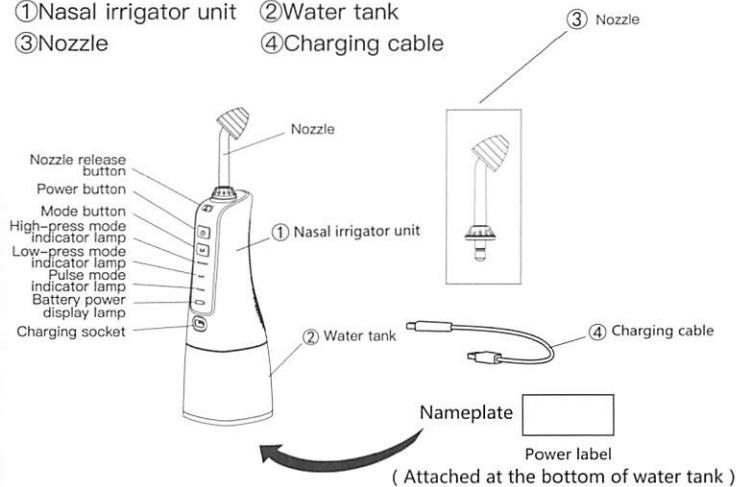

INSTRUCTIONS FOR USE

IMPORTANT! Please fully charge your Nasal Irrigator before first use by plugging the USB cable into the unit and the other end into a power source. The product cannot be used while charging. (When the Irrigator is being charged, the indicator light on the body will flash; when the Irrigator is fully charged, the indicator light will stop flashing and stay on. If the battery level indicator light flashes when the Irrigator is in service, it means that the battery level is low, and charging is required.

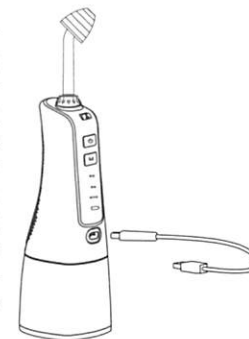

Installation of nozzle



Insert the nozzle into the middle hole of the nozzle knob until a click is heard.

If changing or removing the jet tip, please press on the disassembly button on the jet tip handle before removing the jet tip.




**Pour water and saline solution into water tank
ONLY USE THE FOLLOWING TYPES OF WATER:**

Distilled or sterile water, which you can buy in stores. The label will state “distilled” or “sterile.”

– Boiled and cooled tap water — boiled for 3 to 5 minutes, then cooled until it is lukewarm. Previously boiled water can be stored in a clean, closed container for use within 24 hours.

– Water passed through a filter designed to trap potentially infectious organisms.

– Temperature of water should be lukewarm (no greater than 100 degrees Fahrenheit) or room temperature. Hot or cold water should not be used.

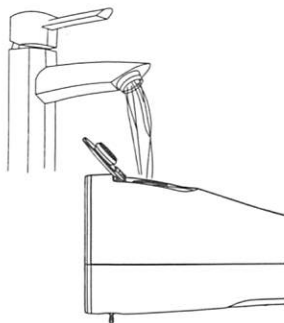

TAP WATER SHOULD NEVER BE USED

Mix Saline Solution with the water.

(4) Gently shake the Irrigator for a couple of seconds to mix Saline Solution with the water in the tank.

(5) Set Irrigator to lowest setting for first use.

(6) Place your tongue against the roof of your mouth to help with drainage out of your nose instead of your mouth.

(7) Place the Irrigator tip firmly but gently into one nostril a lean over your sink while looking at the drain.

(8) Turn on the Irrigator: water will enter through one nostril and exit the other nostril. Use about half of the water supply before stopping the water flow and switching nostrils.

(9) After you have finished irrigating, gently blow your nose. In order to prevent pressure on your eardrums, do not block your nasal passages completely when blowing.

WARNING: Do not use if your nasal passage is completely blocked. Do not use if you have an ear infection. Stop use and contact your physician if you experience high pressure in your ears while using.


AFTER USING THE IRRIGATOR

Turn off the Irrigator, open the water tank cover, and direct the opening downwards as shown to drain out the residual water in the tank. Turn on the apparatus and discharge the residual water in the Irrigator, and when no more water flows out, turn off the apparatus.



Removal of water tank

As shown in Fig. 1, hold the body of the product with one hand and the water tank with the other hand. Rotate counter-clockwise downwards as shown in Fig. 2 to remove the water tank.

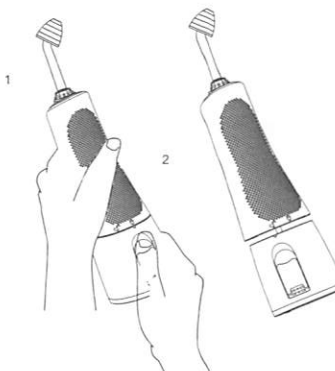

Troubleshooting

Problem	Possible Cause	Solution
It does not work	You've just purchased the product or it has been left unused for more than 3 months.	Charge the unit for at least 8 hours.
It only works for a few minutes even after charging	The battery has reached the end of its life cycle.	Contact service center for support.
	Charging time was insufficient.	Charge the unit for at least 8 hours.
Water pressure is very low	There is dirt stuck in the nozzle	Replace with a new nozzle
	The nozzle is deformed	
	There is dirt stuck in the filter	Clean the filter
Water does not come out	The tank is empty	Fill up tank
	The appliance is tilted too much while in use	Use with the appliance in a vertical state

Need Help? Have a product question? Feedback?
Contact us:
Info@ToiletTreeProducts.com
or Toll Free 1-888-251-0985

TOILETTREE WARRANTY

ToiletTree Products provides 30 day, no risk, unconditional warranty. We guarantee you will be completely satisfied with your purchase, simply return it to us and we will gladly refund your money, in full, no strings attached.

ToiletTree warrants this product to be free from all defects in material and workmanship for a period of 30 days from the date of purchase. This warranty does not apply to normal wear and tear or neglect. Any defective item will be replaced, at no charge to you, if returned with proof of purchase. ToiletTree Products Replacement, 41 Orchard Street, New York, NY 10014. No authorized ToiletTree dealer shall be liable for claims or damages unless otherwise provided by law.