DISCOVER POLARIS
WITH ITS **NEW ARM**

WHY POLARIS ?

LED TECHNOLOGY

SCIALYTIC EFFECT

LIGHT INTENSITY

ADJUSTMENT UP TO

50.000 LUX

COLOUR TEMPERATURE

ADJUSTMENT


100% ALUMINIUM

COLOUR TEMPERATURE ADJUSTMENT

CHOOSE

THE RIGHT COLOUR TEMPERATURE FOR EVERY TREATMENT.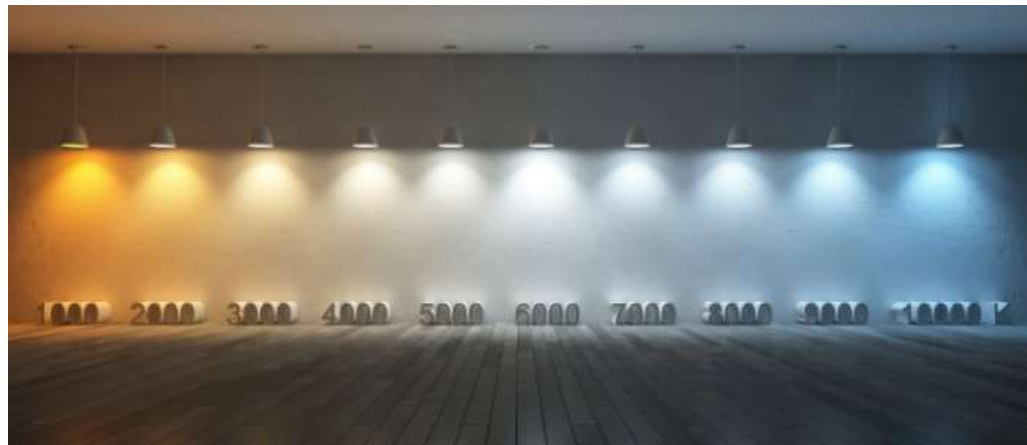

- COLD LIGHT (OPTIMISES THE COLOUR CONTRAST ON SOFT TISSUES) --> SURGERY
- 5.500 K (SUN LIGHT) --> COLOUR MATCHING

LIGHT INTENSITY ADJUSTMENT

HOW MUCH LIGHT ?

35.000 LUX

**BEST DETAIL PERCEPTION
PREVENTING EYESTRAIN**

50.000 LUX

**FOR SPECIFIC NEEDS DURING
SHORT PERIODS OF TIME**

SCIALYTIC EFFECT

**OVERLAPPING LIGHT RAYS
AVOID SHADOWS PROJECTION
ON THE OPERATING FIELD**

RISK FROM WRONG LIGHTING


INADEQUATE COLOUR TEMPERATURE

- POSSIBLE MISTAKES IN
- COLOUR MATCHING
- PROCEDURES
- MAKES IT DIFFICULT TO
- DISTINGUISH SHADES OF
- RED OF SOFT TISSUES

LACK OF SCIALYTIC EFFECT

- BLURRED PERCEPTION
- OF THE OPERATING FIELD
- INNATURAL POSTURE
- WITH POSSIBLE
- CERVICAL AND LUMBAR
- DISEASES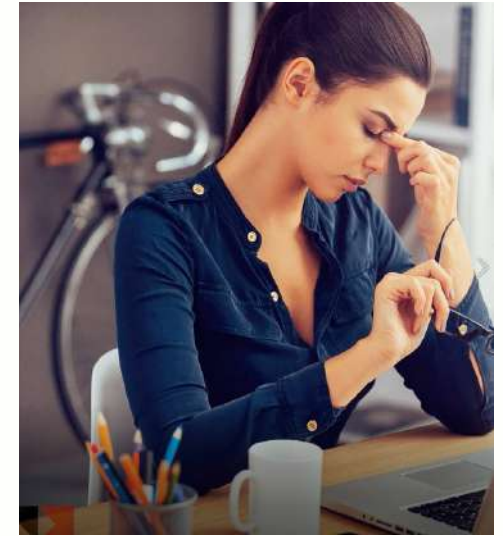

WRONG LIGHT INTENSITY

- BLURRED SIGHT: DETAILS
- ARE NOT CLEAR

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ANTIPOLYMERIZATION FILTER; REDUCTION OF COMPOUND CURING SPEED. EASY TO INSTALL AND CLEAN.

MIRROR: ENHANCED COMMUNICATION WITH THE PATIENT

OPTIONAL

EASY HYGIENE

*** EASY CLEANABLE FRONT COVER**

*** PARABOLES ARE PROTECTED AND
NEED NO CLEANING**

*** REMOVABLE HANDLE COVERS**
**- AUTOCLAVABLE 15 MINUTES AT
135° (ISO 11134)-**
**- 2 PAIRS PROVIDED WITH EVERY
LIGHT**


NEW ARM

- MODERN DESIGN
- ENHANCED MOTION
- 100% ALUMINIUM

FLOW MOTION MOVEMENT

OPTIMIZED MOVEMENT
THROUGH **RE-ENGINEERED**
MECHANICAL **SYSTEM**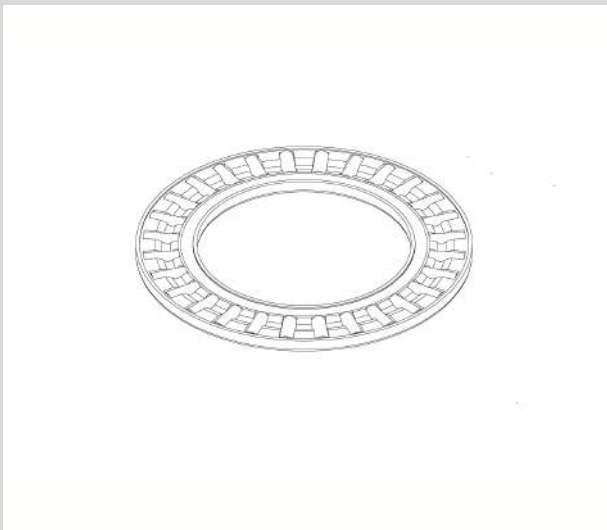

ENHANCED ROTATION
SYSTEM BETWEEN MIDDLE
JOINT AND MOBILE ARM,
THANKS TO ROLLER
BEARINGS.

EASY TO INSTALL & ADJUST

- NEW **ASSEMBLY PROCEDURE**: FIRST FIX THE ARM TO THE HEAD, THEN CONNECT THE WIRES
- EASY **ARM SPRING ADJUSTMENT**
- **CENTRAL JOINT FRICTION** ADJUSTMENT
- 2 MOV FRICTION ADJUSTMENT
- 3 MOV FRICTION ADJUSTMENT
- **SMOOTH SWITCH** FROM ROOF MOUNT TO UNIT MOUNT
- **DIFFERENT SOLUTIONS**: UNIT MOUNT, CEILING MOUNT, WALL MOUNT, TROLLEY MOUNT, ON SURGICAL CART.
- **SUITABLE TO DIFFERENT UNITS**, THANKS TO A WIDE RANGE OF PIVOTS.

VERSATILITY

- MECHANICAL
- FEATURES

PIVOT OF DIFFERENT
MEASURE AND
CUSTOMIZED
ADAPTORS

- ELECTRICAL
- FEATURES


VARIABLE TENSION
FROM 12 TO 24 V

- COLOUR
- FEATURES

LIGHT CAN BE
PAINTED IN EVERY
REQUESTED RAL
COLOUR

ARM MEASURES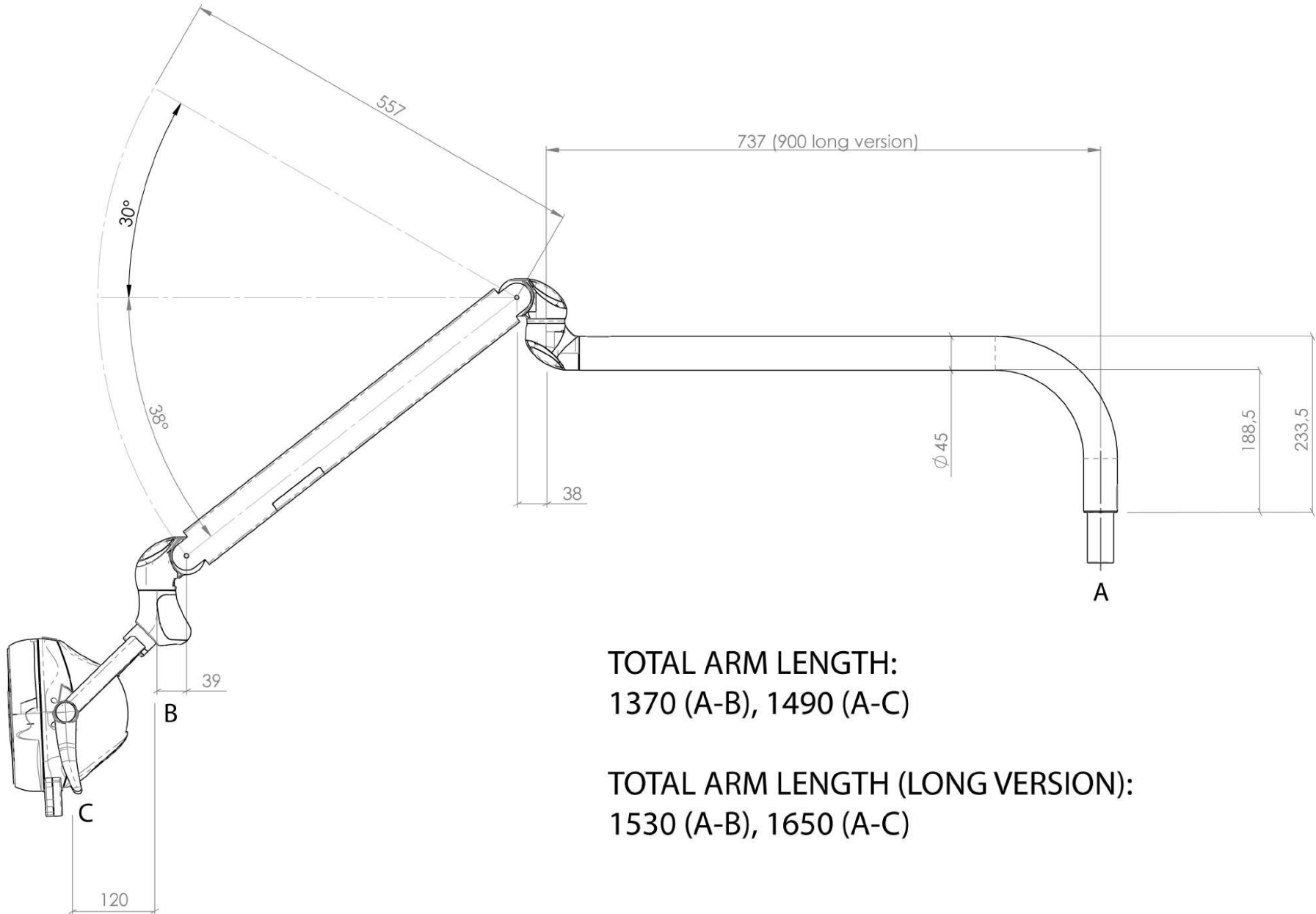


TOTAL ARM LENGTH:
1370 (A-B), 1490 (A-C)


TOTAL ARM LENGTH (LONG VERSION):
1530 (A-B), 1650 (A-C)

RECOMMENDED DISTANCES

RECOMMENDED DISTANCES
CEILING MOUNT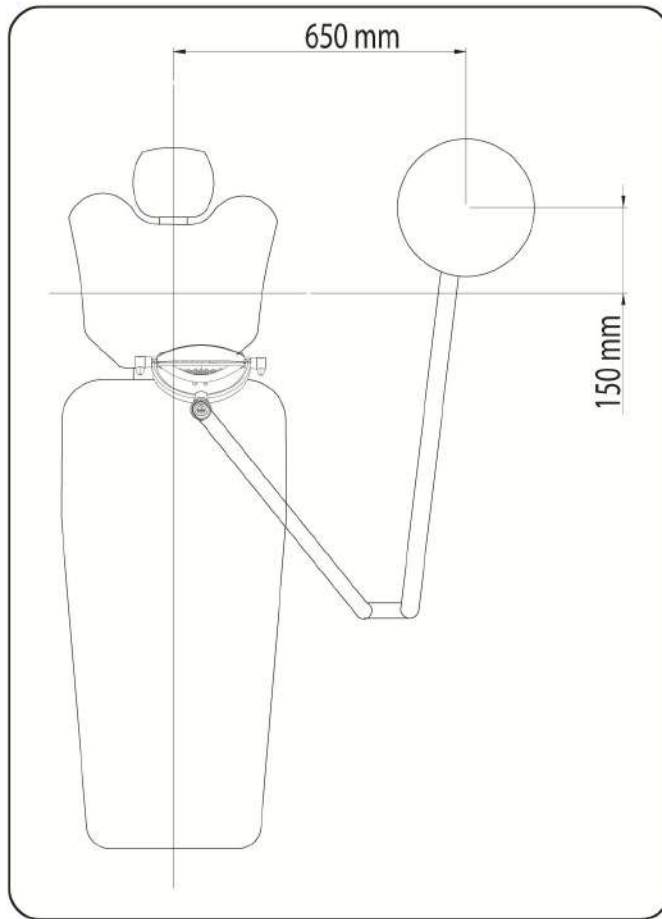

RECOMMENDED DISTANCES
WALL MOUNT

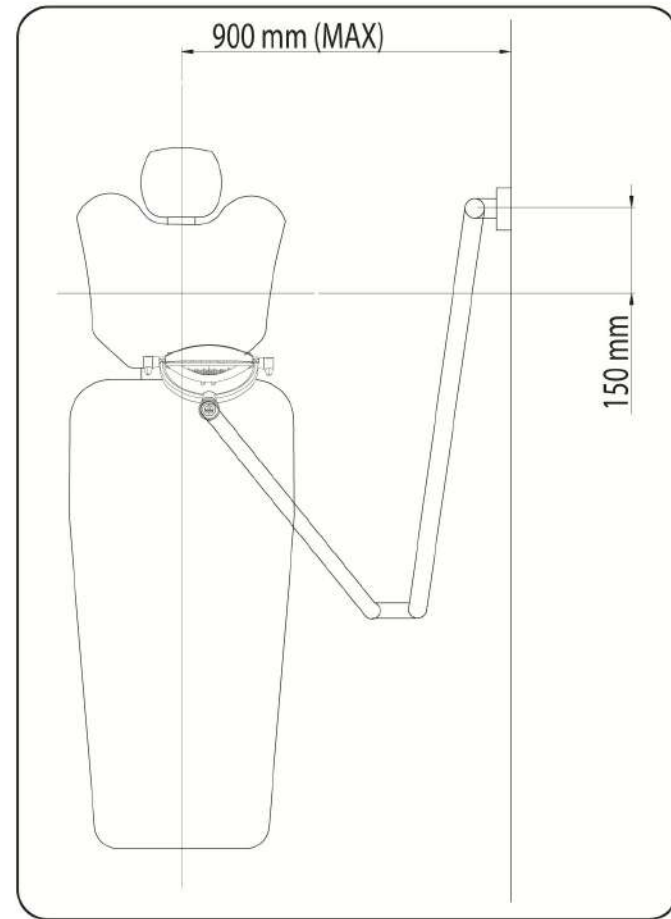TECHNICAL FEATURES

POLARIS

- HEAD NET WEIGHT: 1.900 KG
- ARM NET WEIGHT : 5.150 KG
- LIGHT PIVOT: ϕ 35 MM (OTHER ϕ ON REQUEST)
- SUPPLY VOLTAGE: 12 - 24 VAC 50-60 HZ
- MAXIMUM POWER ABSORBED: 24 VA
- MAX. ABSORBED CURRENT: 1.5 A
- ILLUMINATED SPOT DIMENSION: 80 X 200 MM
- LIGHT INTENSITY
- (ADJUSTABLE): FROM 8.000 TO 50.000 LUX
- COLOUR TEMPERATURE (ADJUSTABLE): FROM 4200 K
- TO 6000 K
- OPERATING LENGTH: 70 CM
- LIGHT SOURCE: 10 LEDS
- GUARANTEE: 24 MONTHS

CORE WHITE WHITENING LIGHT

- WEIGHT: 800 G
- POWER SUPPLY: 17-24 VAC
- MAXIMUM POWER CONSUMPTION: 30 VA
- CURRENT CONSUMPTION: 1.5 A
- LIGHT PATTERN: 80X20 MM
- LIGHT SOURCE: 4 LED (3.5 W CAD.)
- EMISSION SPECTRUM: 420-480 NM
- MAXIMUM LIGHT POWER EMITTED:
- 2000MW
- MAXIMUM DISTANCE FROM PATIENT: 10 CM

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LIGHTS THAT
GUARANTEE OPTIMUM VISUAL
COMFORT, COLOUR PERCEPTION,
AND PREVENT EYE-STRAIN.

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