


# What's Inside

The Sleep8 Device

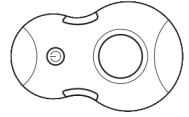

The Bag



The Charger



Have questions or need replacements?  
Visit [MySleep8.com](http://MySleep8.com) or call 888-549-9799.

## Indications of Use, Cautions, & Specifications

---

Electrical Characteristics:

Environmental Characteristics:

sleep8™

# Quick Start Guide

For sanitizing all PAP devices and accessories. CPAP, APAP, Bi-PAP




# 3 Easy Steps

1

## Connect CPAP tubing inside the bag. Put in mask & chamber\*. Zip it closed.


Unzip the bag and find the round valve inside. Detach the tubing from your CPAP machine and attach it to this valve. Put all CPAP tubing, mask, and dry water chamber inside the bag, then zip the bag closed.

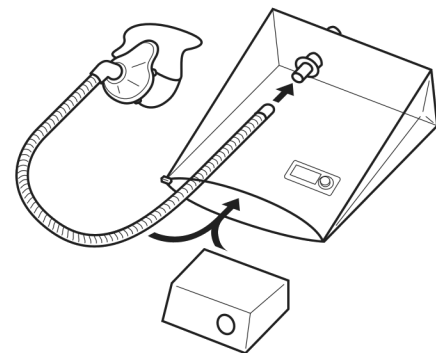

2

## Connect Sleep8 outside the bag.


Slide your Sleep8's triangular port onto the triangular valve on the outside of the bag. It should be a snug fit. The device will only work once there's a secure connection.

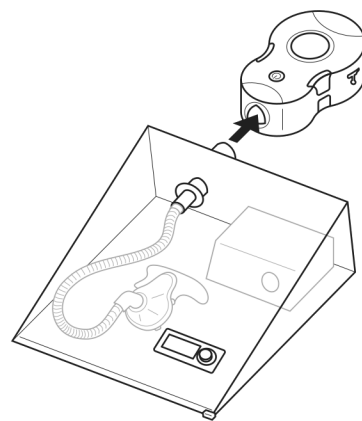

3

## Press start & relax.

Press the power button until you hear a beep and see the Sleep8 screen light up. The screen will count down until the sanitization is complete. It's normal to hear the fan oscillate and pause for short periods. When done, your Sleep8 will automatically turn off. We recommend leaving the bag closed for 1 hour after the cleaning is completed. Then unzip the bag to find your sanitized CPAP ready to use.

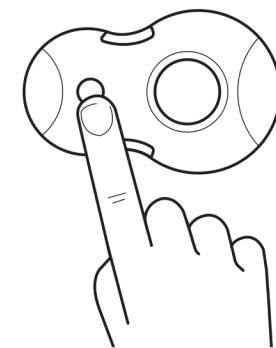

## Charging your Sleep8 device.

When you notice the battery is low, plug it in using the Sleep8 charger included. You cannot overcharge your Sleep8.

When the device is plugged in using the Sleep8 charger, the device will beep once, the screen will be dim, and the battery icon will show the charging progress. Once the icon shows 4 full bars, your Sleep8 is fully charged and ready for action.



Visit [MySleep8.com](https://www.mysleep8.com) for a video tutorial and other resources.

# The Screen

## Battery

The battery icon indicates the power level of the device. The battery should last about 3 uses per charge. When this icon is low, be sure to charge your Sleep8.

## Fan

The fan icon will pulse during the sanitization process.

## O<sub>3</sub>


The O<sub>3</sub> icon will display to indicate the bag is correctly connected and when sanitization is in process.

## Timer

The 2-digit countdown indicates how many minutes remain. Once fully sanitized, the device will beep 5 times, then "OK" will display in place of the numbers.

## X

A red "X" will display next to O<sub>3</sub> when the bag is not properly connected.

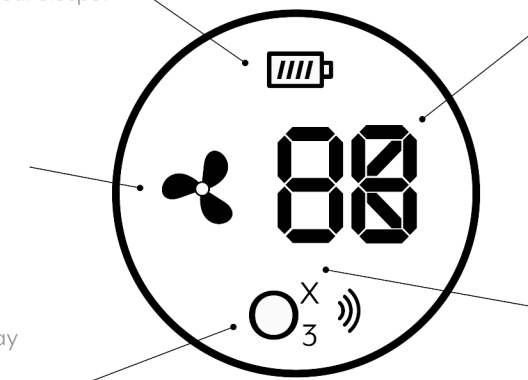

## Alarm Cues

**X** **Device not connected properly**  
"X" will display next to "O<sub>3</sub>" and device will beep 30 for seconds. Press the power button to cancel the alarm.

**E1** **Low battery**  
"E1" will pulse and device will beep. Recharge your Sleep8. *Note: The device will not function while being recharged.*

**E2** **Fan failure**  
"E2" will pulse and device will beep. Contact customer support at [MySleep8.com](https://www.mysleep8.com) for assistance.

\*If applicable